


Back Canes

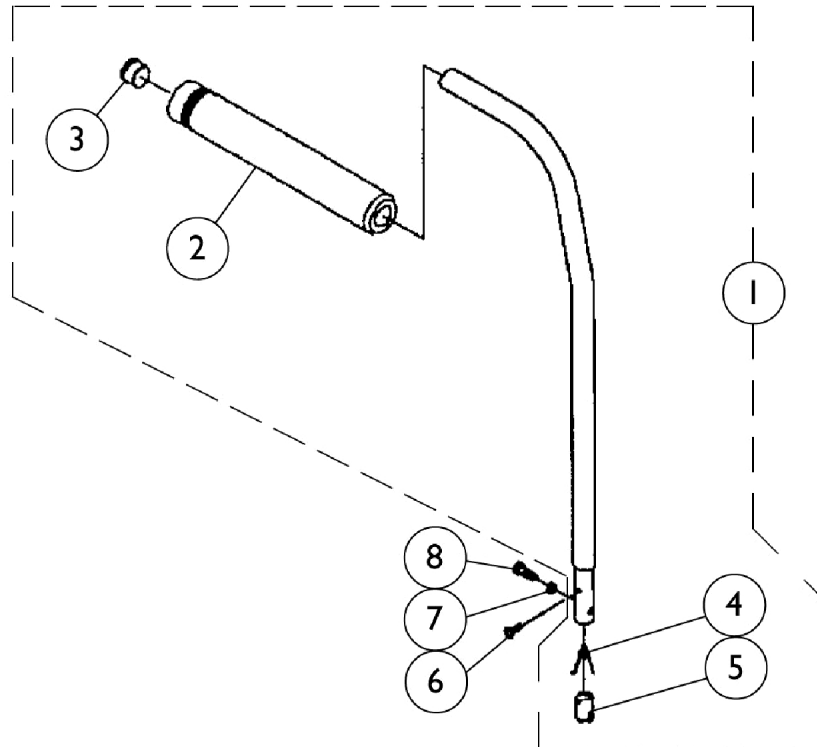

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B,B1	See Chart	#1 Cane w/Handgrip, Threaded Insert, & Spring - Action Ranger & Multi-Function Frames	1	1
1		See Chart	#1 Cane w/Handgrip, Threaded Insert, and Spring - Arrow Action	1	1
2	A,C	1024761-NLA	NLA - Grip, Cane, Black	1	1
2	D	1032246	Grip, Cane	1	1
3		1027095	Plug Button, Black (7/8")	1	1
4		40188X002	Spring, Release	1	1
5		1005834	Insert, Threaded	1	1
6		1028484	Screw, Phillips Pan Head Tap (#10-32 x 1/2")	1	1
7		1026108	Lockwasher (1/4 x 15/32 x 1/64")	1	1
8		1007219	Screw, Socket Head (1/4-20 x 5/8")	1	1
NOTE: A - Item number 3 is only used with Cane Grip part number 1024761. B - Hemi's and L-2's (Sectional Backs) are manufactured with a 1" shorter back height. B1 - Example: A 2" Lower Chair with a 16" back height will actually have a 15" high canes and back upholstery. The end result (16") will be the same when the back upholstery is put back onto the chairs canes. C - Before 7/92 D - After 6/92					

Item #1 Cane w/Handgrip, Threaded Insert, and Spring

Arrow Action

12" Height	14" Height	15" Height	16" Height	17" Height	18" Height	19" Height	20" Height
1031080-NLA	1031081-NLA	1031082-NLA	1031083-NLA	1031084-NLA	1031085-NLA	1031086-NLA	1031087-NLA

Back Canes

NOTE: For All Widths

Item #1 Cane w/ Handgrip, Threaded Insert, & Spring

Action Ranger & Multi-Function Frames

12" Height	14" Height	15" Height	16" Height	17" Height	18" Height	19" Height	20" Height
1031049-NLA	1031050-NLA	1031051-NLA	1031052-NLA	1031053-NLA	1031054-NLA	1031055-NLA	1031056-NLA
NOTE: For All Widths							