
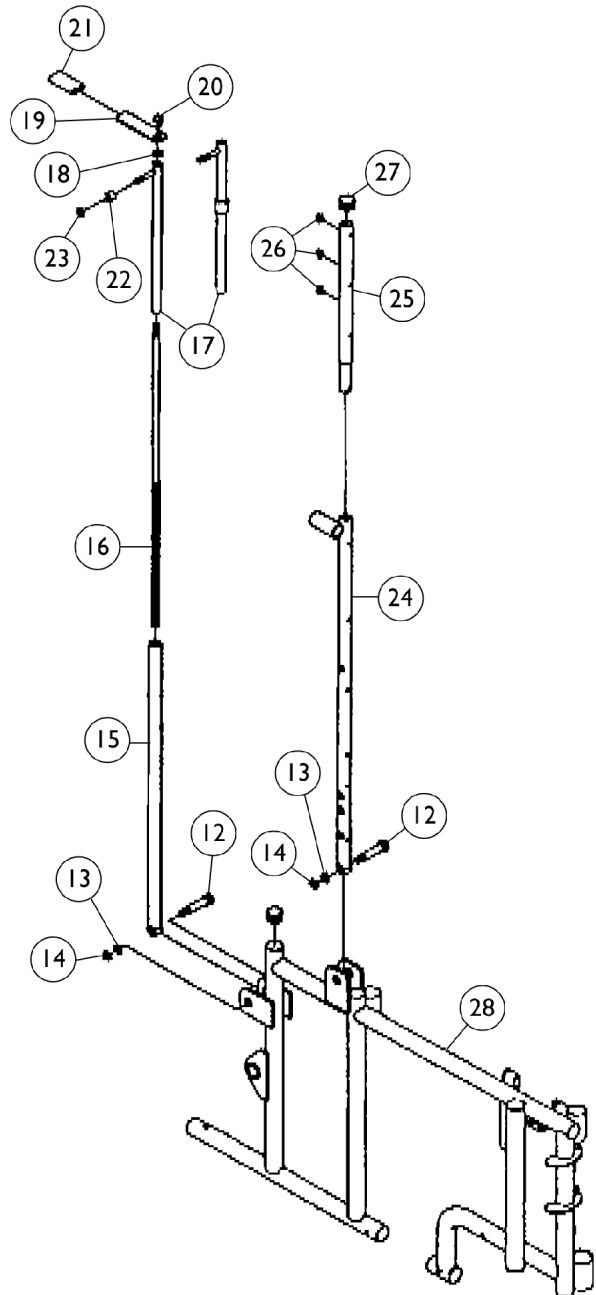


Manual Back Reclining Mechanism

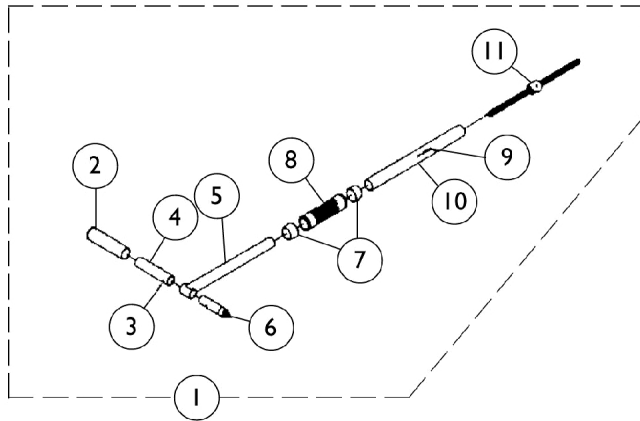

Manual Back Reclining Mechanism

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1029097-NLA	NLA - Push Handle Assembly, Adjustable, Black	1	1
1	F	1034868-NLA	NLA - Push Handle Assembly, Adjustable, Black	1	1
2		1029096-NLA	NLA - Grip, Hand, Black	1	2
3		40197X000-NLA	NLA - Pin, Roll (3/16 x 7/8")	1	2
4		1029087-NLA	NLA - Handle, Black	1	2
5	E	1029090-NLA	NLA - Tube, Adjuster, Black - Right	1	1
5	F	1029093-NLA	NLA - Tube, Adjuster, Black - Left	1	1
5	E	1029274-NLA	NLA - Tube, Adjuster, Black - Right	1	1
5	F	1029275-NLA	NLA - Tube, Adjuster, Black - Left	1	1
6		47340X006-NLA	NLA - Handle Tie Rod	1	1
7		15680X013	Anti-Rattle (1")	1	2
8		1029099-NLA	NLA - Grip, Hand, Center	1	1
9		40187X014-NLA	NLA - Pin, Roll (3/16 x 1")	1	1
10		1029095-NLA	NLA - Cover, Strut	1	1
11		04419X011-NLA	NLA - Rod, Adjusting	1	1
12		1029134	Screw, Shoulder Hex Socket Head w/ Patch (1/2" x 1-1/4", 3/8-16)	1	1
13		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
14		1028486	Locknut, Low Profile (3/8-16)	1	2
15		1029098-NLA	NLA - Tube, Rear, Black	1	1
16		1029128-NLA	NLA - Rod, Threaded Locking	1	1
17		1029127-NLA	NLA - Tube, Slide, Black, Full Recline	1	1
17		1029139-NLA	NLA - Tube, Slide, Black, Semi-Recline	1	1
18		7930331-NLA	NLA - Locknut (7/16-20")	25	1
19		1029133-NLA	NLA - Handle, Locking, Black	1	1
20		1029135-NLA	NLA - Nut, Acorn (7/16-20")	1	1
21		47376X025-NLA	NLA - Tip, Plastic, Black	1	1
22		1026160	Spacer (5/16 x 7/16 x 3/8")	1	1
23		7930330-NLA	NLA - Locknut (5/16-24)	25	1
24		1029130-NLA	NLA - Tube, Recliner Back, Black - Right	1	1
24		1029131-NLA	NLA - Tube, Recliner Back, Black - Left	1	1
25	B,C	1037481-NLA	NLA - Headrest Extension Tube Assembly, Black	1	1
25	B,D	1037482-NLA	NLA - Tube, Headrest Extension (Black) w/ Heat Shrink Tape	1	1
26		47447X009-NLA	NLA - Screw, Phillips (#8 x 3/8")	1	1
27		1027095	Plug Button, Black (7/8")	1	3
28	A	See Page	See Footnote	1	1
NOTE: A - See Frames with Hardware page. B - Includes items 26 and 27 C - After 9/92 D - Before 10/92 E - 14" to 18" Wide F - 19" to 24" Wide					