


Seat Upholstery for Multi-Function Frame

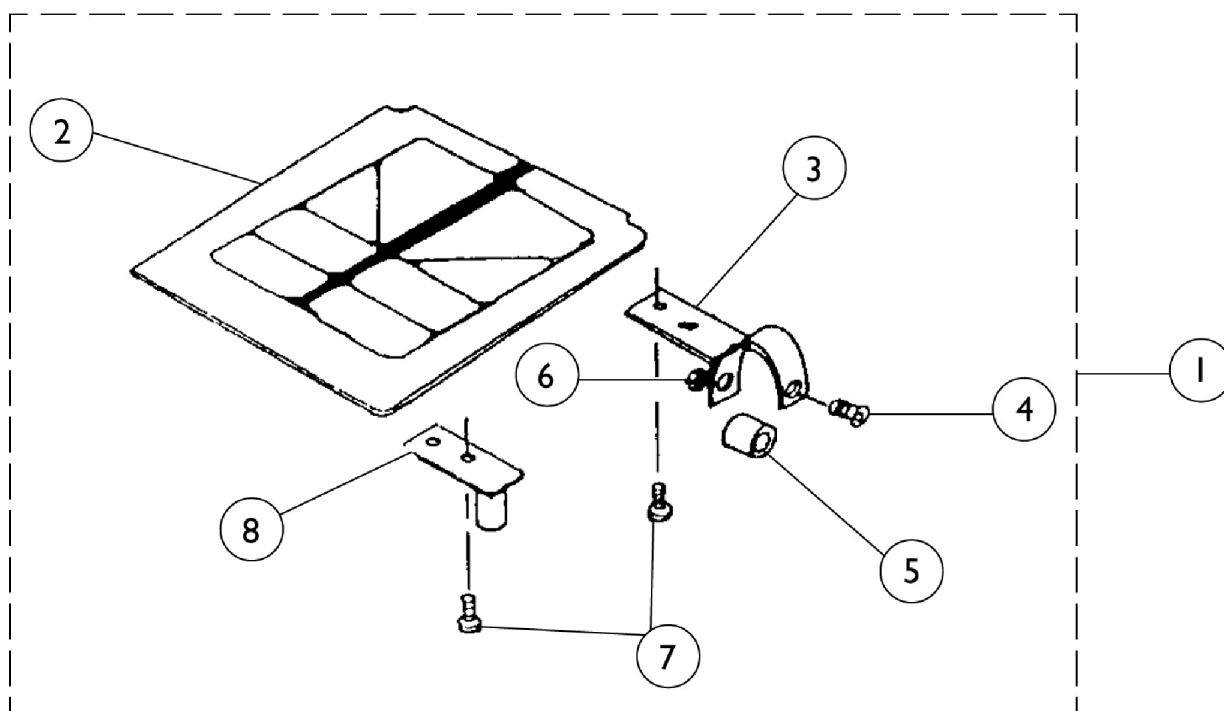

Seat Upholstery for Multi-Function Frame

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	8881013857-NL A	NLA - Solid Seat w/ Hardware	1	1
1	A,C	8881013931-NL A	NLA - Solid Seat w/ Hardware	1	1
1	A,D	8881013995-NL A	NLA - Solid Seat w/ Hardware	1	1
1	A,E	8881013933-NL A	NLA - Solid Seat w/ Hardware	1	1
2	A,B	8881012998-NL A	NLA - Seat, Solid	1	1
2	A,C	8881013886-NL A	NLA - Seat, Solid	1	1
2	A,D	8881013853-NL A	NLA - Seat, Solid	1	1
2	A,E	8881013892-NL A	NLA - Seat, Solid	1	1
3		1009768-NLA	NLA - Clamp, Seat, Rear	1	2
4		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	2
5		47324X011	Bushing, Pivot (5/16 x 7/8 x 3/4")	1	2
6		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	2
7		18858X007-NLA	NLA - Screw, Phillips Pan Head (#10-32X3/4")	1	4
8		1009886-NLA	NLA - Guide, Front Seat	1	2
NOTE: A - Specify color when ordering, see Frame Finishes, Touch-Up Paints, Upholstery Options and Accent Packages page. B - 16" Wide x 16" Deep C - 16" Wide x 17" Deep D - 18" Wide x 16" Deep E - 18" Wide x 17" Deep					