



SuperPole / SuperPole Heavy Duty / SuperPole Ultra / SuperPole Stainless Steel


PARTS DIAGRAM: SUPERPOLE
/SUPERPOLE HEAVY DUTY

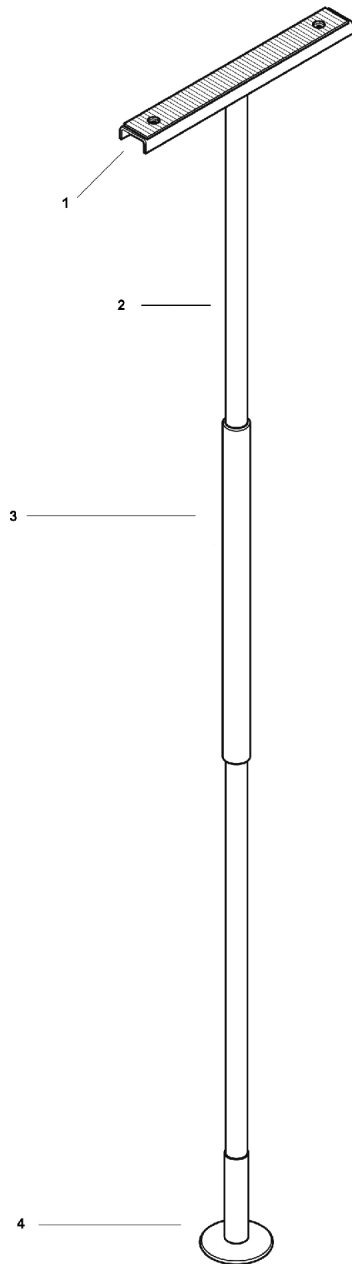

PARTS DIAGRAM: POLE / CEILING ANCHOR PLATE
ASSEMBLY

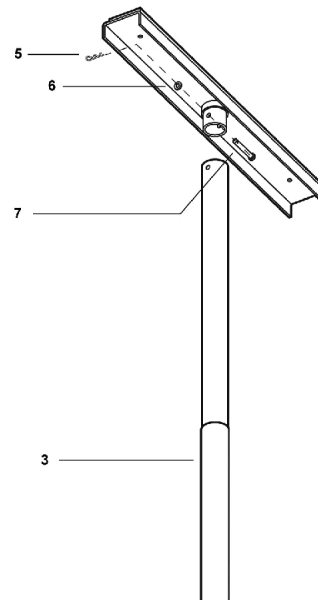

PARTS DIAGRAM: POLE / EXTENSION SCREW /
BASE PLATE ASSEMBLY

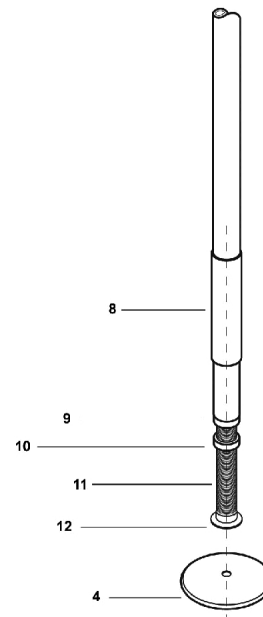

SuperPole / SuperPole Heavy Duty / SuperPole Ultra / SuperPole Stainless Steel

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	SP403	Ceiling Anchor Plate Assembly	1	1
1		SP403HDU	Top Plate, Ultra	1	1
1		SP403HD	Ceiling Anchor Plate Assembly, Heavy Duty	1	1
1		SP210	Top Plate, Stainless Steel	1	1
2		SP-P	Pole Assembly	1	1
2		SP-PHD	Pole Assembly, Heavy Duty	1	1
2		SP-PU	Pole Assembly, Ultra	1	1
2		SP-PHD-SS	Pole, Stainless Steel	1	1
3		GR-101	Grip, 20"	1	1
4		SP402	Base Plate Assembly	1	1
4		SP212-NLA	NLA - Base Plate Assembly, Stainless Steel	1	1
5		HW323	Hitch Pin	1	1
6		HW318	Washer, Nylon	1	1
7		HW302	Clevis Pin	1	1
8		SP401	Cover Sleeve, Plastic	1	1
8		SP207	Cover, Sleeve, Stainless Steel	1	1
9		SP105	Threaded Collar	1	1
9		SP105HD	Threaded Collar, Heavy Duty	1	1
9		SP105HDU	Threaded Collar, Ultra	1	1
10		SP172	Jam Nut	1	1
11		SP107	Extension Screw	1	1
12		SP145	Washer, Plastic	1	1
NOTE: A - Not available on the Stainless Steel SuperPole.					