
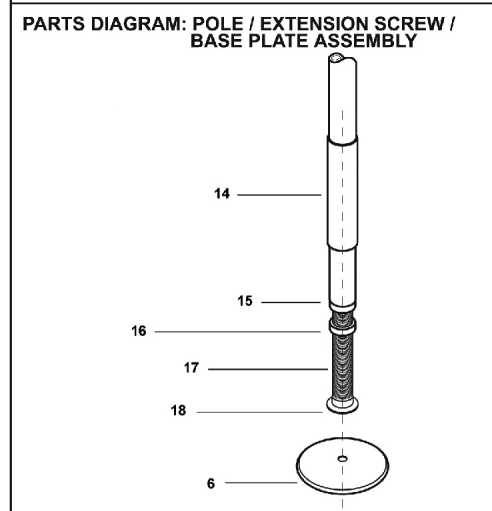
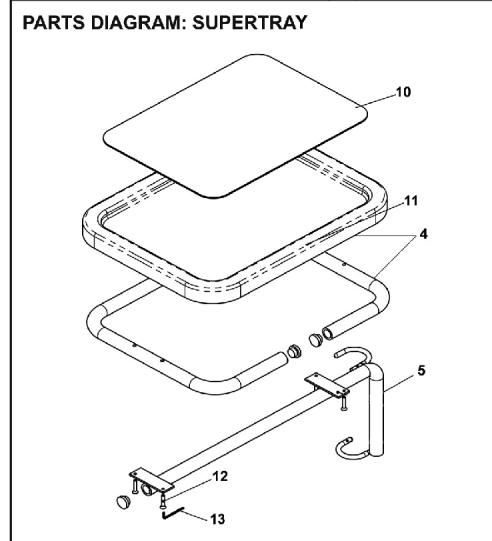
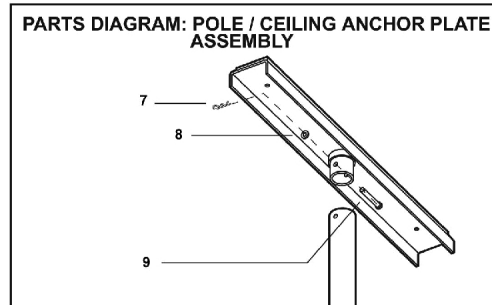


SuperPole with SuperTray / SuperPole with SuperTray Heavy Duty

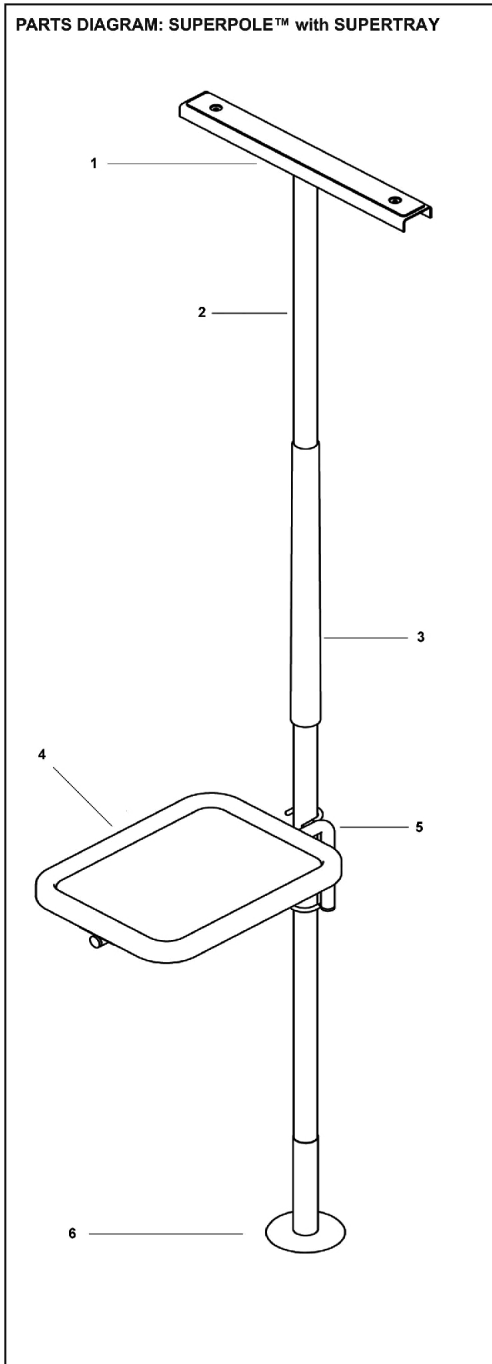

SuperPole with SuperTray / SuperPole with SuperTray Heavy Duty

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP403	Ceiling Anchor Plate Assembly	1	1
1		SP403HD	Ceiling Anchor Plate Assembly, Heavy Duty	1	1
2		SP-P	Pole Assembly	1	1
2		SP-PHD	Pole Assembly, Heavy Duty	1	1
3		GR-101	Grip, 20"	1	1
4		SP406	Tray Frame	1	1
5		SP407	Tray Arm Assembly	1	1
6		SP402	Base Plate Assembly	1	1
7		HW323	Hitch Pin	1	1
8		HW318	Washer, Nylon	1	1
9		HW302	Clevis Pin	1	1
10		SP136	Rubber Tray Mat	1	1
11		SP405	Tray	1	1
12		HW102	Flat Screws	1	4
13		HW301	Hex Key, Short Arm (1/8")	1	1
14		SP401	Cover Sleeve, Plastic	1	1
15		SP105	Threaded Collar	1	1
15		SP105HD	Threaded Collar, Heavy Duty	1	1
16		SP172	Jam Nut	1	1
17		SP107	Extension Screw	1	1
18		SP145	Washer, Plastic	1	1
Not Shown		SP-BX	Parts Box, SuperPole	1	1
Not Shown		SPT-BX	SuperTray Parts Box	1	1
Not Shown		SP-BXHD	Parts Box, SuperPole Heavy Duty	1	1
Not Shown		SPT-BXHD	Parts Box, SuperTray Heavy Duty	1	1
Not Shown		SP506	Parts Pack, SuperTray	1	1
Not Shown		SP-500	Parts Pack, SuperPole	1	1