


Adjustable Head Support

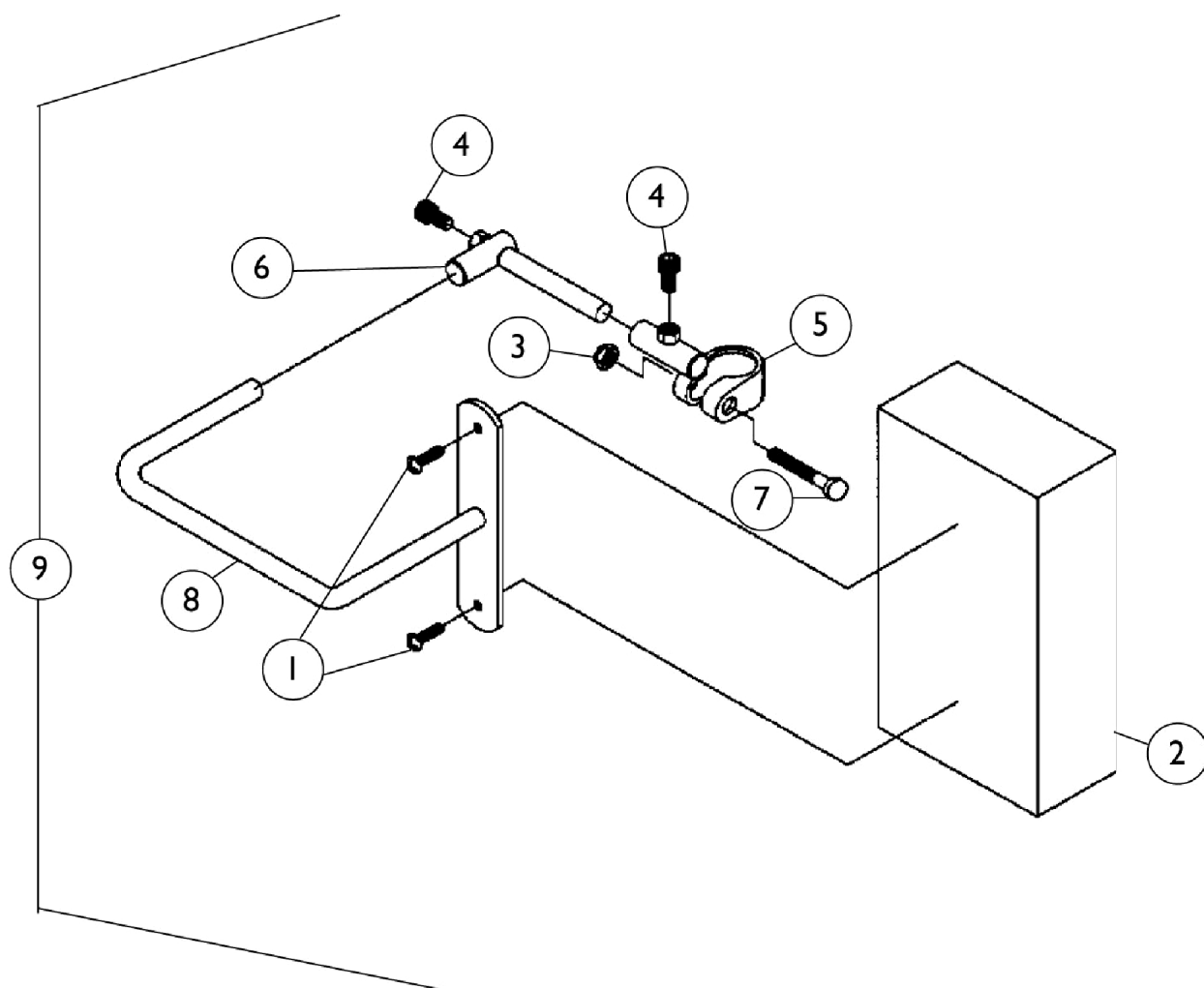

Adjustable Head Support

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	2
2	A	88804529X040-NLA	NLA - Pad, Headrest	1	1
3		7930330-NLA	NLA - Locknut (5/16-24)	25	1
4		1028481-NLA	NLA - Screw, Socket Head (1/4-20 x 1/2")	1	2
5		1028758-NLA	NLA - Bracket, Clamp, Black - Right	1	1
5		1028759-NLA	NLA - Bracket, Clamp, Black - Left	1	1
6		03018X042	Wing Bracket Assembly, Chrome	1	1
7		1031026	Screw w/ Lug (5/16-24 x 1-3/8")	1	1
8	C	03018X045	Bracket, Pad Support, Chrome	1	1
8	D	03018X044-NLA	NLA - Bracket, Pad Support, Black	1	1
9	A	8881044754-NLA	NLA - Head Support Assembly, Adjustable, Non-Recliner (Pair)	2	1
9	A	8881044756-NLA	NLA - Head Support Assembly, Adjustable, Recliner (Pair)	2	1
	B,E	1023610-NLA	NLA - Instruction Sheet	1	1
NOTE: A - Specify color when ordering, see Frame Finishes, Touch-Up Paints, Upholstery Options and Accent Packages page. B - Not Shown C - Non-Recliner D - Recliner E - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.					