



# Arm Assemblies - Full

For Standard, Tall, and Wide Chairs

## #18 ARM

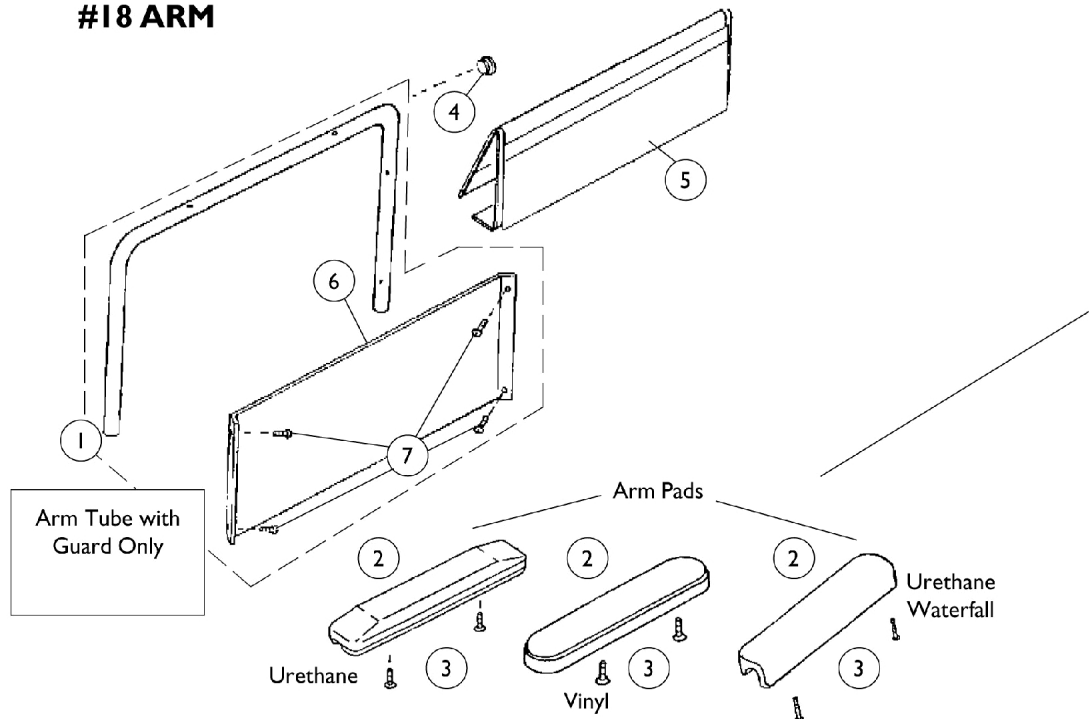

## #19 ARM

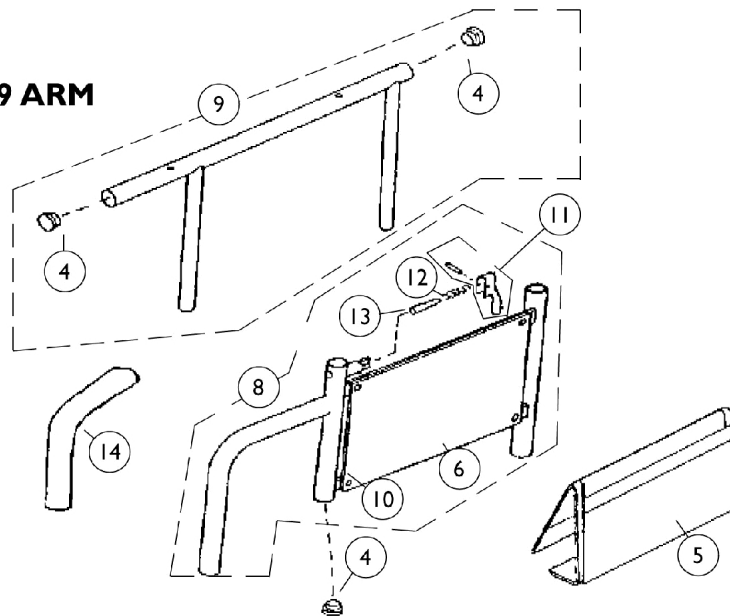

## Arm Assemblies - Full

### For Standard, Tall, and Wide Chairs

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,E	1029729-NLA	NLA - Arm Assembly, Full Length, Chrome - Right/Left	1	1
1	A,E	1028573-NLA	NLA - Arm Assembly, Full Length, Black - Right/Left	1	1
2		1030351-NLA	NLA - Pad, Arm, Full Length, Urethane, Grey (15" L x 2" W)	1	1
2	D	1117328	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)	1	1
2		1110818	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) (Pair)	2	1
2	B	8881015932-NLA A	NLA - Pad, Arm, Vinyl, Full Length - Right (14" L x 2" W)	1	1
2	B	8881015933-NLA A	NLA - Pad, Arm, Vinyl, Full Length - Left (14" L x 2" W)	1	1
3	H	7930327-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/8")	25	2
3	I	1026107	Screw, Phillips Pan Head (#10-32 x 1-3/8")	1	2
3	J,K	1094270-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/4")	2	1
4		1027095	Plug Button, Black (7/8")	1	1
5		1029201-NLA	NLA - Cover, Clothing Guard with Pocket, Full Fixed Arm	1	1
5		1029200-NLA	NLA - Cover, Clothing Guard with Pocket, Adjustable Arm	1	1
6		47234X001-NLA	NLA - Guard, Clothing, Full Length, Fixed Arm, Chrome	1	1
6		47234X045-NLA	NLA - Guard, Clothing, Chrome	1	1
6		47234X064-NLA	NLA - Guard, Clothing, Full Length, Fixed Arm, Black	1	1
6		47234X066-NLA	NLA - Guard, Clothing, Adjustable Arm, Black	1	1
7		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	4
8	F,L	1010452-NLA	NLA - Arm Assembly, Lower Adjustable Height, Silver Vein - Right	1	1
8	F,L	1010453-NLA	NLA - Arm Assembly, Lower Adjustable Height, Silver Vein - Left	1	1
8	F	1028646-NLA	NLA - Arm Assembly, Lower Adjustable Height, Black - Right	1	1
8	F	1028647-NLA	NLA - Arm Assembly, Lower Adjustable Height, Black - Left	1	1
9	G	1041522	Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right/Left	1	1
10		10351X001-NLA	NLA - Rivet	1	4
11		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
12		47471X074	Spring, Arm Lock	1	1
13		1028594	Pin, Arm Lock, Black	1	1
14	C	1032245	Grip, Foam Hand (6-7/8" L)	1	1

## **Arm Assemblies - Full**

### **For Standard, Tall, and Wide Chairs**

NOTE: A - Arm tubes are universal (right or left) after July 1992. Arm tube is retrofitable.  
B - Specify upholstery color when ordering, see Frame Finishes, Touch-Up Paints, Upholstery Options and Accent Packages page.  
C - Foam Grip 1 Pad with ribbed edges are no longer available  
D - As of 1/3/02 upholstered waterfall armpads were replaced with black urethane waterfall armpads. Order in pairs with screws for chairs serviced before 1/3/02.  
E - Includes items 6 and 7  
F - Includes items 6 and 10-13  
G - Includes item 4  
H - Vinyl Arm Pad  
I - Urethane Pad, Grey  
J - Pair  
K - For Black Urethane Waterfall Arms Pads Only  
L - As of 7/6/07, chrome finish option is discontinued and no longer available.