



Arm Assemblies - Desk

For Standard, Tall, and Wide Chairs

#28 ARM

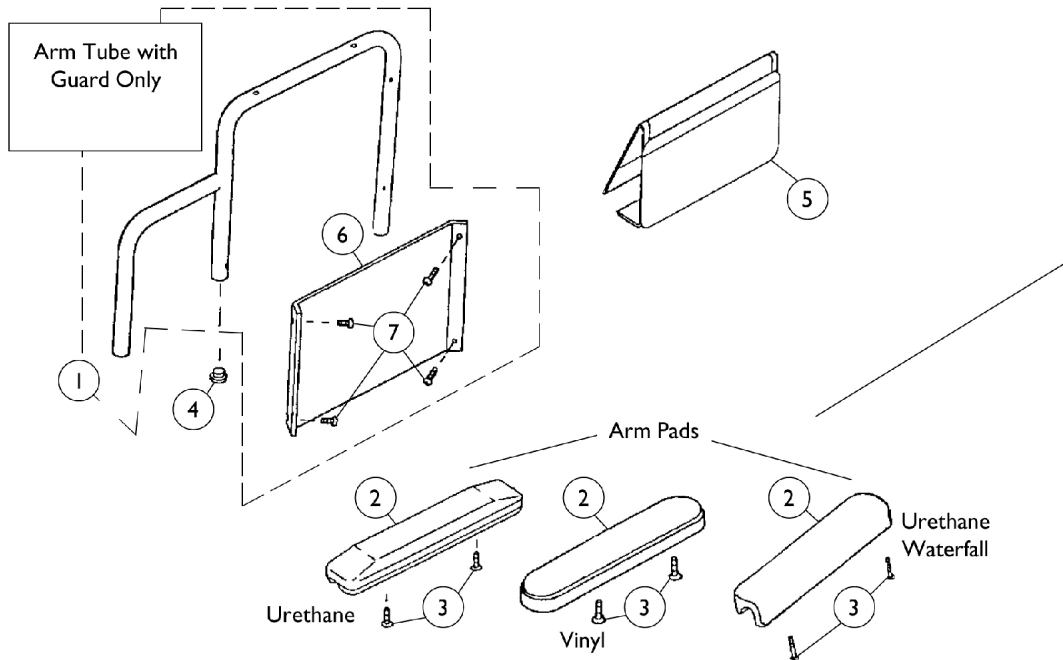

#29 ARM

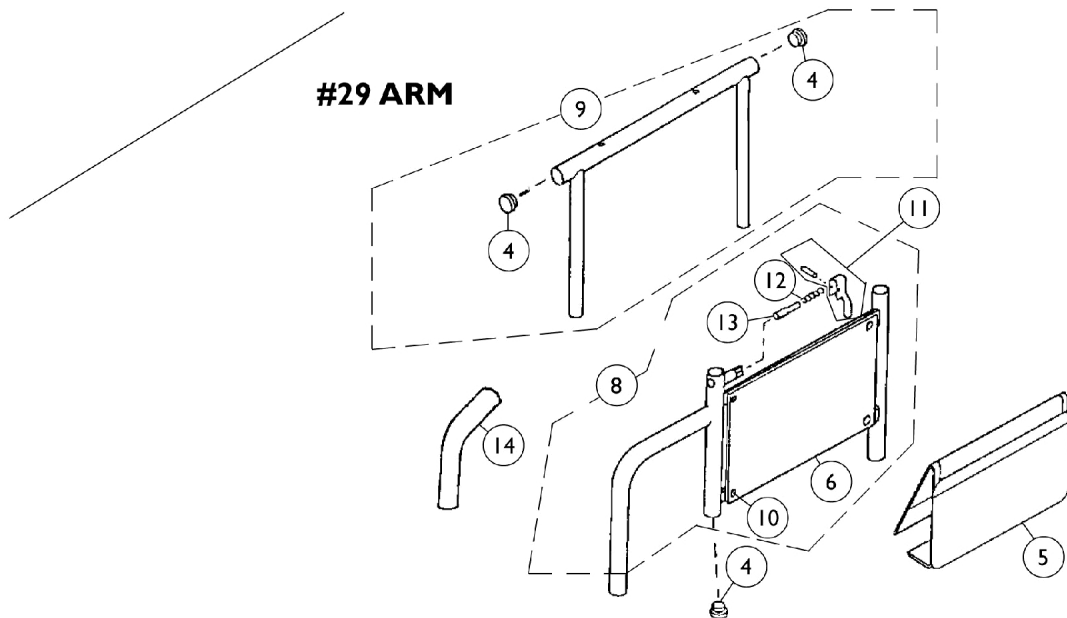

Arm Assemblies - Desk

For Standard, Tall, and Wide Chairs

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D	1029731-NLA	NLA - Arm Assembly, Desk Length, Chrome - Right	1	1
1	D	1029732-NLA	NLA - Arm Assembly, Desk Length, Chrome - Left	1	1
1	D	1028637-NLA	NLA - Arm Assembly, Desk Length, Black - Right	1	1
1	D	1028638-NLA	NLA - Arm Assembly, Desk Length, Black - Left	1	1
2		1030352-NLA	NLA - Pad, Arm, Desk Length, Urethane, Grey (9" L x 2" W)	1	1
2	C	1117327	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W)	1	1
2		1110817	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W) (Pair)	2	1
2	A	8881011811-NLA	NLA - Pad, Arm, Vinyl, Desk Length (9" L x 2" W)	1	1
3	H	7930327-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/8")	25	2
3	I	1026107	Screw, Phillips Pan Head (#10-32 x 1-3/8")	1	2
3	G	1094270-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/4")	2	1
4		1027095	Plug Button, Black (7/8")	1	2
5		1029200-NLA	NLA - Cover, Clothing Guard with Pocket, Adjustable Arm	1	1
6		47234X002-NLA	NLA - Guard, Clothing, Desk Length, Fixed Arm, Chrome	1	1
6		47234X045-NLA	NLA - Guard, Clothing, Chrome	1	1
6		47234X065-NLA	NLA - Guard, Clothing, Desk Length, Fixed Arm, Black	1	1
6		47234X066-NLA	NLA - Guard, Clothing, Adjustable Arm, Black	1	1
7		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	4
8	E	1010452-NLA	NLA - Arm Assembly, Lower Adjustable Height, Silver Vein - Right	1	1
8	E	1010453-NLA	NLA - Arm Assembly, Lower Adjustable Height, Silver Vein - Left	1	1
8	E	1028646-NLA	NLA - Arm Assembly, Lower Adjustable Height, Black - Right	1	1
8	E	1028647-NLA	NLA - Arm Assembly, Lower Adjustable Height, Black - Left	1	1
9	F	1070859	Arm Tube Assembly w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
10		10351X001-NLA	NLA - Rivet	1	4
11		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
12		47471X074	Spring, Arm Lock	1	1
13		1028594	Pin, Arm Lock, Black	1	1
14	B	1032245	Grip, Foam Hand (6-7/8" L)	1	1

Arm Assemblies - Desk

For Standard, Tall, and Wide Chairs

NOTE: A - Specify upholstery color when ordering, see Frame Finishes, Touch-Up Paints, Upholstery Options and Accent Packages page.
B - Foam Grip/Pad with ribbed edges are no longer available. 1032245 is retrofittable.
C - As of 1/3/02 upholstered waterfall armpads were replaced with black urethane waterfall armpads. Order in pairs with screws for chairs serviced before 1/3/02.
D - Includes items 6 and 7
E - Includes items 6 and 10-13
F - Includes item 4
G - For Black Urethane Waterfall Arm Pads only
H - Vinyl Arm Pad
I - Urethane Pad, Grey