


Arm Assemblies


Full and Desk For 2" Lower Arms

NOTE: Desk length arms shown for reference. This page refers to both full and desk length 2" lower arm assemblies.

#28 ARM

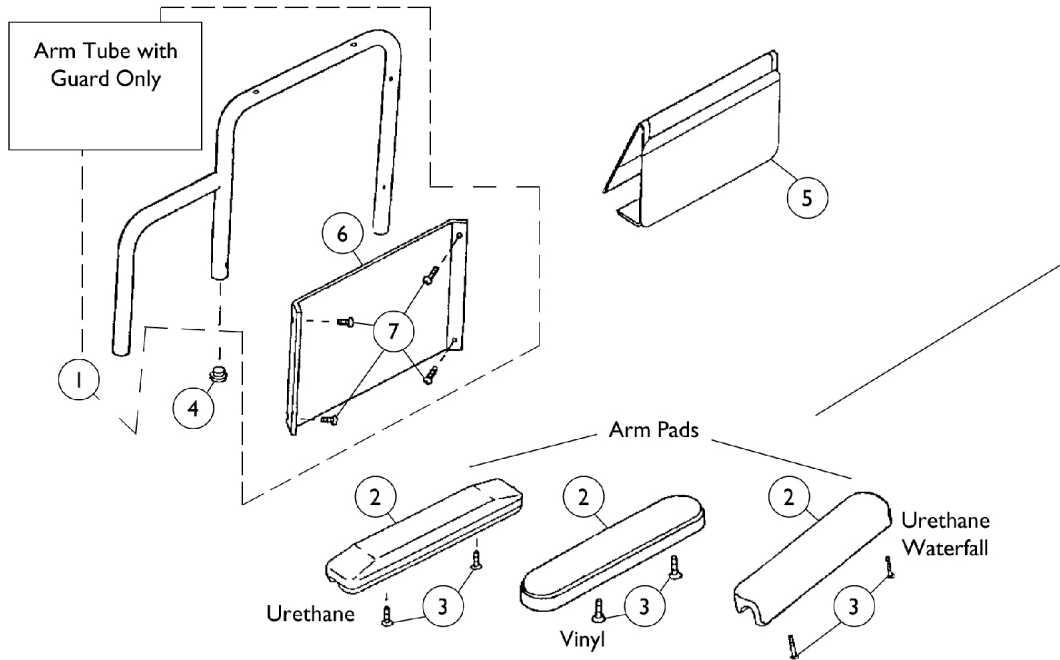

#29 ARM

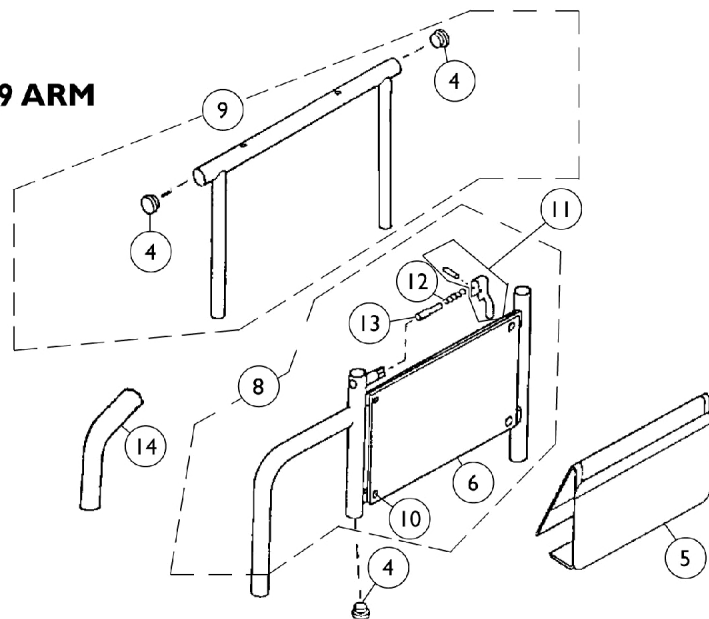

Arm Assemblies

Full and Desk For 2" Lower Arms

NOTE: Desk length arms shown for reference. This page refers to both full and desk length 2" lower arm assemblies.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1035475-NLA	NLA - Tube, Desk Length Arm with Clothing Guard, Fixed Height, Chrome - Right	1	1
1	E	1035476-NLA	NLA - Tube, Desk Length Arm with Clothing Guard, Fixed Height, Chrome - Left	1	1
1	E	1035473-NLA	NLA - Tube, Desk Length Arm with Clothing Guard, Fixed Height, Black - Right	1	1
1	E	1035474-NLA	NLA - Tube, Desk Length Arm with Clothing Guard, Fixed Height, Black - Left	1	1
2		1030351-NLA	NLA - Pad, Arm, Full Length, Urethane, Grey (15" L x 2" W)	1	1
2	D	1117328	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)	1	1
2		1110818	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) (Pair)	2	1
2	A	8881015932-NL A	NLA - Pad, Arm, Vinyl, Full Length - Right (14" L x 2" W)	1	1
2	A	8881015933-NL A	NLA - Pad, Arm, Vinyl, Full Length - Left (14" L x 2" W)	1	1
2		1030352-NLA	NLA - Pad, Arm, Desk Length, Urethane, Grey (9" L x 2" W)	1	1
2	D	1117327	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W)	1	1
2		1110817	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W) (Pair)	2	1
2	A	8881011811-NL A	NLA - Pad, Arm, Vinyl, Desk Length (9" L x 2" W)	1	1
3	J	7930327-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/8")	25	2
3	K	1026107	Screw, Phillips Pan Head (#10-32 x 1-3/8")	1	2
3	F	1094270-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/4")	2	1
4		1027095	Plug Button, Black (7/8")	1	2
5		1035065	Cover, Clothing Guard w/ Pocket, FXDESK & ADJFULL/ADJDESK	1	1
6		1034635	Guard, Clothing Desk Length, Fixed Arm, Chrome	1	1
6	C,L	1064130-NLA	NLA - Guard, Clothing	1	1
6		1035472-NLA	NLA - Guard, Clothing, Desk Length	1	1
6	C,L	1064130-NLA	NLA - Guard, Clothing	1	1
7		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	4
8	C,G	1034936-NLA	NLA - Arm Assembly, Lower Adjustable Height, Chrome - Right	1	1
8	C,G	1034937-NLA	NLA - Arm Assembly, Lower Adjustable Height, Chrome - Left	1	1
8	C,G	1034934-NLA	NLA - Arm Assembly, Lower Adjustable Height, Black - Right	1	1

Arm Assemblies

Full and Desk For 2" Lower Arms

NOTE: Desk length arms shown for reference. This page refers to both full and desk length 2" lower arm assemblies.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
8	C,G	1034935-NLA	NLA - Arm Assembly, Lower Adjustable Height, Black - Left	1	1
9	H	1041522	Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right/Left	1	4
9	H	1070859	Arm Tube Assembly w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
10	I	1028645	Rivet	1	6
11		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
12		47471X074	Spring, Arm Lock	1	1
13		1028594	Pin, Arm Lock, Black	1	1
14	B	1032245	Grip, Foam Hand (6-7/8" L)	1	1

NOTE: A - Specify upholstery color when ordering, see Frame Finishes, Touch-Up Paints, Upholstery Options and Accent Packages page.
 B - Foam grip 1 pad with ribbed edges are no longer available. 1032245 is retrofittable.
 C - Stainless steel clothing guards were replaced with ABS plastic. Chairs manufactured before 7/30/97 should be ordered in pairs to match.
 D - As of 1/3/02 upholstered waterfall armpads were replaced with black urethane waterfall armpads. Order in pairs with screws for chairs serviced before 1/3/02.
 E - Includes items 6 and 7
 F - For Black Urethane Waterfall Arm Pads only.
 G - Includes items 6 and 10-13
 H - Includes item 4
 I - Adjustable height arms require quantity of 6
 J - Vinyl Arm Pad
 K - Urethane Pad, Grey
 L - Used for both Chrome and Black