
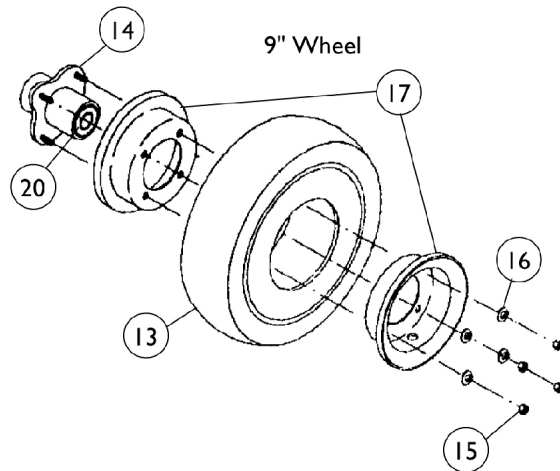
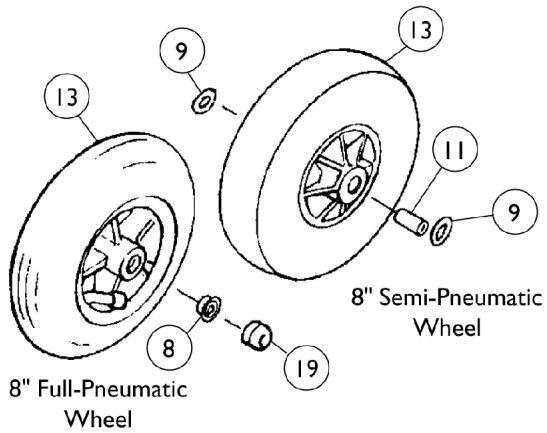
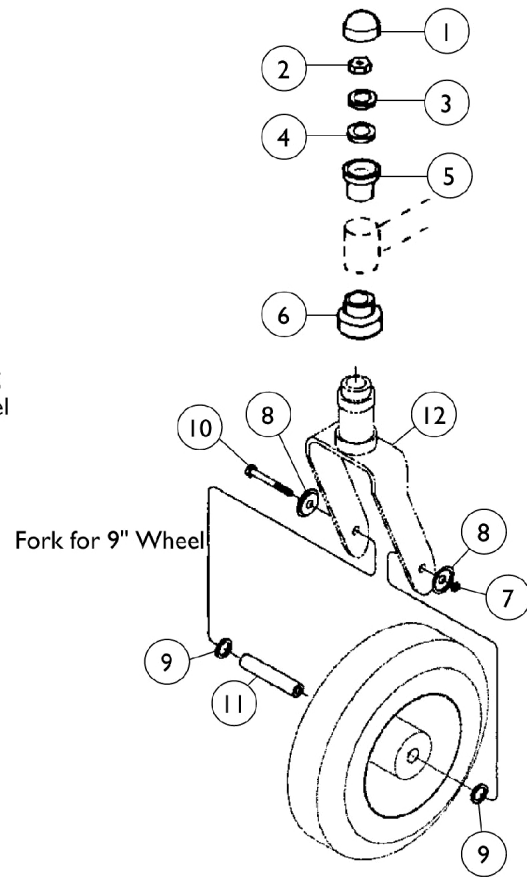


Front Caster and Forks

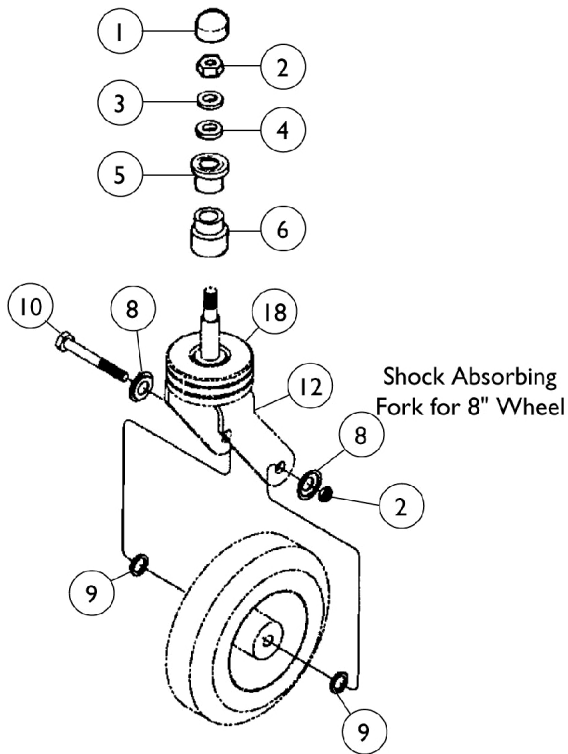

Front Caster and Forks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1001843	Cap, Head Tube, Black	1	1
1		1131189	Package, Head Tube Cap	10	1
2		1025734	Locknut (7/16-20)	1	1
3		40014X044-NLA	NLA - Washer, Nylon (7/16 x 1 x 1/8")	1	1
4		1006049	Bearing (7/16")	1	1
5		57768X030-NLA	NLA - Sleeve, Aluminum	1	1
6		1029703-NLA	NLA - Bottom Sleeve w/ Bearing	1	1
7		1025730	Locknut (5/16-24)	1	1
8	C	1016289	Washer, Caplugs (15/32" I.D. x 1-3/32" O.D.)	1	1
8	B	1028491-NLA	NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.)	1	1
9	A,C	1016032	Washer, Steel (15/32 x 5/8 x 3/64")	1	2
9	A,B	1027794-NLA	NLA - Washer, Flat (41/64 x 7/8 x 3/32")	1	2
10	C	1025726	Screw, Hex Head (7/16-20 x 3-1/4")	1	1
10	B	1027795-NLA	NLA - Axle, Hex Head (5/16-24 x 4")	1	1
11	B	1027712-NLA	NLA - Reducer, Axle	1	1
12		See Chart	#12 Caster Forks	1	1
13		See Chart	#13 Caster Wheels	1	1
14		1013466	Hub with Bearings Assembly	1	4
15		1024426-NLA	NLA - Nut, Hex (5/16-24)	1	4
16		1024442-NLA	NLA - Washer, Lock (5/16 x 9/16 x 1/16")	1	1
17		1025399-NLA	NLA - Kit, Rim, Outer/Inner Half w/ Hardware (9")	1	1
18		1028669-NLA	NLA - Boot, Fork, Shock Absorbing	1	1
19		1028490-NLA	NLA - Cap, Hub, Grey	1	1
19		1028488-NLA	NLA - Cap, Hub, Teal	1	1
19		1028489-NLA	NLA - Hub Cap, Lavender	1	1
19	D	1028835-NLA	NLA - Cap, Hub, Black	1	1
20		1015610	Bearing with Grip Ring (5/8")	1	1
NOTE: A - Used with Accent Hub Caps B - For 9" Wheel C - For 8" Wheel D - For the Red Accent Package					

Item #12 Caster Forks

Description	Black Finish	Chrome Finish	Color Finish
Ranger Fork	1029705-NLA See Note(s) A	N/A	N/A
Shock Absorbing Fork for 8" Wheel	1040339-NLA	04540X251-NLA See Note(s) B	8881040340-NLA
Fork for 9" Wheel	1040337-NLA	N/A	8881040338-NLA
NOTE: A - Standard on Ranger B - Chrome forks do not come with fork boot, they must be ordered separately.			

Item #13 Caster Wheels

Front Caster and Forks

Item #13 Caster Wheels

Description	8 x 1-3/4" Wheel, Semi-Pneumatic	8 x 1-3/4" Wheel, Semi-Pneumatic	8 x 2" Wheel, Full Pneumatic	9x2-3/4" Wheel, Full Pneumatic	9x2-3/4" Wheel, w/Flat Free Insert
Wheel w/Bearings Pair	Before 6/9/97	1071666 See Note(s) B	N/A	N/A	N/A
Wheel w/Bearings Piece	1006058-NLA	1070811 See Note(s) B	1002002	1025400-NLA	1029871-NLA
Replacement Tire	N/A	N/A	40176X034	1027418-NLA	1028896-NLA
Replacement Tube	N/A	N/A	1034942	1027417	N/A See Note(s) A
Hub Bearing	1006049	1006049	1006049	N/A	N/A
Hub Spacer	1020592	1020592	1020592	N/A	N/A

NOTE: A - Tire comes with insert molded inside.
B - For chairs manufactured before 6/9/97, order the replacement number 1071666.