


# Rear Wheels


## CONVENTIONAL WHEEL

All Chairs Except Slim Narrow & P.P.8.

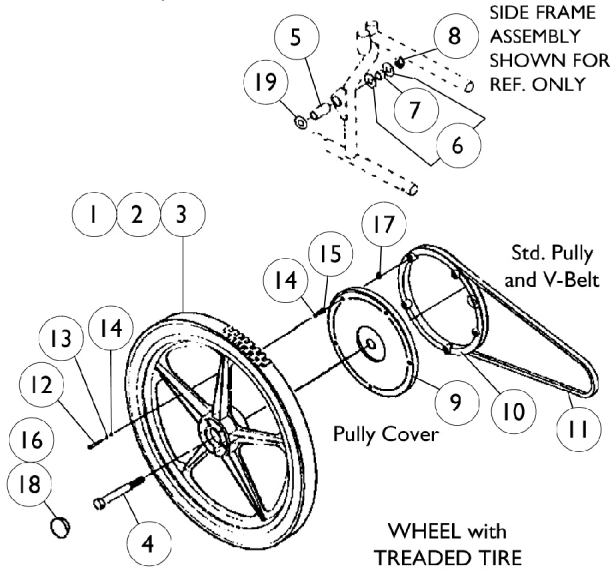

## CONVENTIONAL WHEEL


For Slim Narrow Chairs

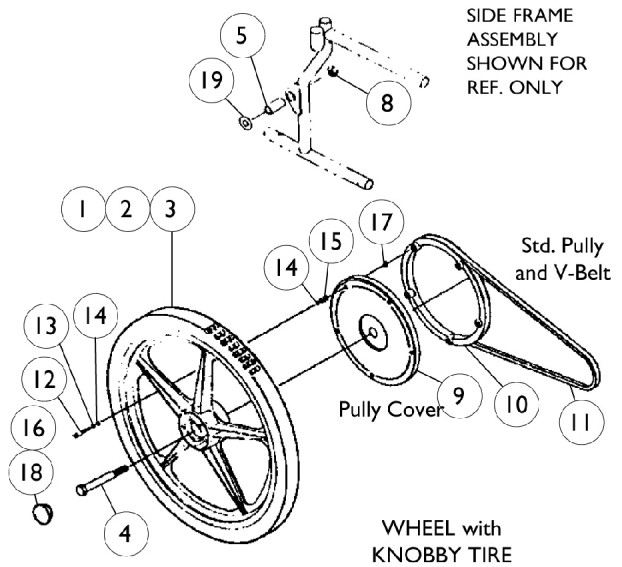

## P.P.8. WHEEL

(Performance Plus Package)


All Chairs Except Slim Narrow

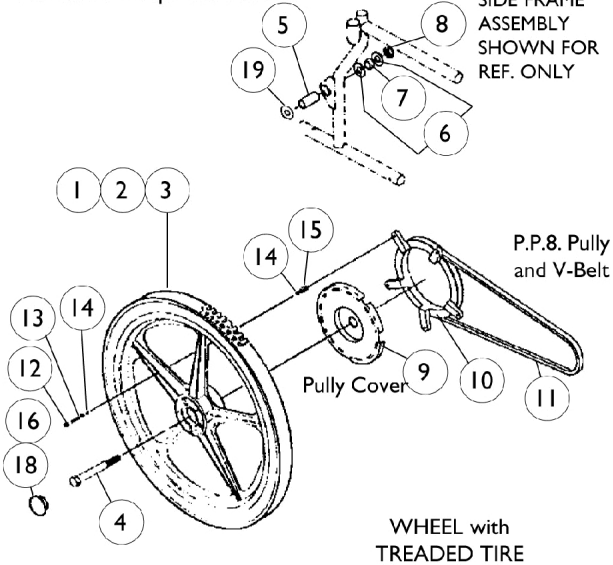

## P.P.8. WHEEL

(Performance Plus Package)

For Slim Narrow Chairs

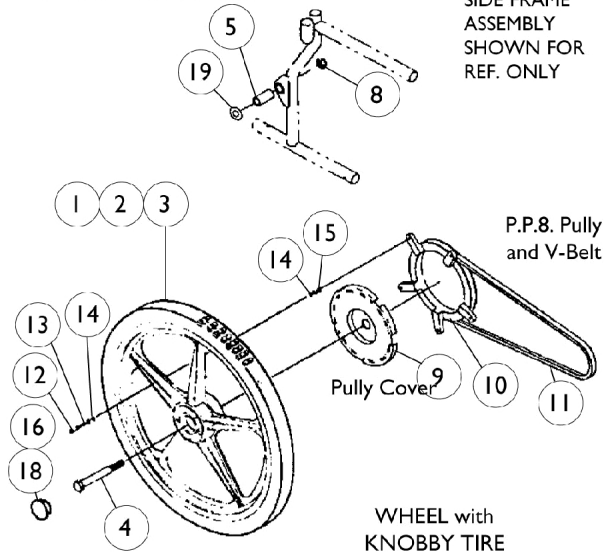

## Rear Wheels

| Item Number                                                                                                                                                                                                                                                                                                              | Note | Part Number | Description                                        | Quantity Per |          |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|-------------|----------------------------------------------------|--------------|----------|
|                                                                                                                                                                                                                                                                                                                          |      |             |                                                    | Package      | Assembly |
| 1-3                                                                                                                                                                                                                                                                                                                      | C    | See Chart   | Rear Wheels, Tires and Tube Insert                 | 1            | 1        |
| 4                                                                                                                                                                                                                                                                                                                        |      | 1029749-NLA | NLA - Screw, Hex Head ( 5/8-18 x 5-3/4")           | 1            | 1        |
| 4                                                                                                                                                                                                                                                                                                                        | A    | 1028554-NLA | NLA - Screw, Hex Head (5/8-18 x 5")                | 1            | 1        |
| 5                                                                                                                                                                                                                                                                                                                        |      | 1028551-NLA | NLA - Spacer (5/8 x 7/8 x 1-3/4")                  | 1            | 1        |
| 6                                                                                                                                                                                                                                                                                                                        |      | 1028556-NLA | NLA - Washer, Flat (21/32 x 1-5/16 x 3/32")        | 1            | 2        |
| 7                                                                                                                                                                                                                                                                                                                        |      | 1028555-NLA | NLA - Spacer (5/8 x 7/8 x 3/8")                    | 1            | 1        |
| 8                                                                                                                                                                                                                                                                                                                        |      | 1028550     | Locknut (5/8-18)                                   | 1            | 1        |
| 9                                                                                                                                                                                                                                                                                                                        |      | 1028667-NLA | NLA - Cover, Pulley, Standard (10-17/32") Diameter | 1            | 1        |
| 9                                                                                                                                                                                                                                                                                                                        |      | 1028668-NLA | NLA - Cover, Pulley, P.P.8 (7-25/32") Diameter     | 1            | 1        |
| 10                                                                                                                                                                                                                                                                                                                       | F    | 1004093-NLA | NLA - Pulley, Standard (10-17/32") Diameter        | 1            | 1        |
| 10                                                                                                                                                                                                                                                                                                                       | F    | 1007102-NLA | NLA - Pulley, P.P.8 (7-25/32") Diameter            | 1            | 1        |
| 11                                                                                                                                                                                                                                                                                                                       |      | 1004107-NLA | NLA - V-Belt, Standard (4M & 5M Size AX51)         | 1            | 1        |
| 11                                                                                                                                                                                                                                                                                                                       |      | 1005900     | V-Belt, 2" Lower (4M/5M Size AX52)                 | 1            | 1        |
| 11                                                                                                                                                                                                                                                                                                                       |      | 1007299-NLA | NLA - V-Belt, P.P.8 (4M & 5M Size AX46 )           | 1            | 1        |
| 11                                                                                                                                                                                                                                                                                                                       |      | 1004855-NLA | NLA - V-Belt, Recliner (4M/5M Size AX56)           | 1            | 1        |
| 11                                                                                                                                                                                                                                                                                                                       |      | 50000X356   | V-Belt, Recliner, 2" Lower (4M/5M Size AX55)       | 1            | 1        |
| 12                                                                                                                                                                                                                                                                                                                       |      | 1038379     | Screw, Socket Head Cap w/ Patch (1/4-20 x 1-1/2")  | 1            | 5        |
| 13                                                                                                                                                                                                                                                                                                                       |      | 1013970-NLA | NLA - Lockwasher, High Collar (1/4")               | 1            | 5        |
| 14                                                                                                                                                                                                                                                                                                                       |      | 1013971     | Washer, Nylon (1/4 x 7/16 x 5/64")                 | 1            | 5        |
| 15                                                                                                                                                                                                                                                                                                                       | E    | 1028553-NLA | NLA - Spacer (1/4 x 3/8 x 3/4")                    | 1            | 5        |
| 15                                                                                                                                                                                                                                                                                                                       | D    | 1029286-NLA | NLA - Spacer (1/4 x 3/8 x 5/8")                    | 1            | 1        |
| 16                                                                                                                                                                                                                                                                                                                       |      | 1027954-NLA | NLA - Hub Cap, Grey                                | 1            | 1        |
| 16                                                                                                                                                                                                                                                                                                                       |      | 1027955-NLA | NLA - Hub Cap, Teal                                | 1            | 1        |
| 16                                                                                                                                                                                                                                                                                                                       | B    | 1027953-NLA | NLA - Hub Cap, Black Hub                           | 1            | 1        |
| 17                                                                                                                                                                                                                                                                                                                       |      | 1035423     | O'Ring (5/16 x 5/8")                               | 1            | 5        |
| 18                                                                                                                                                                                                                                                                                                                       |      | 1029752-NLA | NLA - Lock Ring, Hub Cap                           | 1            | 1        |
| 19                                                                                                                                                                                                                                                                                                                       | G    | 1027794-NLA | NLA - Washer, Flat (41/64 x 7/8 x 3/32")           | 1            | 2        |
|                                                                                                                                                                                                                                                                                                                          | H    | 40117X006   | Bearing (5/8")                                     | 1            | 1        |
|                                                                                                                                                                                                                                                                                                                          | H    | 1004313-NLA | NLA - Spacer, Nylon                                | 1            | 1        |
| NOTE: A - Used on MFF, Manual Recline and Slim Narrow<br>B - Black hub caps are used when red or lavender caps are requested<br>C - Rear wheel is supplied with 2 bearings (40117X006) and 1 spacer (1004313 NLA)<br>D - P.P.8 Pully<br>E - Standard Pully<br>F - 20" Wheel<br>G - 2" lower chairs ONLY<br>H - Not Shown |      |             |                                                    |              |          |

## Rear Wheels, Tires and Tube Insert

| Description<br>20" Wheel with | Item #1<br>20" Wheel Only<br>No Pulley | Item #1<br>20" Wheel<br>Standard Pulley<br>& Pulley Cover | Item #1<br>20" Wheel<br>with PP8 Pulley<br>& Pulley Cover | Item #2<br>Tire Part | Item #3<br>Tube Insert |
|-------------------------------|----------------------------------------|-----------------------------------------------------------|-----------------------------------------------------------|----------------------|------------------------|
| Treaded Tire                  | 1030407-NLA                            | 1037179-NLA                                               | 1037182-NLA                                               | 1006858-NLA          | 40176X071-NLA          |
| Knobby Tire                   | 1030406-NLA                            | 1037180-NLA                                               | 1037183-NLA                                               | 40176X072-NLA        | 40176X071-NLA          |

# Rear Wheels

## Rear Wheels, Tires and Tube Insert

| Description<br>20" Wheel with       | Item #1<br>20" Wheel Only<br>No Pulley | Item #1<br>20" Wheel<br>Standard Pulley<br>& Pulley Cover | Item #1<br>20" Wheel<br>with PP8 Pulley<br>& Pulley Cover | Item #2<br>Tire Part | Item #3<br>Tube Insert |
|-------------------------------------|----------------------------------------|-----------------------------------------------------------|-----------------------------------------------------------|----------------------|------------------------|
| Treaded Tire<br>w/ Flat Free Insert | 1030408-NLA                            | 1037181-NLA                                               | 1037184-NLA                                               | 1006858-NLA          | 1010905-NLA            |