


Tilt Hardware

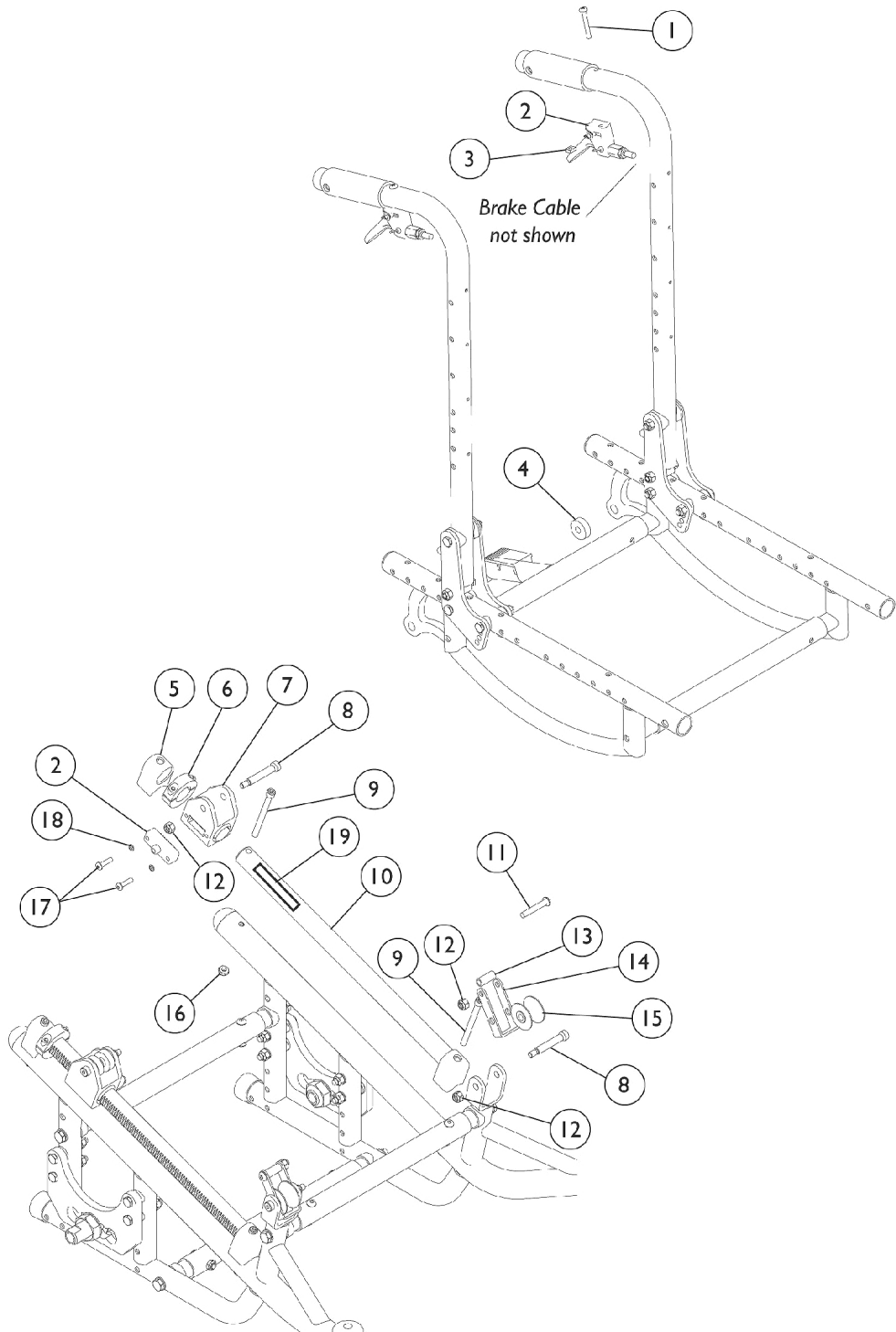

Tilt Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	1026107	Screw, Phillips Pan Head (#10-32 x 1-3/8")	1	2
2		1161550-NLA	NLA - Cable and Brake Assembly	1	2
3		1086207	Nut, Square (#10-32)	1	2
4		1104627-NLA	NLA - Spacer, Hinge Seat Frame, Black (5/16 x 1 x 7/16")	1	4
5		1104645-NLA	NLA - Stand-Off, Shaft	1	4
6		1083088	Collar, Tilt Lock Out (7/8")	1	2
7		1080622-NLA	NLA - Housing, Bearing	1	2
8		1071736	Screw, Shoulder Hex Socket Head (5/16" x 1-5/8", 1/4-20)	1	4
9		1083177-NLA	NLA - Screw, Socket Head (1/4-28 x 2-7/8")	1	4
10		1079554-NLA	NLA - Rack, Gear, Grey Aluminum	1	2
11		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	2
12		1025736	Locknut (1/4-20)	1	6
13		1075146-NLA	NLA - Sleeve (1/4-10 x 7/8 x 1/6" O.D.)	1	2
14		1080620-NLA	NLA - Roller, Link	1	2
15		1083098-NLA	NLA - Roller w/ Bushings	1	2
16		1025765	Locknut (1/4-28)	1	4
17		1003645	Screw, Socket Button Head (#10-32 x 3/4")	1	4
18		1028527-NLA	NLA - Washer, Lock (3/16 x 5/16 x 1/32")	1	4
19	A,B	1104628-NLA	NLA - Hardware, Tilt	1	1
19		1085303-NLA	NLA - Decal, Graduated Tilt - Right	1	1
19		1085304-NLA	NLA - Decal, Graduated Tilt - Left	1	1
NOTE: A - Not Shown B - Includes items 1-18. C - Includes 1080623 and Item 3 (1086207).					