


Wheel Lock Assembly

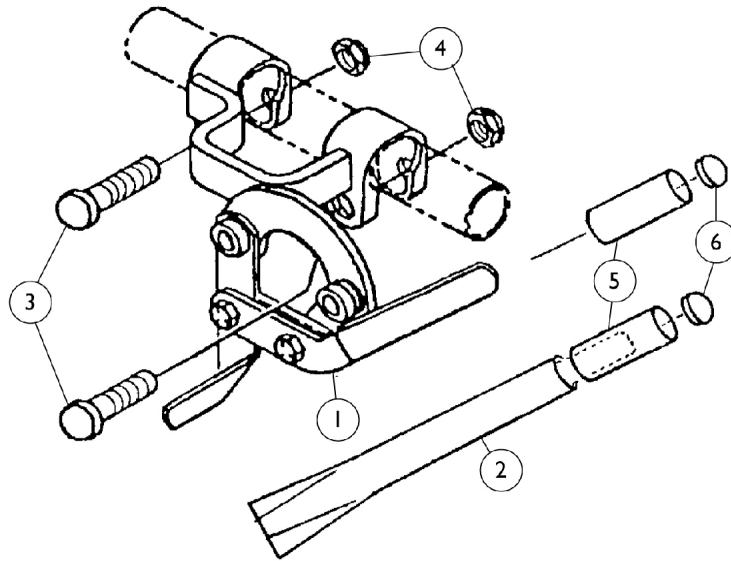

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1028585-NLA	NLA - Wheel Lock, Less Lever Grip and Tip, Standard Configuration, Pull to Lock, Right	1	1
1		1028586-NLA	NLA - Wheel Lock, Less Lever Grip and Tip, Standard Configuration, Pull to Lock, Left	1	1
1		1043324-NLA	NLA - Wheel Lock, Less Lever Grip and Tip, Tall Configuration, Push to Lock, Right	1	1
1		1043323-NLA	NLA - Wheel Lock, Less Lever Grip and Tip, Tall Configuration, Push to Lock, Left	1	1
1		1034932-NLA	NLA - Wheel Lock, Less Lever Grip and Tip, 2" Lower Configuration, Push to Lock - Right	1	1
1		1034933-NLA	NLA - Wheel Lock, Less Lever Grip and Tip, 2" Lower Configuration, Push to Lock - Left	1	1
2		1028782-NLA	NLA - Wheel Lock Extension Handle	1	1
3		1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	2
4		7930330-NLA	NLA - Locknut (5/16-24)	25	2
5		1027956-NLA	NLA - Grip, Lever w/o Tip for Wheel Lock	1	1
5		1027957-NLA	NLA - Grip, Lever w/o Tip for Extension Handle	1	1
6		1030347-NLA	NLA - Cap, End, Grey	1	1
6		1030348-NLA	NLA - Cap, End, Teal	1	1
6		1030349-NLA	NLA - Cap, End, Red	1	1
6		1030350-NLA	NLA - Cap, End, Lavender	1	1