


Footrest Support Assembly

93E - Black, Chrome and Color

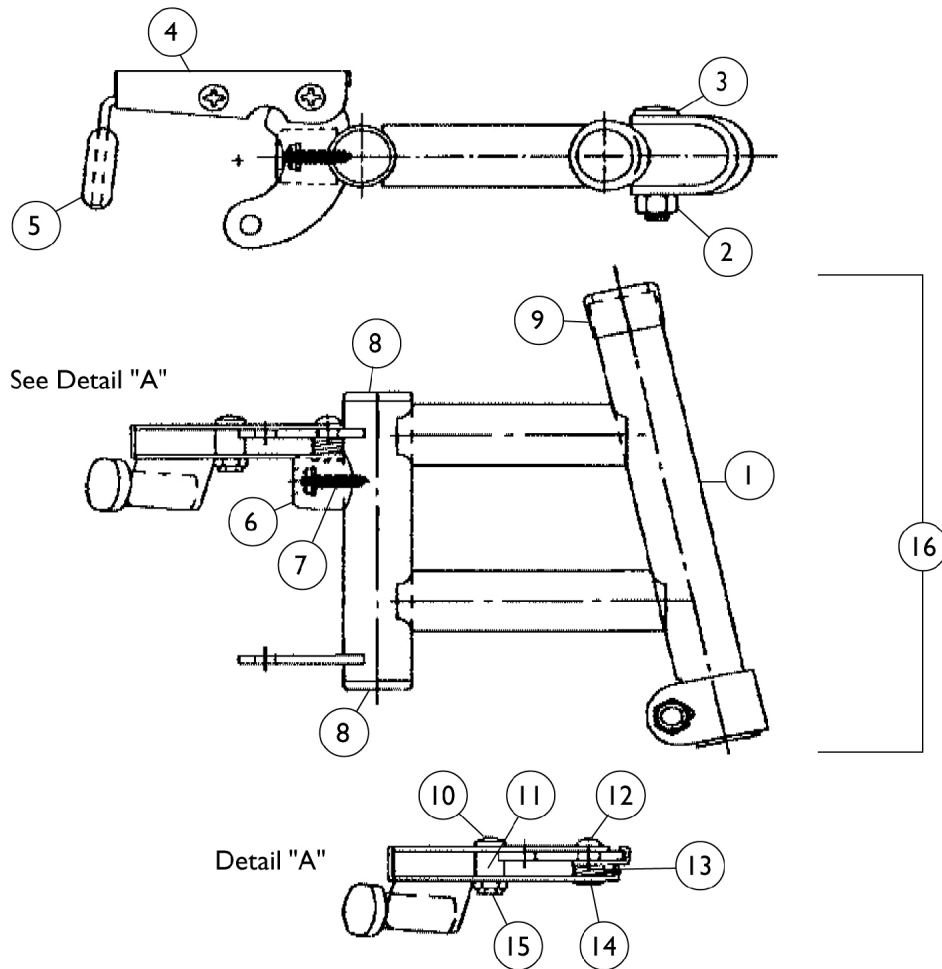

Footrest Support Assembly

93E - Black, Chrome and Color

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1037685-NLA	NLA - Tube, Footrest Support, Adult, Black - Right	1	1
1		1037686-NLA	NLA - Tube, Footrest Support, Adult, Black - Left	1	1
1		1016724-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Right	1	1
1		1016725-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Left	1	1
1	A	8881037689-NLA	NLA - Tube, Footrest Support, Adult, Color - Right	1	1
1	A	8881037690-NLA	NLA - Tube, Footrest Support, Adult, Color - Left	1	1
2		1107321	Nut, Low Cap (5/16"-24)	1	1
3		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
4		1035539	Handle, Release, Black - Right	1	1
4		1035540	Handle, Release, Black - Left	1	1
5	B	1006490	Tip, Plastic, Black	1	1
6		1022874	Bumper	1	1
7		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
8		1027095	Plug Button, Black (7/8")	1	2
9		15680X012	Collar, Anti-Rattle (7/8")	1	1
10		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
11		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
12		40163X002	Screw, Phillips Round Head (#10-32 x 1/2")	1	1
13		1008935	Spring, Release - Right	1	1
13		1008936	Spring, Release - Left	1	1
14		67005X000	Rivnut (#10-32)	1	1
15		7930365-NLA	NLA - Locknut (#10-32)	25	1
16	C	1037687-NLA	NLA - Footrest Support Assembly, Adult, Black - Right	1	1
16	C	1037688-NLA	NLA - Footrest Support Assembly, Adult, Black - Left	1	1
16	C	1030994-NLA	NLA - Footrest Support Assembly, Adult, Chrome - Right	1	1
16	C	1030995-NLA	NLA - Footrest Support Assembly, Adult, Chrome - Left	1	1
16	A,C	8881037691-NLA	NLA - Footrest Support Assembly, Adult, Color - Right	1	1
16	A,C	8881037692-NLA	NLA - Footrest Support Assembly, Adult, Color - Left	1	1
	D	See Chart	Pivot Slide Tubes with Aluminum Footplates	1	1

Footrest Support Assembly

93E - Black, Chrome and Color

NOTE: A - Specify frame color when ordering, see Frame Finishes, Touch-Up Paints, Upholstery Options and Accent Packages page.
B - Before 10/1/00 only
C - Assembly includes Release Handle, Tip, Bolt, and Locknut.
D - Not Shown

Footrest Support Assembly

93E - Black, Chrome and Color

Pivot Slide Tubes with Aluminum Footplates

Description	Size	Right	Left
Adult, and Narrow Adult and Large Adult (Rubber Edged Plate)	(16-24")	1028905-NLA See Note(s) A	1028906-NLA See Note(s) A
Slim Narrow (Plain Aluminum Plate)	(14" - 15")	1033506-NLA See Note(s) B	1033507-NLA See Note(s) B
NOTE: A - Front Riggings built before 12/93 had footplates with a flat black finish. After 11/93 the finish is "Wrinkle Black." For Front Riggings before 12/93, order in pairs to match. B - Aluminum footplates on chairs built before 2/96 had a "Bright Aluminum" finish. Footplates after 1/96 now have a black finish. For Front Riggings before 2/96, order in pairs to match.			