


Back Canes

ABS & Metal Curved/Contoura (CGT/CGTE), (CGT/CGTE) w/ BPO Option, (CGE) w/ Sling Back Uph.

NOTE: When Contoura Backs, ABS & Metal Curved Backs or Tilt w/ BPO Option is ordered, only the Straight Back Cane w/ Push Handle is used.

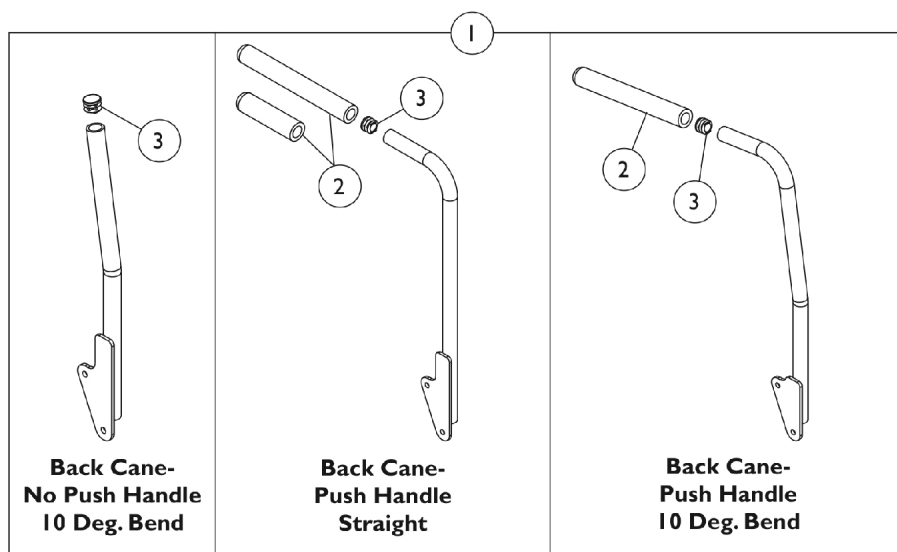

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Back Cane Assemblies	1	1
2		1135506	Package, Foam Hand Grip (9" L)	10	1
2	A	1086460	Grip, Foam, Hand (4-1/2" L)	1	1
3		1134426	Package, Plug Button, Black (3/4")	10	1

NOTE: A - Part number 1086460 is used when a vent tray is ordered on a tilt only system. This allows the double spreader bar to be attached to the 20, 21, 22" back canes. 23 and 24" high back canes do not require the shorter foam grip

Back Canes

ABS & Metal Curved/Contoura (CGT/CGTE), (CGT/CGTE) w/ BPO Option, (CGE) w/ Sling Back Uph.

NOTE: When Contoura Backs, ABS & Metal Curved Backs or Tilt w/ BPO Option is ordered, only the Straight Back Cane w/ Push Handle is used.

#1 Back Cane Assemblies

Back Height	Back Cane- No Push Handle 10 Deg. Bend Right	Back Cane- No Push Handle 10 Deg. Bend Left	Back Cane- Push Handle Straight Right	Back Cane- Push Handle Straight Left	Back Cane- Push Handle 10 Deg. Bend Right	Back Cane- Push Handle 10 Deg. Bend Left
16"	1134066	1134067	1133793	1133794	1133822	1133823
17"	1134068	1134069	1133795	1133796	1133824	1133825
18"	1134070	1134071	1143334 See Note(s) B	1143335 See Note(s) B	1133826	1133827
19"	1134072	1134073	1133799	1133800	1133828	1133829
20"	1134074	1134075	1133801 See Note(s) C, E	1133802 See Note(s) C, E	1133830	1133831
20" For Vent	N/A	N/A	1148666-NLA See Note(s) D	1148667-NLA See Note(s) D	N/A	N/A
21"	1134076	1134077	1136011 See Note(s) A	1136012 See Note(s) A	1133832	1133833
21" For Vent	N/A	N/A	1148668-NLA See Note(s) D	1148669-NLA See Note(s) D	N/A	N/A
22"	1134078	1134079	1136013 See Note(s) A	1136014 See Note(s) A	1133834	1133835
22" For Vent	N/A	N/A	1148670-NLA See Note(s) D	1148671-NLA See Note(s) D	N/A	N/A
23"	1134080	1134081	1136015 See Note(s) A	1136016 See Note(s) A	1133837	1133838
24"	1134082	1134083	1136017 See Note(s) A	1136018 See Note(s) A	1133839	1133840
25"	N/A	N/A	1160328 See Note(s) A	1160329 See Note(s) A	N/A	N/A
26"	N/A	N/A	1160330 See Note(s) A	1160331 See Note(s) A	N/A	N/A

NOTE: A - 21"-26" high straight back canes with push handles are only offered On-Chair when a BPO Option is ordered
 B - Wheelchairs manufactured Before 5/15/09 with a Contoura Back 25" or 29" high and wheelchairs manufactured After 5/14/09 with a Curve Contoura Back 22" high will use the 18" high Straight Back Cane w/ Push Handle.
 C - All (20"-24" high ABS Curved Backs) or (20"-26" high Metal Curved Backs) ordered On-Chair will use the 20" high Straight Cane w/ Push Handle
 D - This back cane with a short foam grip is used for Vent Tray Applications only. This allows the Double Spreader Bar to be attached. 23 and 24" high back canes do not require the shorter foam grip
 E - Wheelchairs manufactured After 5/14/09 with a Curve Contoura Back 25" or 28" high will use the 20" high Straight Cane w/ Push Handle

NOTE: Back Cane Assemblies include Foam Grips and Plug Buttons where shown