


Footrest Support Assembly

H93E - Black, Chrome, and Color

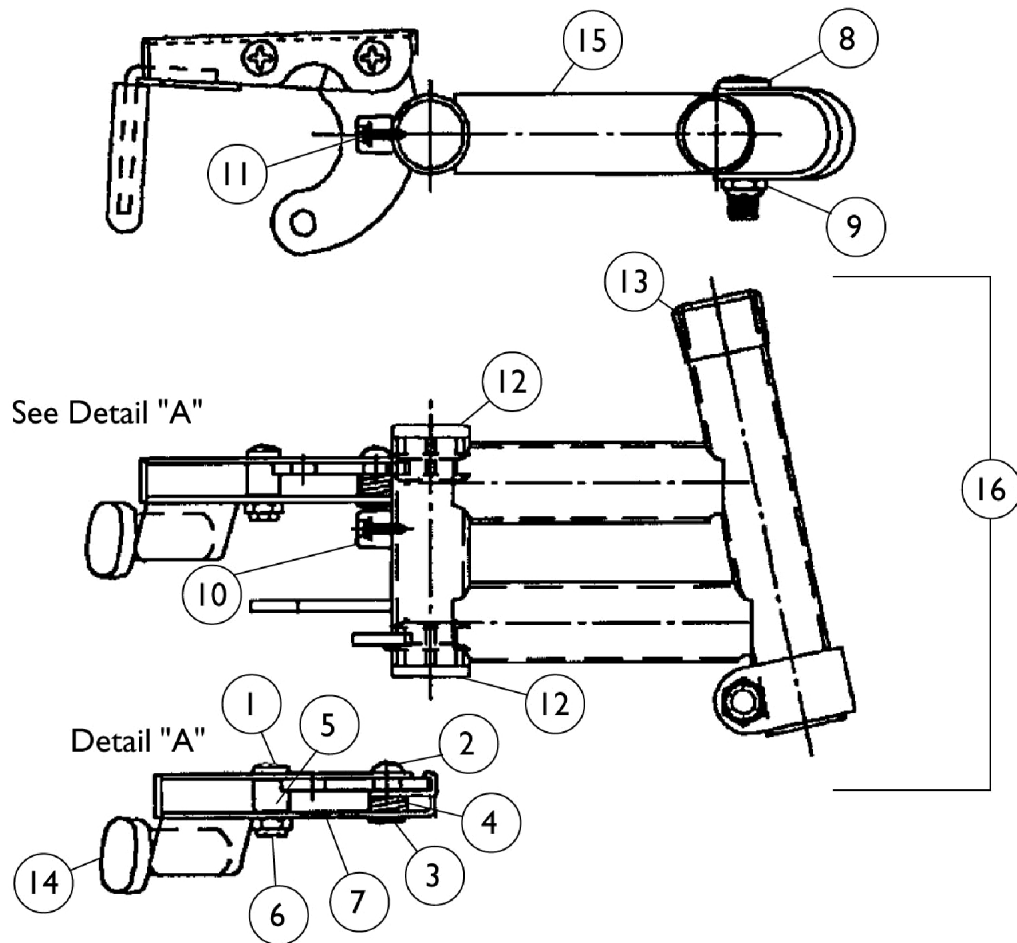

Footrest Support Assembly

H93E - Black, Chrome, and Color

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
2		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
3		67005X000	Rivnut (#10-32)	1	1
4		1008935	Spring, Release - Right	1	1
4		1008936	Spring, Release - Left	1	1
5		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
6		7930365-NLA	NLA - Locknut (#10-32)	25	1
7		1035539	Handle, Release, Black - Right	1	1
7		1035540	Handle, Release, Black - Left	1	1
8		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
9		1107321	Nut, Low Cap (5/16"-24)	1	1
10		1022874	Bumper	1	1
11		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
12		1027095	Plug Button, Black (7/8")	1	2
13		15680X012	Collar, Anti-Rattle (7/8")	1	1
14	B	1006490	Tip, Plastic, Black	1	1
15		1037856-NLA	NLA - Tube, Footrest Support, Hemi, Black - Right	1	1
15		1037857-NLA	NLA - Tube, Footrest Support, Hemi, Black - Left	1	1
15		1035144-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Right	1	1
15		1035145-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Left	1	1
15	A	8881037860-NLA	NLA - Tube, Footrest Support, Hemi, Color - Right	1	1
15	A	8881037861-NLA	NLA - Tube, Footrest Support, Hemi, Color - Left	1	1
16	C	1037858-NLA	NLA - Footrest Support Assembly, Hemi, Black - Right	1	1
16	C	1037859-NLA	NLA - Footrest Support Assembly, Hemi, Black - Left	1	1
16	C	1035525-NLA	NLA - Footrest Support Assembly, Hemi, Chrome - Right	1	1
16	C	1035524-NLA	NLA - Footrest Support Assembly, Hemi, Chrome - Left	1	1
16	A,C	8881037862-NLA	NLA - Footrest Support Assembly, Hemi, Color - Right	1	1
16	A,C	8881037863-NLA	NLA - Footrest Support Assembly, Hemi, Color - Left	1	1
	D	See Chart	Pivot Slide Tubes with Aluminum Footplates	--	--

NOTE: A - Specify frame color when ordering, see Frame Finishes, Touch-Up Paints, Upholstery Options and Accent Packages page.
B - Before 10/1/00 only
C - Assembly includes Release Handle, Tip, Bolt, and Locknut.
D - Not Shown

Footrest Support Assembly

H93E - Black, Chrome, and Color

Pivot Slide Tubes with Aluminum Footplates

Description	Size	Right	Left
Adult and Narrow Adult and Large Adult Rubber Edged Plate)	(16" - 24")	1028905-NLA See Note(s) A	1028906-NLA
Slim Narrow (Plain Aluminum Plate)	(14" - 15")	1033506-NLA	1033507-NLA
NOTE: A - Front Riggings built before 12/93 had footplates with a flat black finish. After 11/93 the finish is "Wrinkle Black." For Front Riggings before 12/93, order in pairs to match.			