




MK5 SPJ+ Joystick and Mounting Hardware

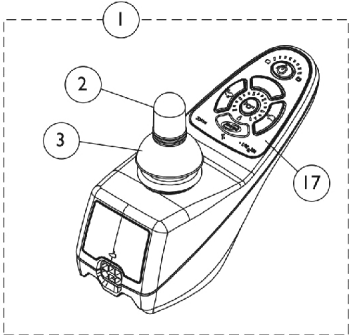


MK5 SPJ+ Joystick

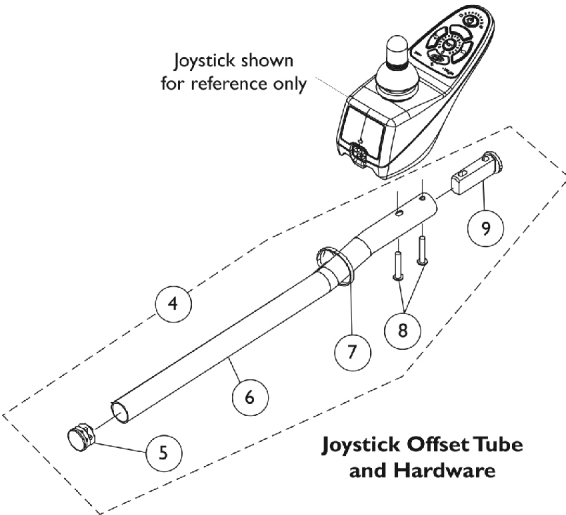


Joystick Offset Tube and Hardware

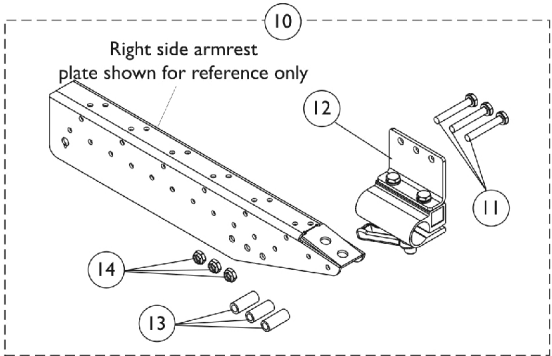

Joystick Tube Mounting Hardware

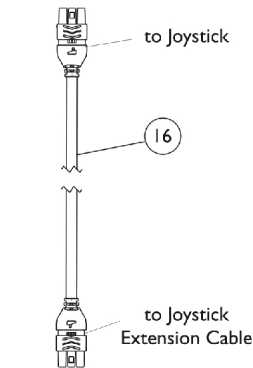

Joystick Cable

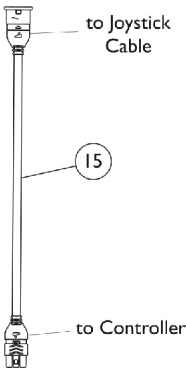

Joystick Extension Cable

MK5 SPJ+ Joystick and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,D,E	1127291	Joystick, SPJ+	1	1
2		1040217	Knob, Joystick Rubber, Black	1	1
3		1040218	Skirt, Joystick Rubber, Black	1	1
4	B	1117464	Kit, 2" Offset Joystick Tube with Hardware	1	1
5			Plug Button, Black (7/8")	1	1
6			Tube, Joystick Offset (2")	1	1
7			Tie Wrap (11-1/2" L)	1	1
8			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2
9			Spacer, Joystick Tube, Black	1	1
5		1130582	Package, Plug Button, Black (7/8")	10	1
7		1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	1
10	C	1115000	Kit, Joystick Tube Mounting Hardware - Captain Seat	1	1
11		1037610	Screw, Hex Head (1/4-20 x 1-5/8")	1	3
12		1105049	Clamp Assembly, Fixed Height, Captain/ Van Seat	1	1
13		1087219	Spacer (1/4 x 3/8 x 1")	1	3
14		1025195	Locknut (1/4-20)	1	3
15	E	1116404	Cable, Joystick Extension	1	1
16	E	1127293	Cable, Joystick SPJ+	1	1
17		1165094	Kit, Overlay, SPJ+ Joystick (MK5/MK6)	1	1
			NLA - Overlay, SPJ+ Joystick	1	1
			Instruction Sheet, MK5/MK6 Overlay	1	1
NOTE: A - Includes joystick and items 2,3 and 17. B - Includes item 5-9. C - Includes items 11-14. D - Battery Charger is plugged into the front port on the joystick to charge batteries. E - Used before 4/1/08.					