


Legrest Support Assembly

94BKR

(16" to 24" Wide Chairs)

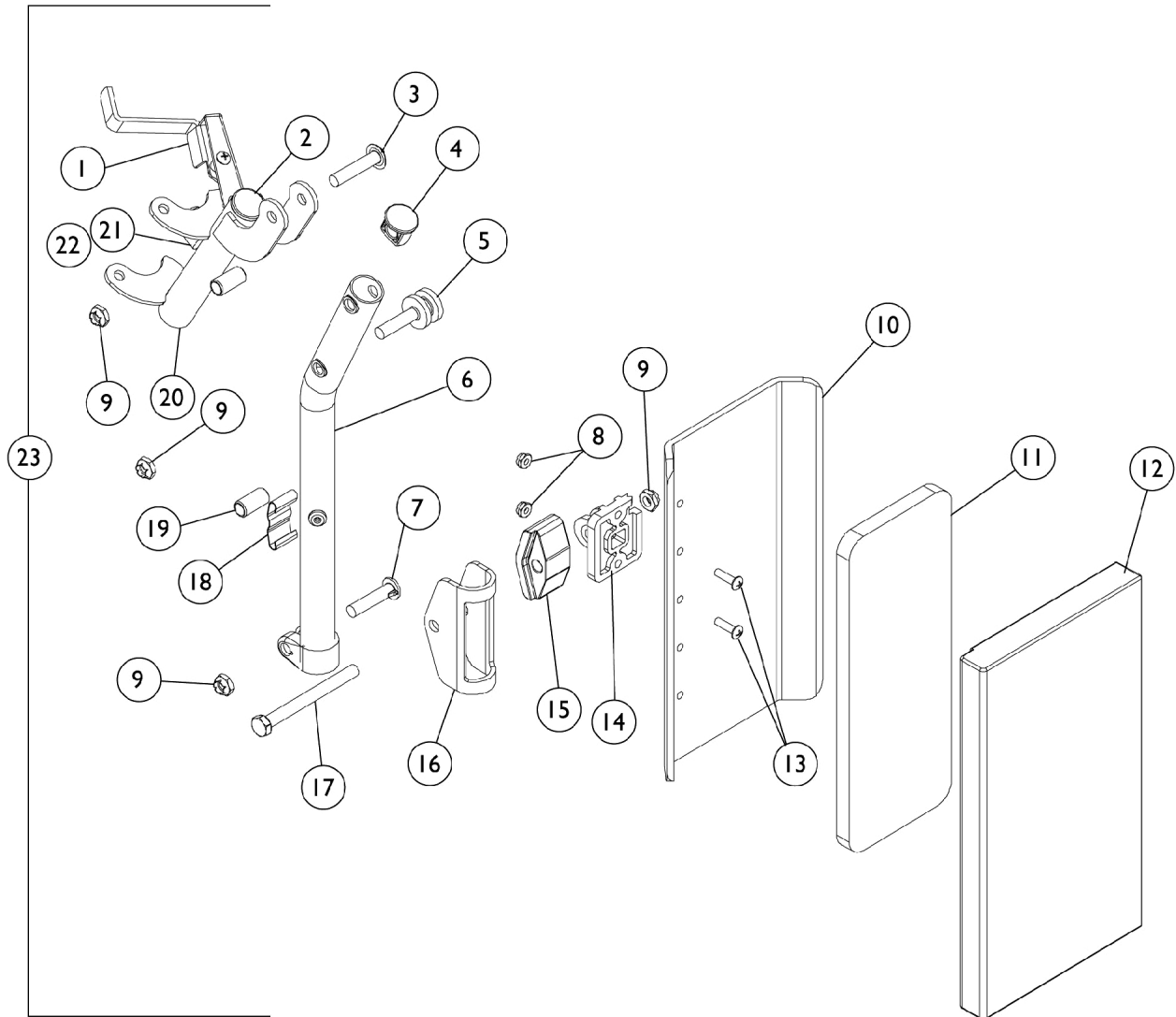

Legrest Support Assembly

94BKR

(16" to 24" Wide Chairs)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1035539	Handle, Release, Black - Right	1	1
1		1035540	Handle, Release, Black - Left	1	1
1	A	1031469-NLA	NLA - Handle, Release, Black - Right	1	1
1	A	1031470-NLA	NLA - Handle, Release, Black - Left	1	1
	B	7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
	B	67005X000	Rivnut (#10-32)	1	1
	B	1008935	Spring, Release - Right	1	1
	B	1008936	Spring, Release - Left	1	1
	B	47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
2		1027095	Plug Button, Black (7/8")	1	1
3		1030948	Screw, Slotted Truss Head (5/16-24 x 1-1/2")	1	1
4		1095433	Plug Button, Black (7/8")	1	1
5		1017021	Stud	1	1
6	C	1031358-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Right	1	1
6	C	1031359-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Left	1	1
6	D	1149473-NLA	NLA - Tube, Support, Legrest, Elevating, Silver Vein - Right	1	1
6	D	1149474-NLA	NLA - Tube, Support, Legrest, Elevating, Silver Vein - Left	1	1
7		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
8		7930365-NLA	NLA - Locknut (#10-32)	25	2
9		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
10		47071X405-NLA	NLA - Trough, Leg	1	1
11		43022X182-NLA	NLA - Foam	1	1
12		1002848-NLA	NLA - Cover	1	1
13		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	2
14		04526X019	Plate, Legrest Pad	1	1
15		1091220	Spacer, Legrest	1	1
16		1025104	Bracket, Legrest, Chrome	1	1
17		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
18		48823X008	Spring, Leaf	1	1
19		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
20		1031581-NLA	NLA - Legrest Support Tube Assembly, Adult, Black - Right	1	1
20		1031582	Legrest Support Tube Assembly, Adult, Black - Left	1	1
20	A,E	1033965	Legrest Support Tube Assembly, Adult, Silver Vein - Right	1	1
20	A,E	1033966	Legrest Support Tube Assembly, Adult, Silver Vein - Left	1	1
21		1022874	Bumper	1	1

Legrest Support Assembly

94BKR

(16" to 24" Wide Chairs)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
22		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
23	E	1031583-NLA	NLA - Legrest Support Assembly, Adult, Silver Vein - Right	1	1
23	E	1031584-NLA	NLA - Legrest Support Assembly, Adult, Silver Vein - Left	1	1
23	A,E	1034602-NLA	NLA - Legrest Support Assembly, Adult, Silver Vein - Right	1	1
23	A,E	1034603-NLA	NLA - Legrest Support Assembly, Adult, Silver Vein - Left	1	1
	B	See Chart	Pivot Slide Tube with Aluminum Footplate	1	1
NOTE: A - Only used on Power Back Recliners with Multi-Function Frames. B - Not Shown C - Before 7/6/07 D - After 7/5/07 E - After 7/5/07 Color changed from chrome to silver vein. It is recommended to order in pairs to match					

Pivot Slide Tube with Aluminum Footplate

Description	Size	Right	Left
Adult, Narrow Adult and Large Adult (Rubber Edged Plate)	(16" - 24")	1031216-NLA See Note(s) A	1031217-NLA See Note(s) A
Slim Narrow (Plain Aluminum Plate)	(14" - 15")	1030912-NLA See Note(s) B	1030913-NLA See Note(s) B
Extra Large (Black Aluminum Plate)	16" - 24")	1031934-NLA	1031935-NLA
NOTE: A - Front Riggings built before 12/93 had footplates with a flat black finish. After 11/93 the finish is "Wrinkle Black." For Front Riggings before 12/93, order in pairs to match. B - Aluminum footplates on chairs built before 2/96 had a "Bright Aluminum" finish. Footplates after 1/96 now have a black finish. For Front Riggings before 2/96, order in pairs to match.			