


Clothing Guard - Fold In Style Side Guard

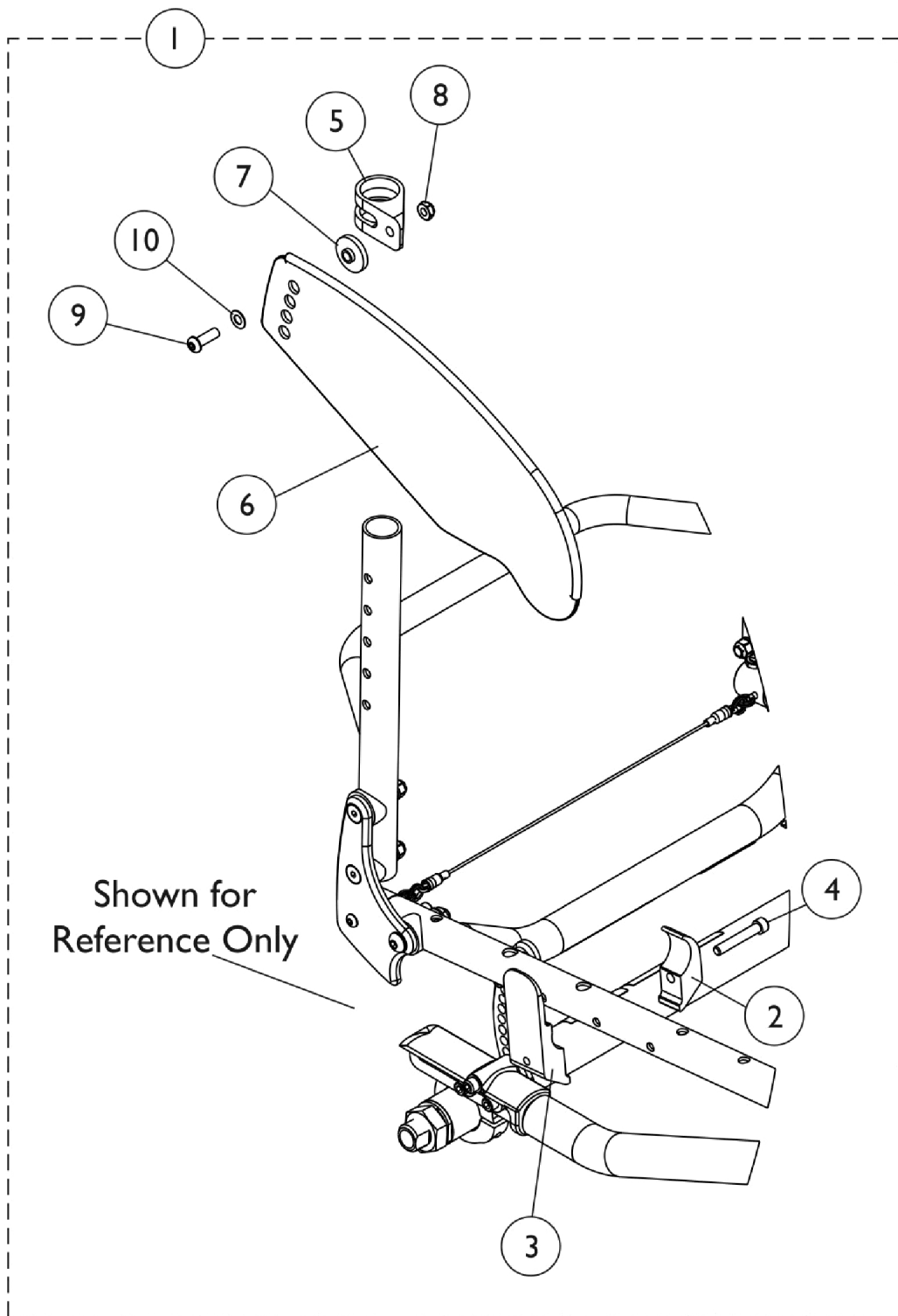

Clothing Guard - Fold In Style Side Guard

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1160550-NLA	NLA - Kit, Guard, Side, Folding, Aluminum, Black - Small	1	1
2			NLA - Clamp, Wedge	1	1
3			NLA - Clamp, Wedge	1	1
4			NLA - Screw, Socket Head (1/4-28 x 1-1/2")	1	1
5			NLA - Bracket, Swing	1	1
6			NLA - Side Guard w/ Edge Trim Assembly, Folding, Aluminum, Black - Small	1	1
7			Spacer (1/4 x 1 x 17/64")	1	1
8			Locknut (1/4-28)	1	1
9			NLA - Screw, Socket, Button Head (1/4-28 x 3/4")	1	1
1	A	1160551-NLA	NLA - Kit, Guard, Side, Folding, Aluminum, Black - Large	1	1
2			NLA - Clamp, Wedge	1	1
3			NLA - Clamp, Wedge	1	1
4			NLA - Screw, Socket Head (1/4-28 x 1-1/2")	1	1
5			NLA - Bracket, Swing	1	1
6			NLA - Side Guard w/ Edge Trim Assembly, Folding, Aluminum, Black - Large	1	1
7			Spacer (1/4 x 1 x 17/64")	1	1
8			Locknut (1/4-28)	1	1
9			NLA - Screw, Socket, Button Head (1/4-28 x 3/4")	1	1
1	B	1160543-NLA	NLA - Kit, Guard, Side, Folding, Carbon Fiber, Black - Small	1	1
2			NLA - Clamp, Wedge	1	1
3			NLA - Clamp, Wedge	1	1
4			NLA - Screw, Socket Head (1/4-28 x 1-1/2")	1	1
5			NLA - Bracket, Swing	1	1
6			NLA - Side Guard w/ Edge Trim Assembly, Folding, Carbon Fiber, Black - Small	1	1
7			Spacer (1/4 x 1 x 17/64")	1	1
8			Locknut (1/4-28)	1	1
9			NLA - Screw, Socket, Button Head (1/4-28 x 3/4")	1	1
10			Washer (1/4 x 1/2 x 1/40)	1	1
1	B	1160544-NLA	NLA - Kit, Guard, Side, Folding, Carbon Fiber, Black - Large	1	1
2			NLA - Clamp, Wedge	1	1
3			NLA - Clamp, Wedge	1	1
4			NLA - Screw, Socket Head (1/4-28 x 1-1/2")	1	1
5			NLA - Bracket, Swing	1	1
6			NLA - Side Guard w/ Edge Trim Assembly, Folding, Carbon Fiber, Black - Large	1	1
7			Spacer (1/4 x 1 x 17/64")	1	1
8			Locknut (1/4-28)	1	1
9			NLA - Screw, Socket, Button Head (1/4-28 x 3/4")	1	1
10			Washer (1/4 x 1/2 x 1/40)	1	1
6		1161980-NLA	NLA - Side Guard w/ Edge Trim Assembly, Folding, Aluminum, Black - Small	1	1

Clothing Guard - Fold In Style Side Guard

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
6		1161981-NLA	NLA - Side Guard w/ Edge Trim Assembly, Folding, Aluminum, Black - Large	1	1
6		1161982-NLA	NLA - Side Guard w/ Edge Trim Assembly, Folding, Carbon Fiber, Black - Small	1	1
6		1161983-NLA	NLA - Side Guard w/ Edge Trim Assembly, Folding, Carbon Fiber, Black - Large	1	1
NOTE: A - Includes items 2-9. B - Includes items 2-10.					