


Mast and Ball Screw Assembly

Model 1200, 1300, 1500, 1600, 1800 Service Parts

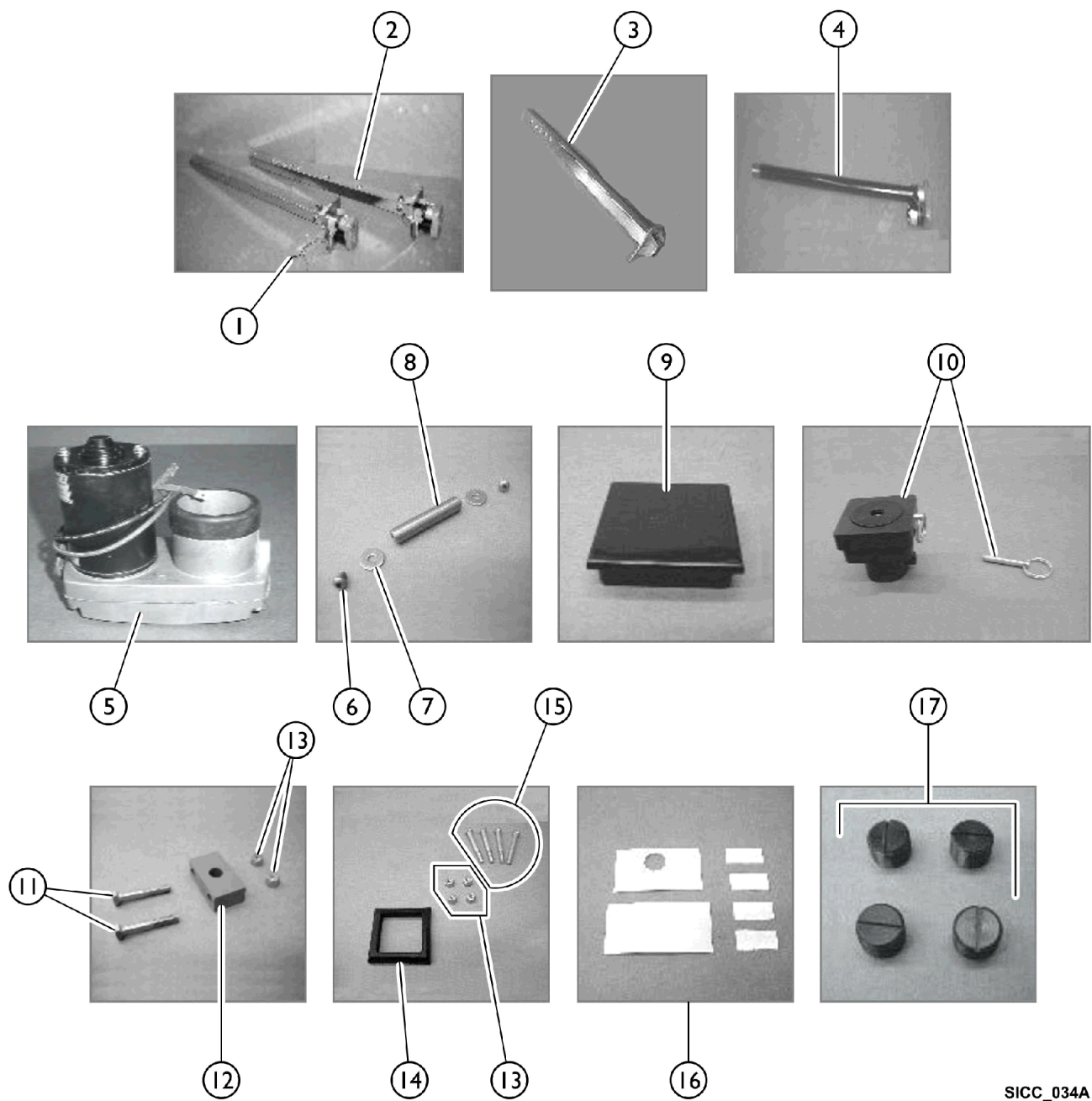

Mast and Ball Screw Assembly

Model 1200, 1300, 1500, 1600, 1800 Service Parts

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		IHTRP1010-NLA	NLA - Outer Column Only	1	1
1		TRP1010-2R	Reverse Inner/Outer Column & Ball Screw Assembly (1500 & 1600 Lifts)	1	1
1		IHTRP1008R	Reverse Inner Column Only	1	1
2		TRP1010-2-NLA	NLA - Inner/Outer Column Ball and Screw Assembly	1	1
2		IHTRP1008-NLA	NLA - Inner Column Only	1	1
2		IHTRP1010-NLA	NLA - Outer Column Only	1	1
3		IHTRP1010C	Stationary Outer Column	1	1
4		TRP1030X	Ball Screw Assembly	1	1
5		IHTRP1820	Ball Screw Motor	1	1
6		IHTRP1065-NLA	NLA - Bolt, Motor Box, Stainless (1/4-20 x 3/8")	1	2
7		IHTRP1082-NLA	NLA - Washer	1	2
8		IHTRP1015-NLA	NLA - Lift Pin	1	1
9		IHTRP1019-NLA	NLA - Column Cap	1	1
10	A	IHTRP1007-NLA	NLA - Emergency Crank Cap	1	1
10	A	IHTRP1007D-NLA	NLA - Replacement Pin	1	1
11		IHTRP1061-NLA	NLA - Bolt, Trunion Block (5/16-18 x 2")	1	2
12		ITHRP1005	Trunion Block	1	1
13		IHTRP1070-NLA	NLA - Nut, Nylon (5/16-18)	1	6
14		IHTRP1016A-NLA	NLA - Column Seal	1	1
15		IHTRP1060-NLA	NLA - Bolt, Column Seal (5/16-18 x 2-3/4")	1	4
16		TRP1041KIT	Bearing Tape Kit	6	1
17		IHTRP1073-NLA	NLA - Bearing Screws	1	4

NOTE: A - Optional part.