


Model 1200 - Stretcher Assembly

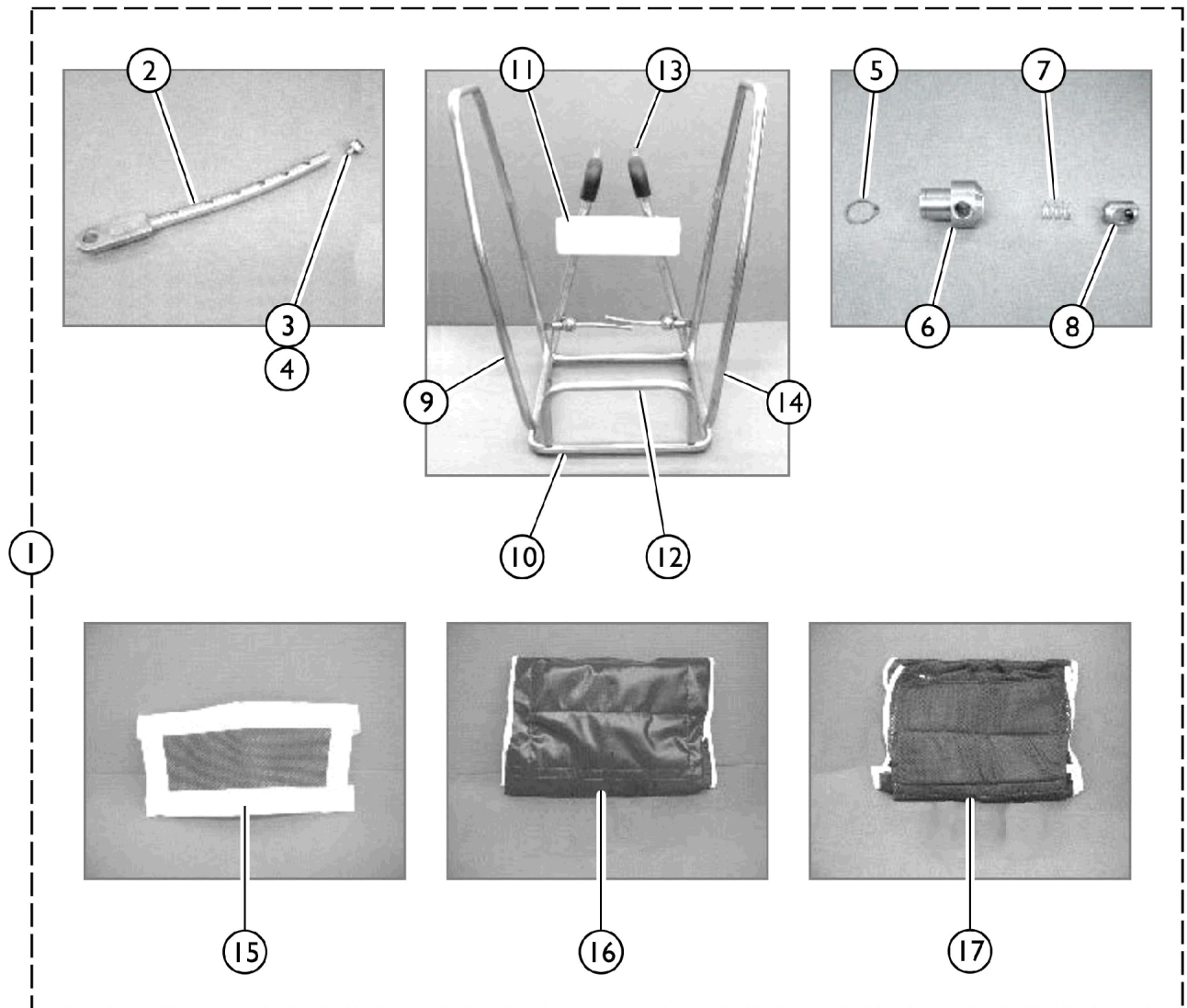

Model 1200 - Stretcher Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	IHTRV1201	Stretcher Assembly w/ Cover (Ref. Only)	1	1
1	A	IHTRV1201N	Stretcher Assembly w/ Cover - Narrow (Ref. Only)	1	1
2		IHTRP1110-NLA	NLA - Adjusting Rod	1	1
3		IHTRP1112S-NLA	NLA - Adjusting Rod Bolt and Washer	1	1
4		TRP1112T-NLA	NLA - Washer, Adjustment	1	1
5		IHTRP1074-NLA	NLA - Snap Ring	1	1
6		IHTRP1108-NLA	NLA - Adjusting Rod Housing	1	1
7		IHTRP1173-NLA	NLA - Adjusting Rod Spring	1	1
8		IHTRP1109-NLA	NLA - Adjusting Rod Plunger	1	1
9		IHTRP1102R-NLA	NLA - Right Stretcher End Section	1	1
10		IHTRP1103-NLA	NLA - Stretcher Hanger Rod	1	1
11	B	TRV1305W-NLA	NLA - Stretcher Backrest, White	1	1
11	B	TRV1305A-NLA	NLA - Stretcher Backrest, Almond	1	1
12		IHTRP1202-NLA	NLA - Stretcher Body Rest - 12" Standard	1	1
13		IHTRP1034B-NLA	NLA - Tube, Foam (7/8" x 6")	1	1
14		IHTRP1102L-NLA	NLA - Left End Stretcher Section	1	1
15		IHTRV1213-NLA	NLA - Stretcher Back Pad	1	1
16		IHTRV1210-NLA	NLA - Blue Mesh Stretcher Cover	1	1
16		IHTRV1210N-NLA	NLA - Blue Mesh Stretcher Cover - Narrow	1	1
17		IHTRV1211-NLA	NLA - Solid Stretcher Cover	1	1
17		IHTRV1211N-NLA	NLA - Solid Stretcher Cover - Narrow	1	1
NOTE: A - Includes Items 2 thru 17. B - Specify color, almond or white when ordering.					