
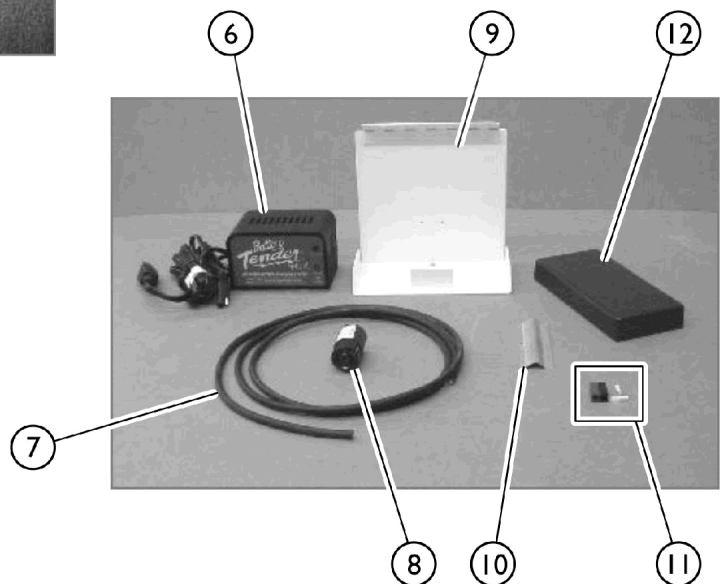


# Battery and Battery Charger Assembly

## Model 1200, 1300, 1500, 1600, 1800 Service Parts

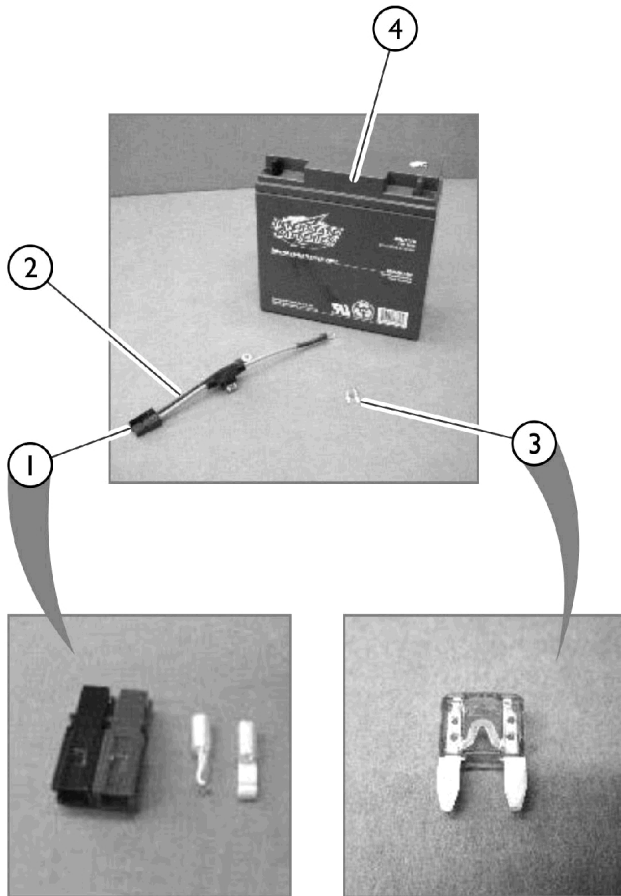

SICC\_036A

# Battery and Battery Charger Assembly

## Model 1200, 1300, 1500, 1600, 1800 Service Parts

| Item Number                                                                                         | Note | Part Number    | Description                             | Quantity Per |          |
|-----------------------------------------------------------------------------------------------------|------|----------------|-----------------------------------------|--------------|----------|
|                                                                                                     |      |                |                                         | Package      | Assembly |
| 1                                                                                                   |      | IHTRP1083      | Wire Connectors - Red and Black         | 1            | 1        |
| 2                                                                                                   |      | IHTRP1087-NLA  | NLA - Battery Harness Assembly          | 1            | 1        |
| 3                                                                                                   |      | IHTRP1085-NLA  | NLA - 30 Amp Fuse (For Battery Harness) | 1            | 1        |
| 4                                                                                                   |      | IHTRP1051-NLA  | NLA - Battery                           | 1            | 1        |
| 5                                                                                                   |      | IHTRP1052W-NLA | NLA - Battery Charger Assembly, White   | 1            | 1        |
| 5                                                                                                   |      | IHTRP1052A-NLA | NLA - Battery Charger Assembly, Almond  | 1            | 1        |
| 5                                                                                                   |      | IHTRP1052C-NLA | NLA - Battery Charger Assembly - 220V   | 1            | 1        |
| 6                                                                                                   |      | IHTRP1052D     | Battery Charger Only                    | 1            | 1        |
| 7                                                                                                   |      | IHTRP1033      | Charger Cord - 6 ft.                    | 1            | 1        |
| 8                                                                                                   |      | IHTRP1033C     | Hubble Plug                             | 1            | 1        |
| 9                                                                                                   |      | TRP1002XW-NLA  | NLA - Battery Housing - White           | 1            | 1        |
| 10                                                                                                  |      | IHTRP1003-NLA  | NLA - Wire Guard                        | 1            | 1        |
| 11                                                                                                  | B    | IHTRP1083      | Wire Connectors - Red and Black         | 1            | 1        |
|                                                                                                     |      | IHTRP1083A-NLA | NLA - Wire Nut, Black                   | 1            | 1        |
|                                                                                                     |      | IHTRP1083B-NLA | NLA - Wire Nut, Red                     | 1            | 1        |
|                                                                                                     |      | IHTRP1083C-NLA | NLA - Contact, Loose (30 Amp)           | 1            | 1        |
| 12                                                                                                  | A    | IHTRP1011      | Battery Cover                           | 1            | 1        |
|                                                                                                     |      | IHTRP1040-NLA  | NLA - Grip Tape                         | 1            | 1        |
| NOTE: A - Not Shown in Photos.<br>B - Includes Wire Nut - Red, Wire Nut - Black, and Loose Contact. |      |                |                                         |              |          |