


# Adjustable Angle Footplates w/ Pivot Tube - AT5543

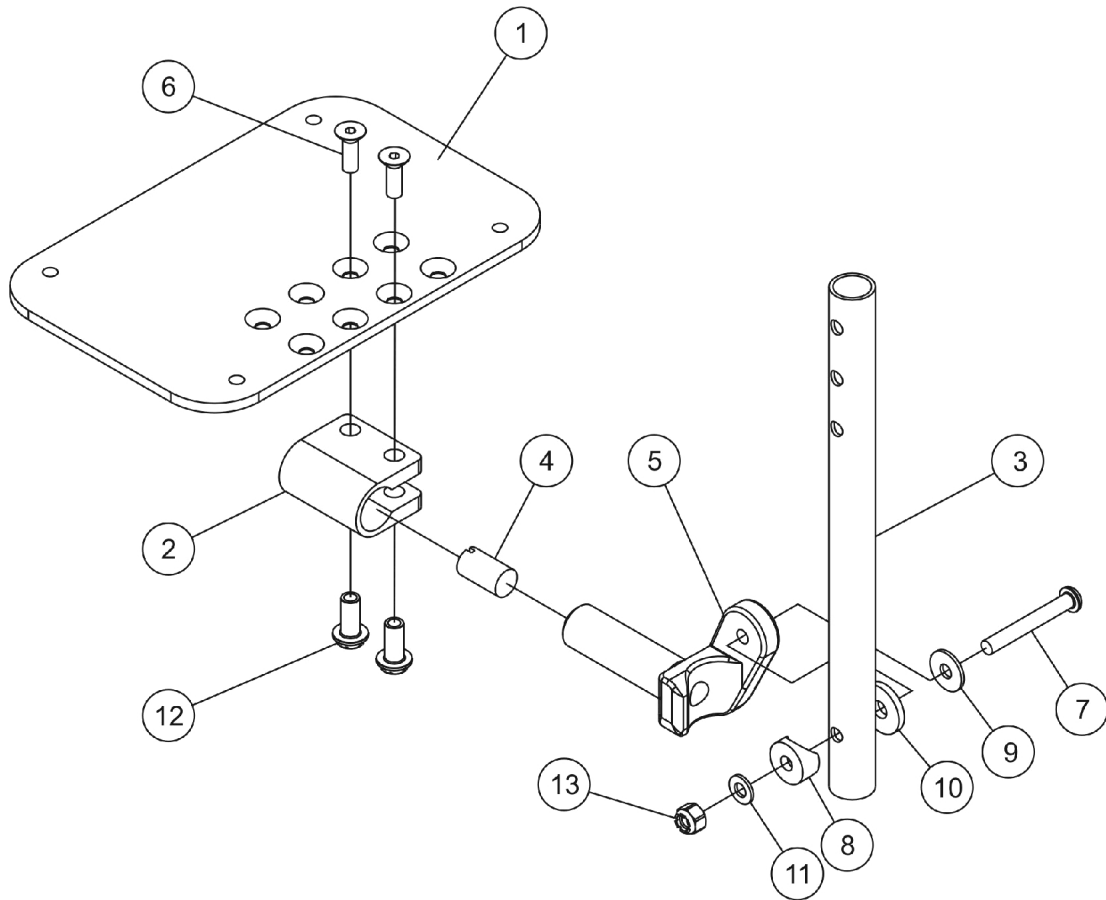

## Adjustable Angle Footplates w/ Pivot Tube - AT5543

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D,F	1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
1	E,F	1113575	Footplate, Adjustable Angle, XX- Large, Black (7-1/4" W x 8" L)	1	1
2		1058517	Clamp, Footplate	1	1
3		1057855	Tube, Pivot and Slide, Straight, Steel, Black (9-7/8" L)	1	1
3	A	1057856	Tube, Pivot/Slide, Straight, Steel, Black (11-7/8" L)	1	1
3	B	1121135	Tube, Pivot/Slide, Straight, Steel, Black (13-7/8" L)	1	1
4		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
5		1052102	Hinge, Footplate - Right	1	1
5		1052103	Hinge, Footplate - Left	1	1
6		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
7		1022831	Screw, Socket Button Head (1/4-20 x 1-7/8")	1	1
8		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
9		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
10		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
11		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
12		1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
13		1025736	Locknut (1/4-20)	1	1
	C	1060474	Instruction Sheet	1	1

NOTE: A - ALPT 2" Longer Pivot and Slide Tube  
 B - ALPT4 4" Longer Pivot and Slide Tube  
 C - Not Shown  
 D - 18-22" Wide chairs  
 E - 23-24" Wide chairs  
 F - This footplate was revised 9/2/2008 with rounder corners and accessory holes changed from 8 to 4. Recommend to replace footplate in pairs to match sides.