


Adjustable Angle Footplate Assemblies - AT5543

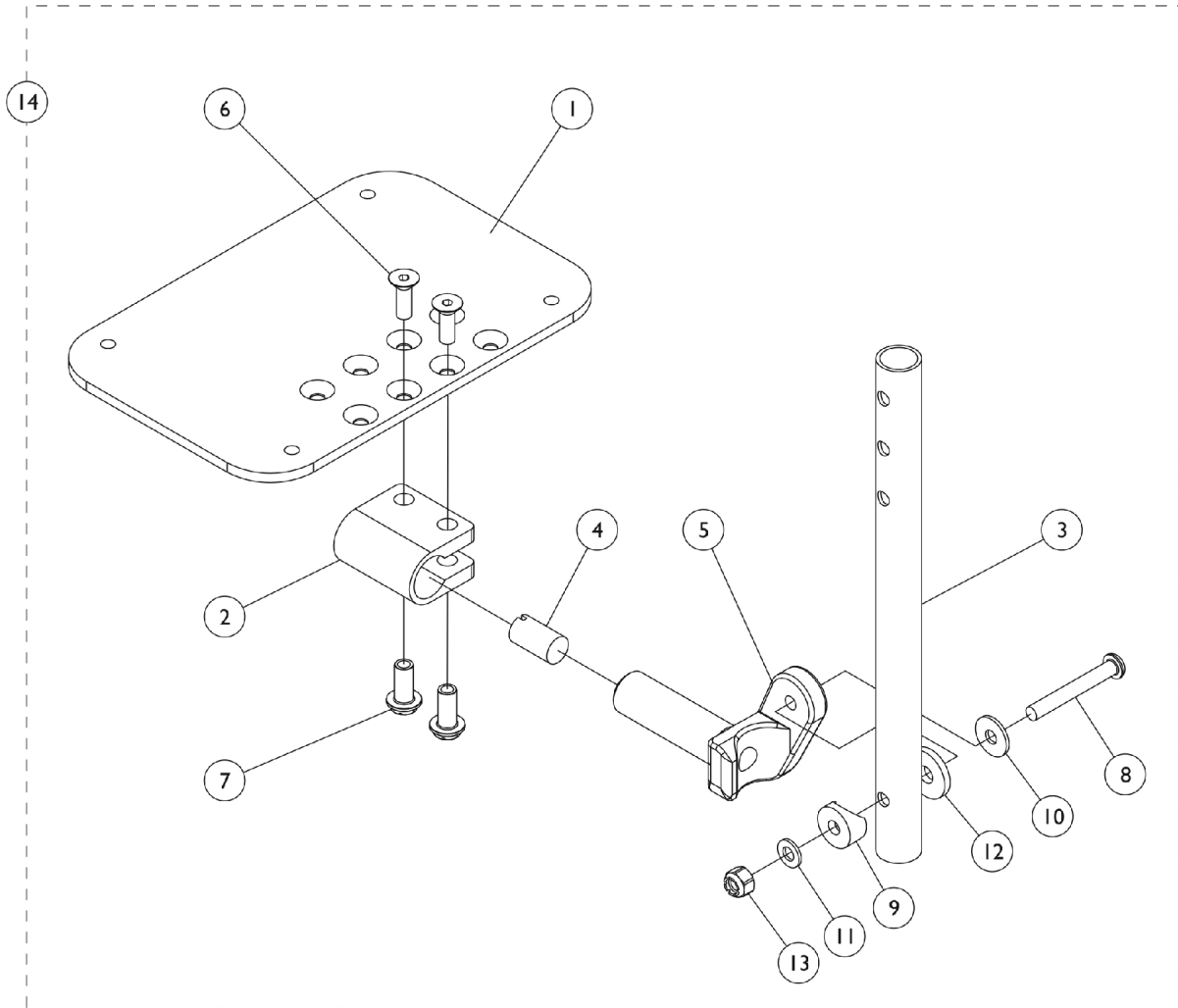

Adjustable Angle Footplate Assemblies - AT5543

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D	1113571	Footplate, Adjustable Angle, Small, Black (3-1/4" W x 8" L)	1	1
1	D	1113572	Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
1	D	1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
1	D	1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
1	D	1113575	Footplate, Adjustable Angle, XX- Large, Black (7-1/4" W x 8" L)	1	1
2		1058517	Clamp, Footplate	1	1
3	A	1120384-NLA	NLA - Tube, Pivot/Slide, Straight - Alum. (9-7/8" L)	1	1
3	A	1120385	Tube, Pivot/Slide, Straight - Black (11-7/8" L)	1	1
3	B	1091745-NLA	NLA - Tube, Pivot/Slide, Straight, Steel, Chrome (13-7/8" L)	1	1
4		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
5		1052102	Hinge, Footplate - Right	1	1
5		1052103	Hinge, Footplate - Left	1	1
6		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
7		1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
8		1022831	Screw, Socket Button Head (1/4-20 x 1-7/8")	1	1
9		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
10		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
11		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
12		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
13		1025736	Locknut (1/4-20)	1	1
14	E	See Chart	60 Degree and 80 Degree Fixed Front	1	1
14	E	See Chart	60 Degree, 60 Degree MFX, 70 Degree Non-Tapered and ELR's	1	1
14	E	See Chart	60 Degree, 60 Degree MFX, 70 Degree Non-Tapered and ELR's - ALPT 2" Longer	1	1
14	E	See Chart	70 Degree Tapered	1	1
14	E	See Chart	70 Degree Tapered - ALPT 2" Longer	1	1
14	E	See Chart	90 Degree and 70 Degree MFX	1	1

NOTE: A - Swingaway Front Riggings
 B - Articulating Legrest
 D - This footplate was revised 9/2/2008 with rounder corners and accessory holes changed from 8 to 4. Recommended to replace footplate in pairs to match sides.
 E - Includes Items 1 thru 13.

#14 60 Degree and 80 Degree Fixed Front (Style chairs)

Straight Slide Tube (5-1/2")	Right	Left
14-15" Wide (3-1/4" Footplate)	1061974-NLA	1061975-NLA
16-17" Wide (4-1/4" Footplate)	1061976-NLA See Note(s) A	1061977-NLA See Note(s) A
18-19" Wide (5-1/4" Footplate)	1061978-NLA	1061979-NLA

Adjustable Angle Footplate Assemblies - AT5543

#14 60 Degree and 80 Degree Fixed Front (Style chairs)

Straight Slide Tube (5-1/2")	Right	Left
20" Wide (6-1/4" Footplate)	1062586	1062587
NOTE: A - Adjustable Angle Footplates 1061976 Rt./ 1061977 Lt. are NLA. Order 1060436 Rt./1060437 Lt. with 1061967 and have assembled.		

#14 60 Degree, 60 Degree MFX, 70 Degree Non-Tapered and ELR's

Straight Slide Tube (9-7/8")	Right	Left
10-11" Wide (3-1/4" Footplate)	1060428-NLA	1060429-NLA
12-13" Wide (4-1/4" Footplate)	1060430	1060431
14-15" Wide (5-1/4" Footplate)	1060432	1060433
16-20" Wide (6-1/4" Footplate)	1062572	1062573
18-22" Wide (7-1/4" Footplate)	1068620	1068621

Adjustable Angle Footplate Assemblies - AT5543

#14 60 Degree, 60 Degree MFX, 70 Degree Non-Tapered and ELR's - ALPT 2" Longer

ALPT Straight Slide Tube (Aluminum 11-7/8")	Right	Left
10-11" Wide (3-1/4" Footplate)	1062574-NLA	1062575-NLA
12-13" Wide (4-1/4" Footplate)	1062576-NLA	1062577-NLA
14-15" Wide (5-1/4" Footplate)	1060440-NLA	1060441
16-18" Wide (6-1/4" Footplate)	1062578	1062579
19-22" Wide (7-1/4" Footplate)	1068622	1068623

#14 70 Degree Tapered

Straight Slide Tube (9-7/8")	Right	Left
13-14" Wide (3-1/4" Footplate)	1060428-NLA	1060429-NLA
15-16" Wide (4-1/4" Footplate)	1060430	1060431
17-18" Wide (5-1/4" Footplate)	1060432	1060433
19-22" Wide (6-1/4" Footplate)	1062572	1062573

#14 70 Degree Tapered - ALPT 2" Longer

ALPT Straight Slide Tube (Aluminum 11-7/8")	Right	Left
13-14" Wide (3-1/4" Footplate)	1062574-NLA	1062575-NLA
15-16" Wide (4-1/4" Footplate)	1062576-NLA	1062577-NLA
17-18" Wide (5-1/4" Footplate)	1060440-NLA	1060441
19-22" Wide (6-1/4" Footplate)	1062578	1062579

Adjustable Angle Footplate Assemblies - AT5543

#14 Adjustable Angle Footplates w/o Pivot Tubes

90 Degree and 70 Degree MFX

No Slide Tube Bolts Through Support	Right	Left
10-11" Wide (3-1/4" Footplate)	1060434-NLA	1060435-NLA
12-13" Wide (4-1/4" Footplate)	1060436-NLA	1060437
14-15" Wide (5-1/4" Footplate)	1060438	1060439
16-17" Wide (6-1/4" Footplate)	1062570	1062571