


Adjustable Angle Footplate Assemblies - AT5543

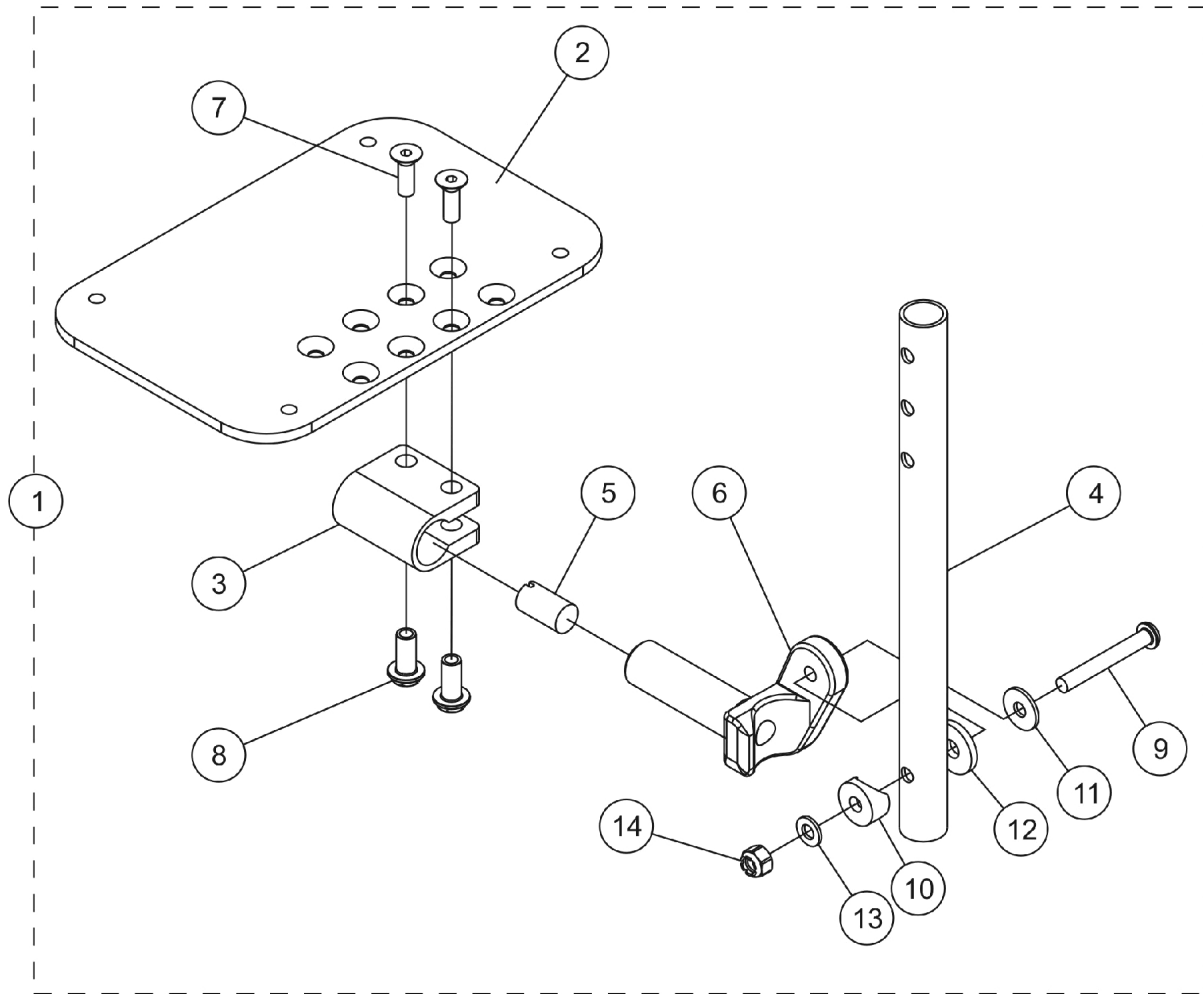

Adjustable Angle Footplate Assemblies - AT5543

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D	See Chart	Adjustable Angle Footplate Assembly	1	1
2	E	1113571	Footplate, Adjustable Angle, Small, Black (3-1/4" W x 8" L)	1	1
2	E	1113572	Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
2	E	1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
2	E	1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
3	A	1058517	Clamp, Footplate	1	1
4	B	1061967	Tube, Pivot/Slide, Straight, Black (5-1/2")	1	1
4	C	1069350-NLA	NLA - Tube, Pivot/Slide, Curved V Front, Black (5-1/2")	1	1
5		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
6		1052102	Hinge, Footplate - Right	1	1
6		1052103	Hinge, Footplate - Left	1	1
7	A	1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
8	A	1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
9		1022831	Screw, Socket Button Head (1/4-20 x 1-7/8")	1	1
10		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
11		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
12		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
14		1025736	Locknut (1/4-20)	1	1
13		1026907	Washer (1/4 x 1/2 x 1/16")	1	1

NOTE: A - Items 3, 7 and 8 must be ordered together for chairs manufactured before 8/10/99 due to a hardware change.
 B - Tapered Front Frame
 C - "V" Front Frame
 D - Includes Items 2 thru 14.
 E - This footplate was revised 9/2/2008 with rounder corners and accessory holes changed from 8 to 4. Recommended to replace footplates in pairs to match sides.

#1 Adjustable Angle Footplate Assembly

Chair Width	Tapered Front Frame -Right	Tapered Front Frame -Left	"V" Front Frame -Right	"V" Front Frame -Left
14-15" Wide (3-1/4" Footplate)	1061974-NLA	1061975-NLA	1069351-NLA	1069352-NLA
16-17" Wide (4-1/4" Footplate)	1061976-NLA See Note(s) A	1061977-NLA See Note(s) A	1069353-NLA	1069354-NLA
18-19" Wide (5-1/4" Footplate)	1061978-NLA	1061979-NLA	1069355-NLA	1069356-NLA
20" Wide (6-1/4" Footplate)	1062586	1062587	1069355-NLA	1069356-NLA

NOTE: A - Adjustable Angle Footplates 1061976 Rt./ 1061977 Lt. are NLA. Order 1060436 Rt./1060437 Lt. with 1061967 and have assembled.