


# Adjustable Angle Footplates - AT5543

## 60 Degree and 70 Degree Swingaway Supports w/ Straight Slide Tube

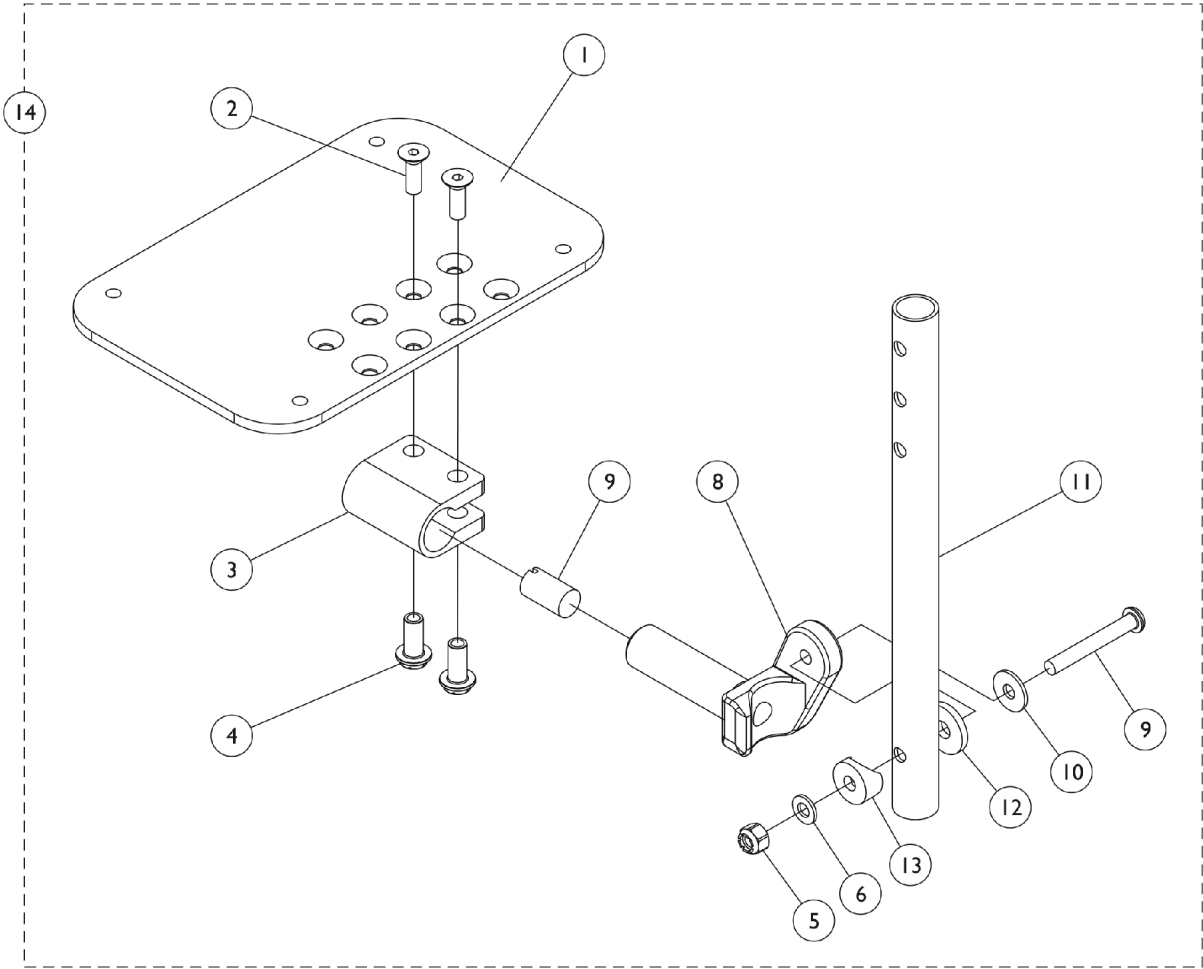

# Adjustable Angle Footplates - AT5543

## 60 Degree and 70 Degree Swingaway Supports w/ Straight Slide Tube

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	60 Degree and 70 Degree Non-Tapered and Elevating Legrest Assembly	1	1
1	A	See Chart	70 Degree Tapered Assembly	1	1
2	C	1113572	Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
2	C	1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
2	C	1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
3	B	1058517	Clamp, Footplate	1	1
4		1120384-NLA	NLA - Tube, Pivot/Slide, Straight - Alum. (9-7/8" L)	1	1
4		1120385	Tube, Pivot/Slide, Straight - Black (11-7/8" L)	1	1
5		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
6		1052102	Hinge, Footplate - Right	1	1
6		1052103	Hinge, Footplate - Left	1	1
7	B	1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
8	B	1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	1
9		7930321-NLA	NLA Package, Screw, Socket Button Head (1/4-20 x 1-7/8")	25	1
10		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	2
11		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
12		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	1
13		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
14		7930333	Locknut (1/4-20)	25	1

NOTE: A - Includes Items 2 thru 14.  
 B - Items 3, 7 and 8 must be ordered together for chairs manufactured before 8/10/99 due to a hardware change  
 C - This footplate was revised 9/2/2008 with rounder corners and accessory holes changed from 8 to 4. Recommended to replace footplate in pairs to match sides.

### #1 60 Degree and 70 Degree Non-Tapered and Elevating Legrest Assembly

Width	Slide Tube	Footplate Size	Right Side	Left Side
14"-15" Wide	Straight (9-7/8")	5-1/4" Footplate	1060432	1060433
16"-20" Wide	Straight (9-7/8")	6-1/4" Footplate	1062572	1062573
14"-15" Wide	ALPT Straight Tube (11-7/8") Aluminum	5-1/4" Footplate	1060440-NLA	1060441
16"-20" Wide	ALPT Straight Tube (11-7/8") Aluminum	6-1/4" Footplate	1062578	1062579
14"-15" Wide	ALPT Straight Tube (11-7/8") Steel	5-1/4" Footplate	1062758	1062759
16"-20" Wide	ALPT Straight Tube (11-7/8") Steel	6-1/4" Footplate	1062760	1062761

**Adjustable Angle Footplates - AT5543**  
**60 Degree and 70 Degree Swingaway Supports w/ Straight Slide Tube**  
**#1 70° Tapered Assembly**

Width	Slide Tube	Footplate Size	Right	Left
15"-16" Wide	Straight Slide Tube (9-7/8")	4-1/4" Footplate	1060430	1060431
17"-18" Wide	Straight Slide Tube (9-7/8")	5-1/4" Footplate	1060432	1060433
19"-20" Wide	Straight Slide Tube (9-7/8")	6-1/4" Footplate	1062572	1062573
15"-16" Wide	ALPT Straight Slide Tube (11-7/8")	4-1/4" Footplate	1062576-NLA	1062577-NLA
17"-18" Wide	ALPT Straight Slide Tube (11-7/8")	5-1/4" Footplate	1060440-NLA	1060441
19"-20" Wide	ALPT Straight Slide Tube (11-7/8")	6-1/4" Footplate	1062578	1062579