


Adjustable Angle Footplate w/ Pivot Tube - AT5543

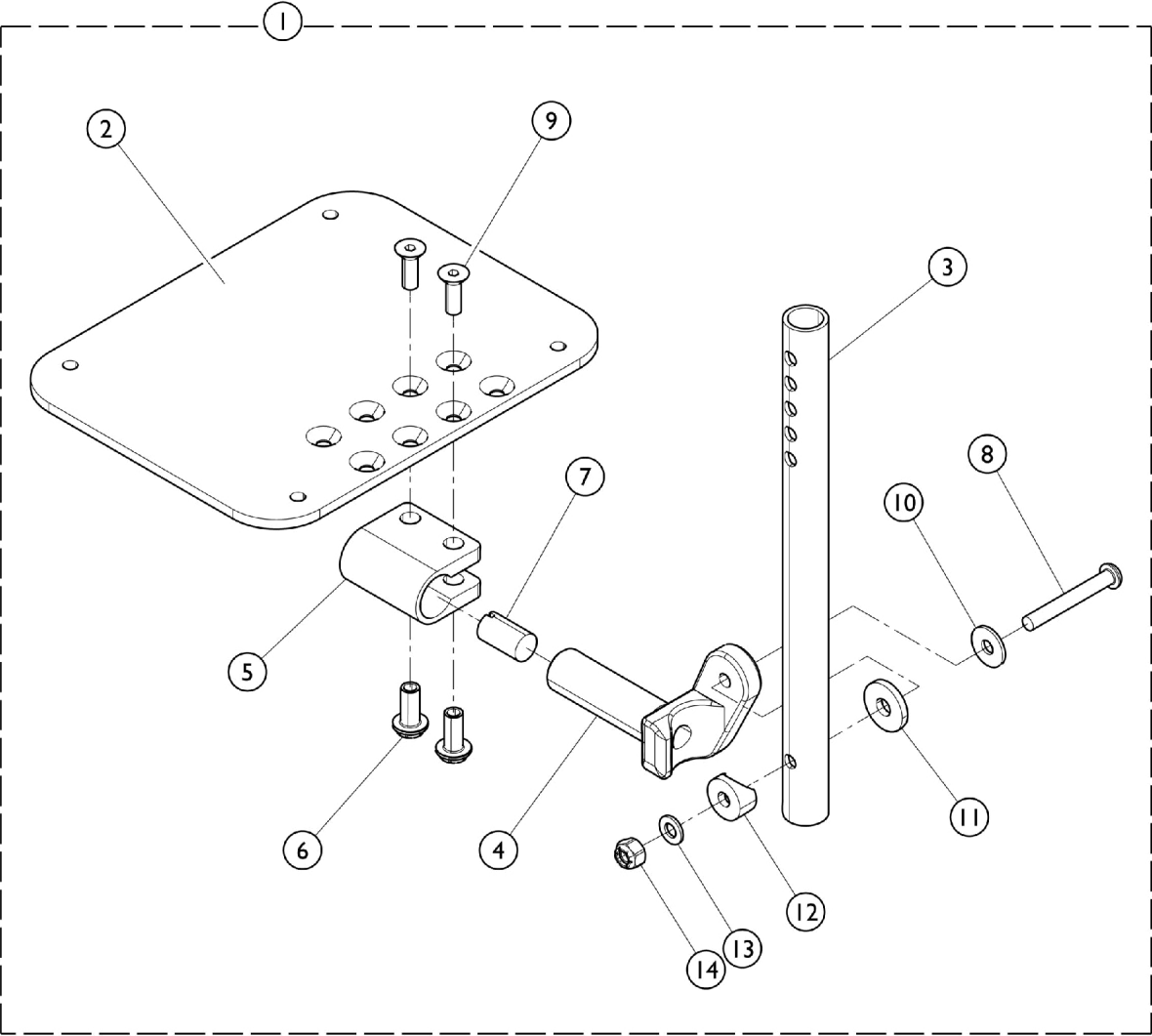

Adjustable Angle Footplate w/ Pivot Tube - AT5543

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	60 Degree, 60 Degree MFX, 70 Degree Non-Tapered, & ELR's Assembly	1	1
1	A	See Chart	70 Degree Assembly	1	1
1	A	See Chart	70 Degree Tapered Assembly	1	1
1	A	See Chart	90 Degree and 70 Degree MFX Assembly	1	1
2	C	1113571	Footplate, Adjustable Angle, Small, Black (3-1/4" W x 8" L)	1	1
2	C	1113572	Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
2	C	1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
2	C	1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
3		1120384-NLA	NLA - Tube, Pivot/Slide, Straight - Alum. (9-7/8" L)	1	1
3		1120385	Tube, Pivot/Slide, Straight - Black (11-7/8" L)	1	1
4		1052102	Hinge, Footplate - Right	1	1
4		1052103	Hinge, Footplate - Left	1	1
5	B	1058517	Clamp, Footplate	1	1
6	B	1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
7		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
8		7930321-NLA	NLA Package, Screw, Socket Button Head (1/4-20 x 1-7/8")	25	1
9	B	1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
10		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
11		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	1
12		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
13		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
14		7930333	Locknut (1/4-20)	25	1

NOTE: A - Includes items 2 - 14.
 B - Due to a hardware change, chairs manufactured before 8/1999, items 5, 6, and 14 must be ordered together.
 C - Footplate was redesigned 9/2/2008 with rounder corners and accessory holes changed from 8 to 4. It is recommended to replace in pairs to match sides.

#1 60 Degree, 60 Degree MFX, 70 Degree Non-Tapered, & ELR's Assembly

Straight Slide Tube (9-7/8")	Right	Left
12-13" Wide (4-1/4" Footplate)	1060430	1060431
14-15" Wide (5-1/4" Footplate)	1060432	1060433
16-20" Wide (6-1/4" Footplate)	1062572	1062573
18-20" Wide (7-1/4" Footplate)	1068620	1068621

#1 70 Degree Assembly

Adjustable Angle Footplate w/ Pivot Tube - AT5543

#1 70 Degree Assembly

ALPT Straight Pivot Tube (11-7/8")	Right	Left
12-13" Wide (4-1/4" Footplate)	1062576-NLA	1062577-NLA
14-15" Wide (5-1/4" Footplate)	1060440-NLA	1060441
16-17" Wide (6-1/4" Footplate)	1062578	1062579
18-20" Wide (7-1/4" Footplate)	1068622	1068623

Adjustable Angle Footplate w/ Pivot Tube - AT5543

#1 70 Degree Tapered Assembly

Straight Pivot Tube (9-7/8")	Right	Left
16" Wide (4-1/4" Footplate)	1060430	1060431
17-18" Wide (5-1/4" Footplate)	1060432	1060433
19-20" Wide (6-1/4" Footplate)	1062572	1062573
ALPT Straight Pivot Tube (11-7/8")	Right	Left
16" Wide (4-1/4" Footplate)	1062576-NLA	1062577-NLA
17-18" Wide (5-1/4" Footplate)	1060440-NLA	1060441
19-20" Wide (6-1/4" Footplate)	1062578	1062579

#1 90 Degree and 70 Degree MFX Assembly

No Pivot Tube Bolts Through Support	Right	Left
12-13" Wide (4-1/4" Footplate)	1060436-NLA	1060437
14-15" Wide (5-1/4" Footplate)	1060438	1060439
16-20" Wide (6-1/4" Footplate)	1062570	1062571
18-20" Wide (7-1/4" Footplate)	1068618	1068619