
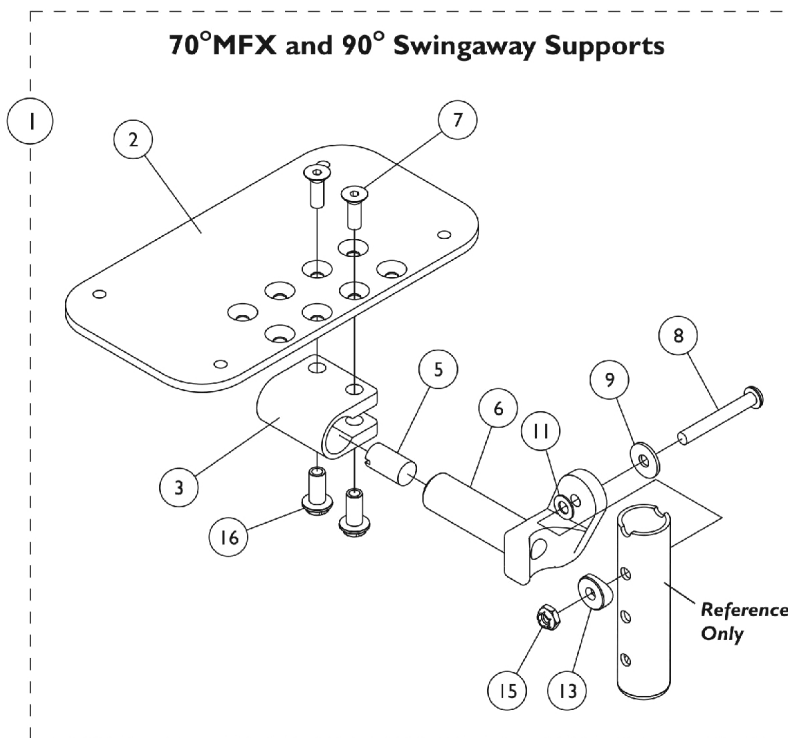


# AT5543 Adjustable Angle Footplates

60°, 60° MFX, 70° NHD, 70° MFX, PW93, PW93E,  
& 90° Swingaway Supports

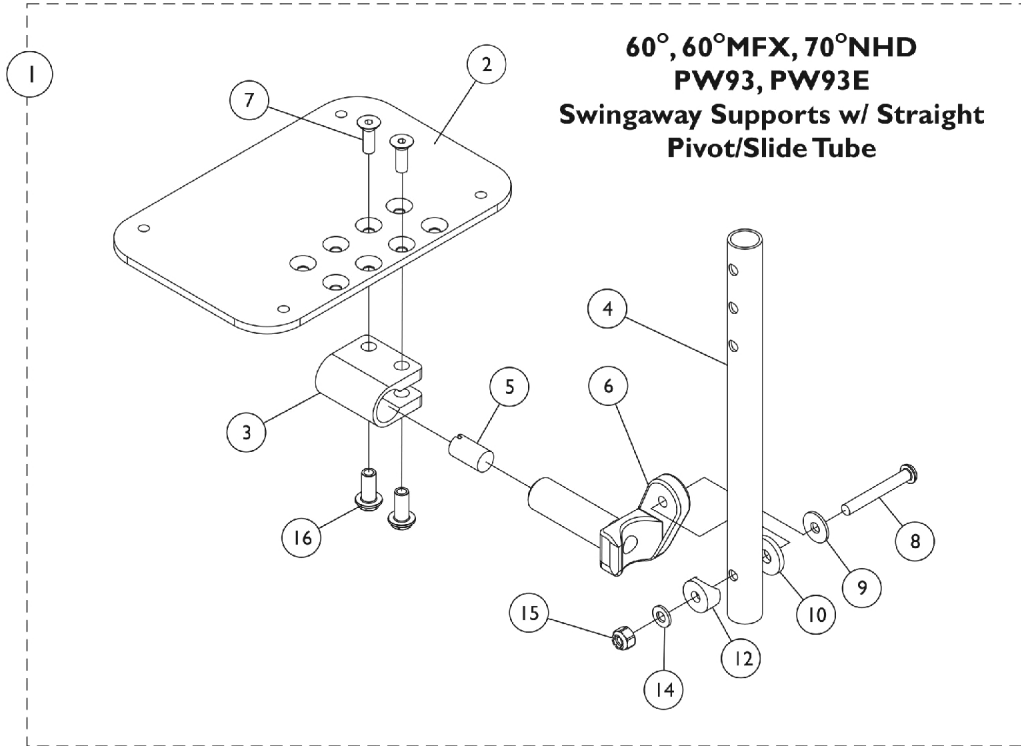

# AT5543 Adjustable Angle Footplates

## 60°, 60° MFX, 70° NHD, 70° MFX, PW93, PW93E, & 90° Swingaway Supports

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	Pivot/ Slide Tube w/ Footplate Assemblies	1	1
2	B,E	1113572	Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
2	C,E	1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
2	D,E	1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
3		1058517	Clamp, Footplate	1	1
4		1057855	Tube, Pivot and Slide, Straight, Steel, Black (9-7/8" L)	1	1
4		1057856	Tube, Pivot/Slide, Straight, Steel, Black (11-7/8" L)	1	1
5		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
6		1052102	Hinge, Footplate - Right	1	1
6		1052103	Hinge, Footplate - Left	1	1
7		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
8		7930321-NLA	NLA Package, Screw, Socket Button Head (1/4-20 x 1-7/8")	1	1
9		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
10		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
11		1036435	Washer, Nylon (1/4 x 7/8 x 1/32")	1	1
12		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
13		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
14		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
15		7930333	Locknut (1/4-20)	1	1
16		1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2

NOTE: A - Includes Items 2 thru 16.  
B - For 12-13" wide chairs.  
C - For 14-15" wide chairs.  
D - For 16" and wider chairs.  
E - This footplate was revised 9/2/2008 with rounder corners and accessory holes changed from 8 to 4. Recommended to replace footplate in pairs to match sides.

### #1 Pivot/ Slide Tube w/ Footplate Assemblies

Rigging Type	90 and 70 MFX w/o Pivot/Slide Tube	60 HD, 60 MFX, 70 NHD PW93, PW93E (9-7/8" Tube)	60 HD, 60 MFX, 70 NHD PW93, PW93E (11-7/8" Tube)
12-13" W - Right (4-1/4" Footplate)	1060436-NLA	N/A	N/A
12-13" W - Left (4-1/4" Footplate)	1060437	N/A	N/A
14-15" W - Right (5-1/4" Footplate)	1060438	1060444	1062758
14-15" W - Left (5-1/4" Footplate)	1060439	1060445	1062759

# AT5543 Adjustable Angle Footplates

## 60°, 60° MFX, 70° NHD, 70° MFX, PW93, PW93E, & 90° Swingaway Supports

### #1 Pivot/ Slide Tube w/ Footplate Assemblies

Rigging Type	90 and 70 MFX w/o Pivot/Slide Tube	60 HD, 60 MFX, 70 NHD PW93, PW93E (9-7/8" Tube)	60 HD, 60 MFX, 70 NHD PW93, PW93E (11-7/8" Tube)
16" and Wider - Right (6-1/4" Footplate)	1062570	1062756	1062760
16" and Wider - Left (6-1/4" Footplate)	1062571	1062757	1062761