


# AT5543 Adjustable Angle Footplate Assemblies

## 70 Degree, PHW93, AT5544, PHAL4A Swingaway Supports

NOTE: Heel loops are not included. See Footrest and Legrest Accessories page.

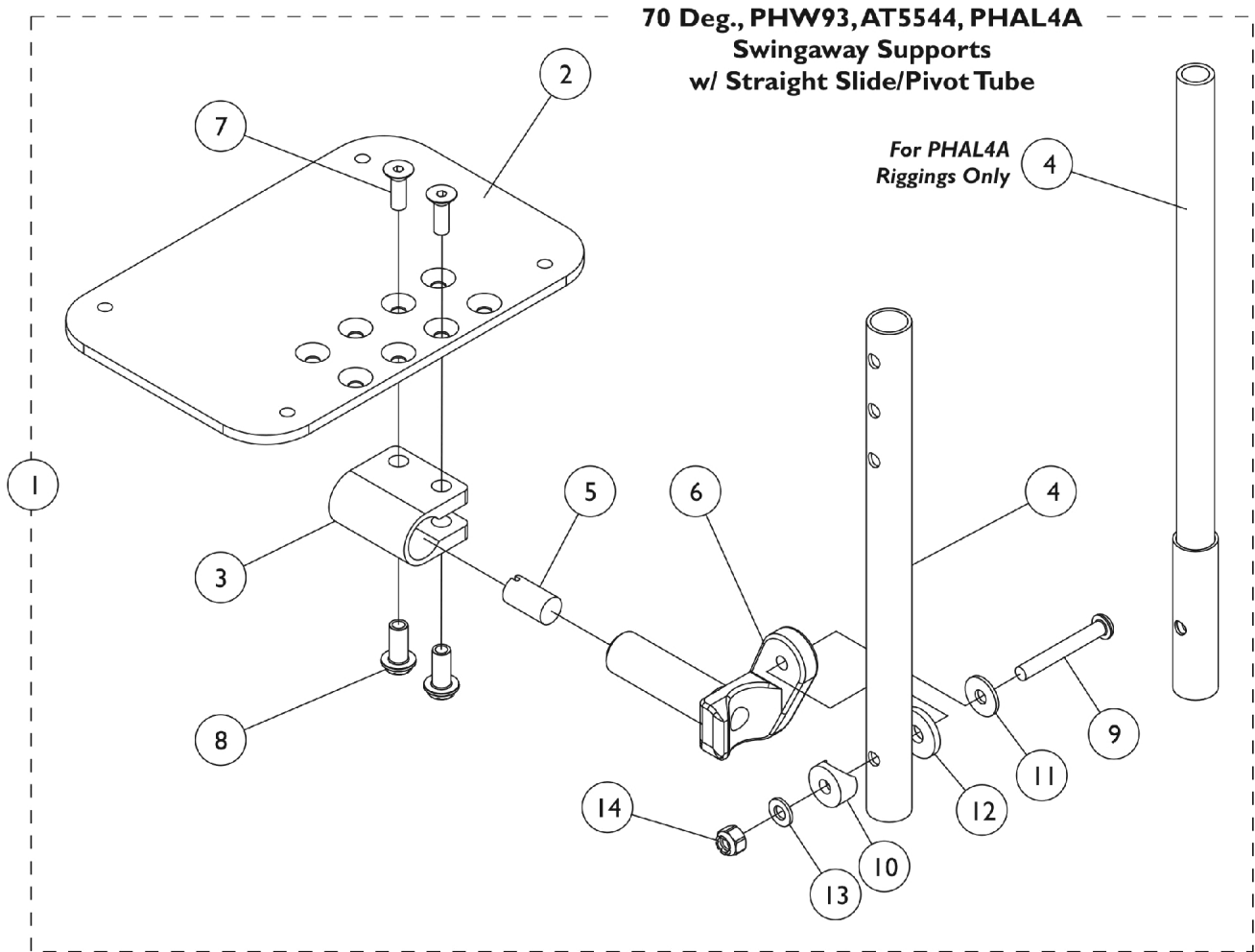

# AT5543 Adjustable Angle Footplate Assemblies

## 70 Degree, PHW93, AT5544, PHAL4A Swingaway Supports

**NOTE: Heel loops are not included. See Footrest and Legrest Accessories page.**

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	See Chart	Pivot/Slide Tubes with Footplates	1	1
2	C	1113572	Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
2	C	1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
2	C	1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
3		1058517	Clamp, Footplate	1	1
4		1057855	Tube, Pivot and Slide, Straight, Steel, Black (9-7/8" L)	1	1
4		1057856	Tube, Pivot/Slide, Straight, Steel, Black (11-7/8" L)	1	1
4		1121135	Tube, Pivot/Slide, Straight, Steel, Black (13-7/8" L)	1	1
4	A	1091745-NLA	NLA - Tube, Pivot/Slide, Straight, Steel, Chrome (13-7/8" L)	1	1
5		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
6		1052102	Hinge, Footplate - Right	1	1
6		1052103	Hinge, Footplate - Left	1	1
7		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
8		1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
9		7930321-NLA	NLA Package, Screw, Socket Button Head (1/4-20 x 1-7/8")	25	1
10		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
11		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
12		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	1
13		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
14		7930333	Locknut (1/4-20)	25	1

NOTE: A - For PHAL4A only.  
 B - Includes Items 2 thru 14.  
 C - The footplate design was revised on 9/2/2008 with rounder corners and accessory holes changed from eight to four. Recommended to replace footplate in pairs to match sides.

## 70 Degree, PHW93, AT5544, PHAL4A Swingaway Supports

### Adjustable Angle Footplates w/o Pivot Slide Tubes

Rigging Type	16"-22" Wide (6-1/4" X-Large Footplate)
70, PHW93, AT5544 -Right	1062756
70, PHW93, AT5544 -Left	1062757
70, PHW93, AT5544, 11-7/8" Tube - Right	1062760
70, PHW93, AT5544, 11-7/8" Tube - Left	1062761
70, PHW93, AT5544, 13-7/8" Tube - Right	1121140
70, PHW93, AT5544, 13-7/8" Tube - Left	1121141
PHAL4A - Right	1091777-NLA

# **AT5543 Adjustable Angle Footplate Assemblies**

## **70 Degree, PHW93, AT5544, PHAL4A Swingaway Supports**

**NOTE: Heel loops are not included. See Footrest and Legrest Accessories page.**

### **70 Degree, PHW93, AT5544, PHAL4A Swingaway Supports**

#### **Adjustable Angle Footplates w/o Pivot Slide Tubes**

Rigging Type	16"-22" Wide (6-1/4" X-Large Footplate)
PHAL4A - Left	1091778-NLA