
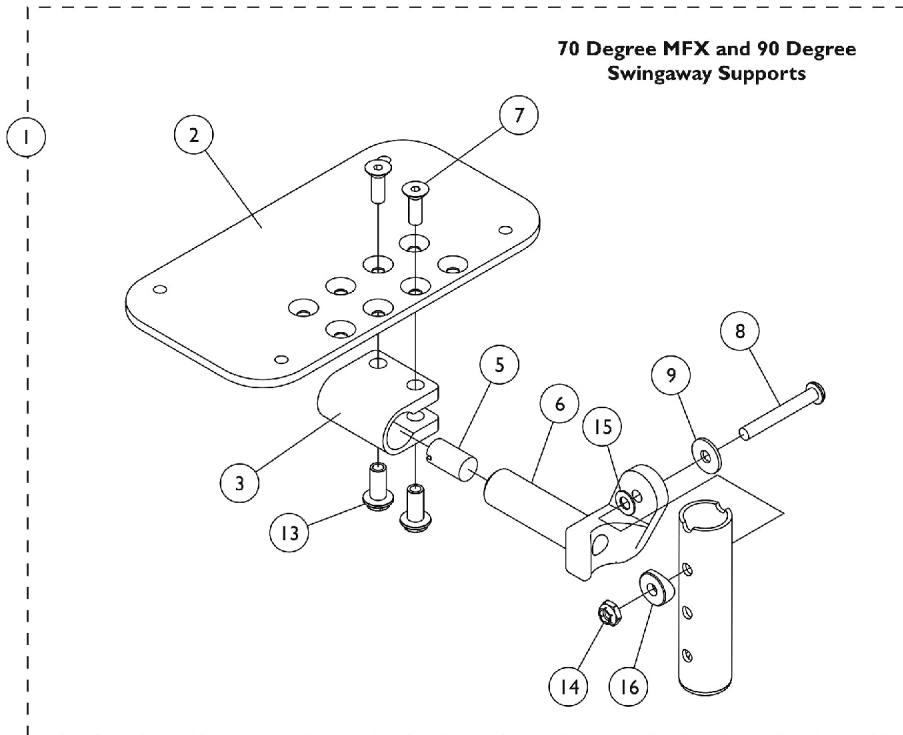


Adjustable Angle Footplates (AT5543)

60 Degree HD, 60 Degree MFX, 70 Degree MFX, PW93, PW93E and 90 Degree Swingaway Supports

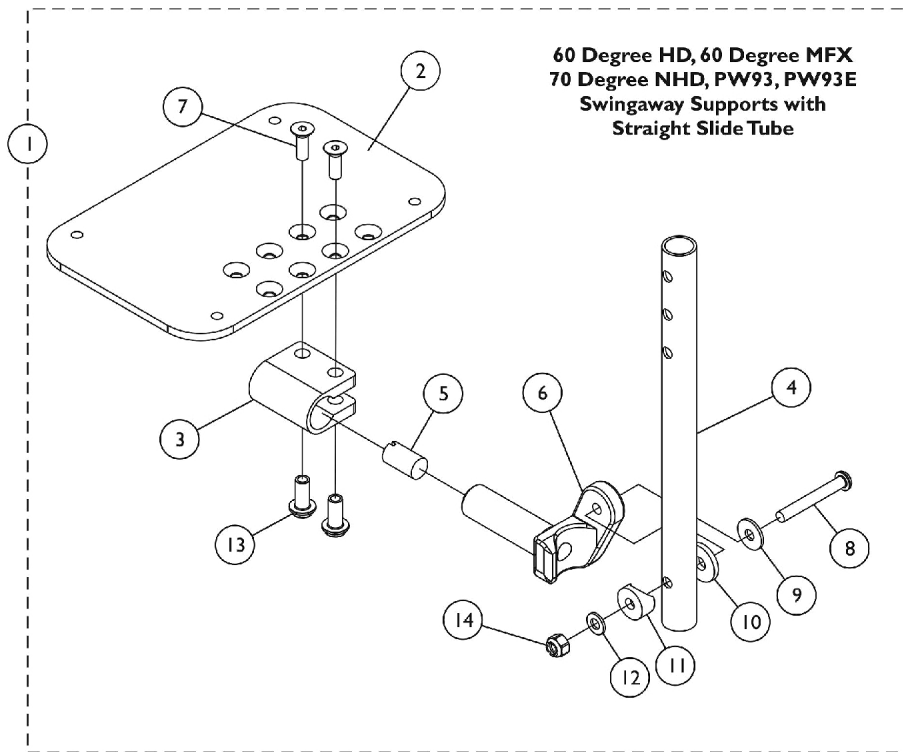

Adjustable Angle Footplates (AT5543)

60 Degree HD, 60 Degree MFX, 70 Degree MFX, PW93, PW93E and 90 Degree Swingaway Supports

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	See Chart	Pivot/Slide Tubes w/ Footplates	1	1
2	B,D	1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
2	C,D	1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
3	A	1058517	Clamp, Footplate	1	1
4		1057855	Tube, Pivot and Slide, Straight, Steel, Black (9-7/8" L)	1	1
4		1057856	Tube, Pivot/Slide, Straight, Steel, Black (11-7/8" L)	1	1
5		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
6		1052103	Hinge, Footplate - Left	1	1
6		1052102	Hinge, Footplate - Right	1	1
7	A	1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
8		7930321-NLA	NLA Package, Screw, Socket Button Head (1/4-20 x 1-7/8")	25	1
9		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
10		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	1
11		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
12		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
13	A	1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
14		7930333	Locknut (1/4-20)	25	1
15		1036435	Washer, Nylon (1/4 x 7/8 x 1/32")	1	1
16		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1

NOTE: A - Items 3, 7 and 13 must be ordered together for chairs manufactured before 8/10/99 due to a hardware change.
 B - For 14-15" wide chairs.
 C - For 16" and wider chairs.
 D - This footplate was revised 9/2/2008 with rounder corners and accessory holes changed from 8 to 4. Recommended to replace footplate in pairs to match sides.
 E - Includes Items 2 thru 16.

#1 Pivot/Slide Tube w/ Footplate Assemblies

Rigging	90 and 70 MFX	60HD, 60MFX, 70NHD, PW93, PW93E (9-7/8" Slide Tube)	60HD, 60MFX, 70NHD, PW93, PW93E (11-7/8" Slide Tube)
5-1/4" Footplate, Right	1060438	1060444	1062758
5-1/4" Footplate, Left	1060439	1060445	1062759
6-1/4" Footplate, Right	1062570	1062756	1062760
6-1/4" Footplate, Left	1062571	1062757	1062761