


Pivot Slide Tube with Footplate

For 93E, 93M, 93M2, H93E, H93M Assemblies

(14" to 15" Chairs)

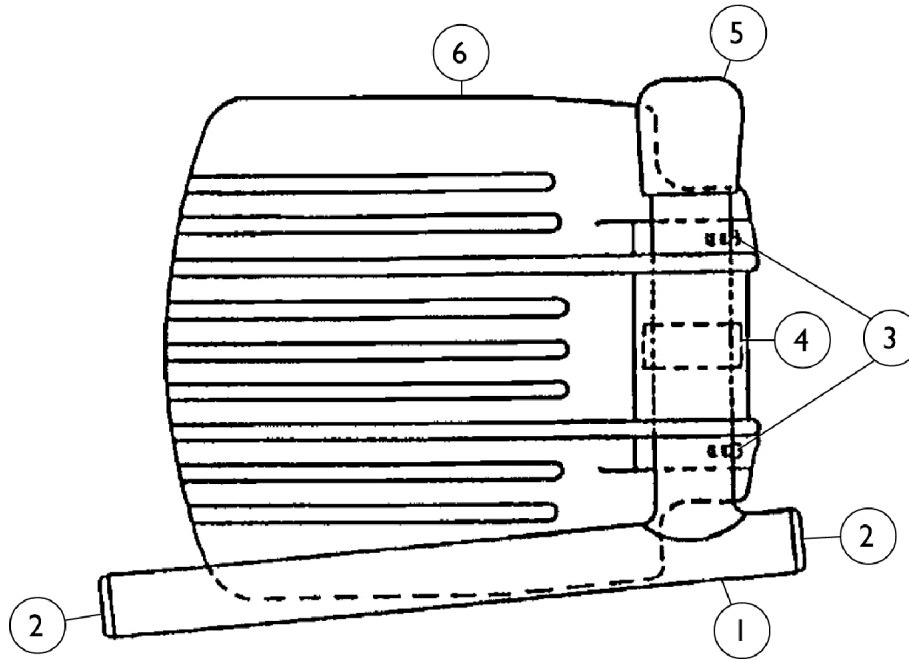

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7500004-NLA	NLA - Pivot/Slide Tube w/ Plug Button	1	1
2		1027097	Plug Button, Black (3/4")	1	2
3		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	2
4		1006981	Spring, Tension	1	1
5		1016346	Cap, End, Tube, Black (7/8")	1	1
6	A,C	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
	A,B	1033506-NLA	NLA - Pivot/Slide Tube w/ Aluminum Footplate Assembly, Chrome - Right	1	1
	A,B	1033507-NLA	NLA - Pivot/Slide Tube w/ Aluminum Footplate Assembly, Chrome - Left	1	1
NOTE: A - Aluminum footplates on chairs built before 2/96 had a "Bright Aluminum" finish. Footplates after 1/96 now have a black finish. For Front Riggings before 2/96, order in pairs to match. B - Not Shown C - Right or left					