


Pivot Slide Tube with Footplate

For 93, H93 Assemblies

(14" to 15" Chairs)

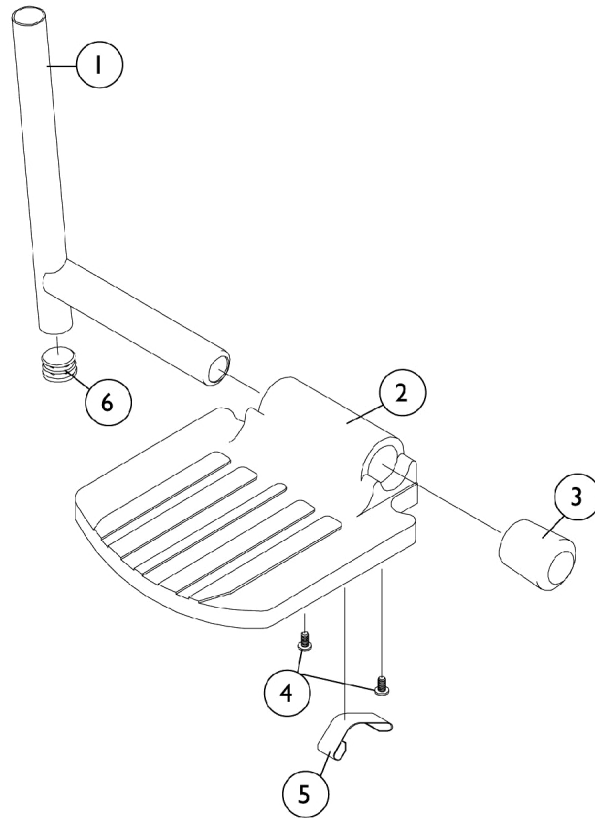

Pivot Slide Tube with Footplate For 93, H93 Assemblies (14" to 15" Chairs)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	A	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
3		1016346	Cap, End, Tube, Black (7/8")	1	1
4		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
5		1006981	Spring, Tension	1	1
6		1027097	Plug Button, Black (3/4")	1	1
	A,B	1058573-NLA	NLA - Pivot/Slide Tube w/ Footplate Assembly Aluminum, Right	1	1
	A,B	1058574-NLA	NLA - Pivot/Slide Tube w/ Footplate Assembly Aluminum, Left	1	1
NOTE: A - Aluminum footplates on chairs before 2/96 had a "Bright Aluminum" finish. Footplates after 1/96 now have a black finish. For Front Riggings before 2/96, order in pairs to match. B - Not Shown					