


Pivot Slide Tube with Footplate For 93ST Assemblies (15" to 17" Chairs)

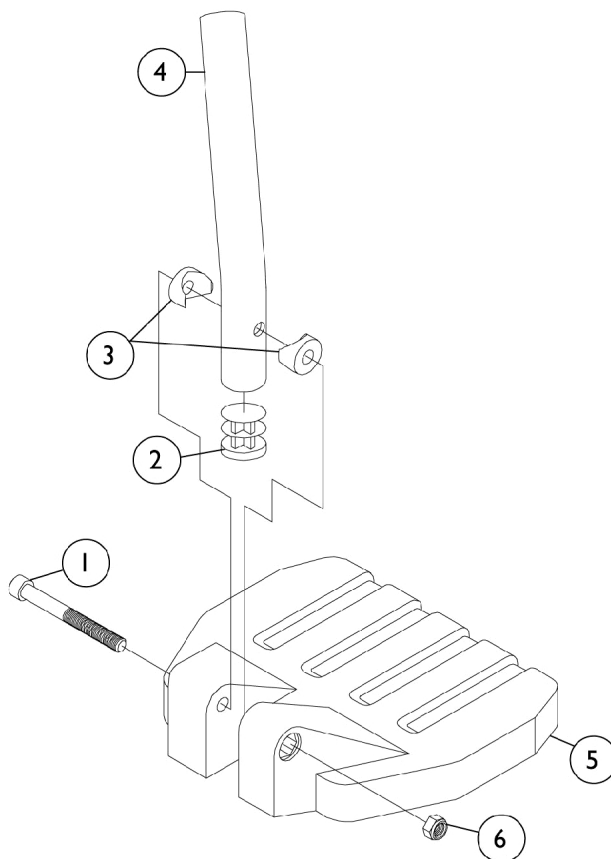

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930366	Package, Socket Head Screw (1/4-20 x 2-1/4")	25	1
2		1027097	Plug Button, Black (3/4")	1	1
3		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	2
4		1120382	Tube, Pivot/Slide, Curved End, Black - Long (9-7/8")	1	1
5		1026879	Footplate, Composite, Small, Black (6" W x 6" L)	1	1
6		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
	A	1028391	Pivot/Slide Tube w/ Small Composite Footplate Assembly, Black - Right	1	1
	A	1028392	Pivot/Slide Tube w/ Small Composite Footplate Assembly, Black - Left	1	1

NOTE: A - Not Shown