


Model 1900 - K Style Base Assembly

Includes K Style Base, Casters and Lower Shroud

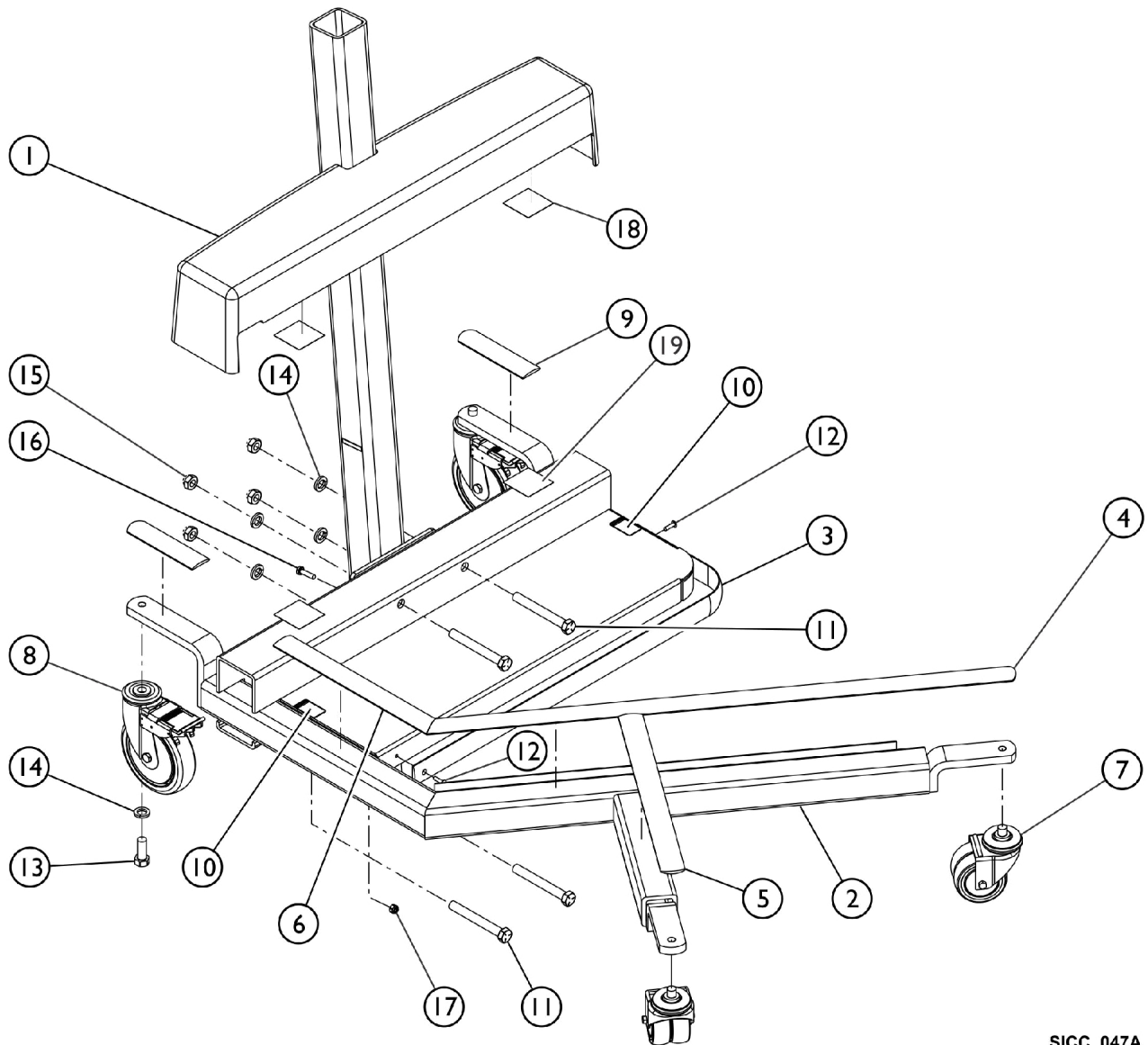

SICC_047A

Model 1900 - K Style Base Assembly

Includes K Style Base, Casters and Lower Shroud

Parts List

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1147301	Shroud, Lower - Base	1	1
2		1147273	Frame Assembly, K Style - Lift	1	1
3		1147295-NLA	NLA - Trim, Strip, Pan	1	1
4		1147286-NLA	NLA - Trim, Strip (33")	1	1
5		1147287-NLA	NLA - Trim, Strip (15")	1	1
6		1147285-NLA	NLA - Trim, Strip (12")	1	1
7		1123560	Caster, Swivel Dual (3")	1	2
8		1123539	Caster Assembly, Swivel w/ Brake (5")	1	2
9		1147288-NLA	NLA - Trim, Strip (5")	1	2
10		1079203	Decal, Warning Pinch Point	1	2
11		1147361-NLA	NLA - Screw, Hex Head (1/2-13 x 4")	1	4
12		IHTRP1614-NLA	NLA - Rivet, Pop, Aluminum	1	2
13		1073884	Screw, Hex Head (1/2-13 x 1-1/4")	1	2
14		1014371	Lockwasher (1/2")	1	6
15		1095483-NLA	NLA - Locknut (1/2-13)	1	4
16		PAR9954-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	4
17		IHTRP1807-NLA	NLA - Locknut (1/4-20)	1	1
18	A	1033800-NLA	NLA - Hook, 2" Adhesive Back	1	1
19	A	1026082-NLA	NLA - Fastener, Dual Lock	1	1

NOTE: A - Cut item from roll into 2" by 1" pieces (2 each).