


Model 1900 - Patient Seat Assembly

Includes Patient Seat, Arms and Shroud

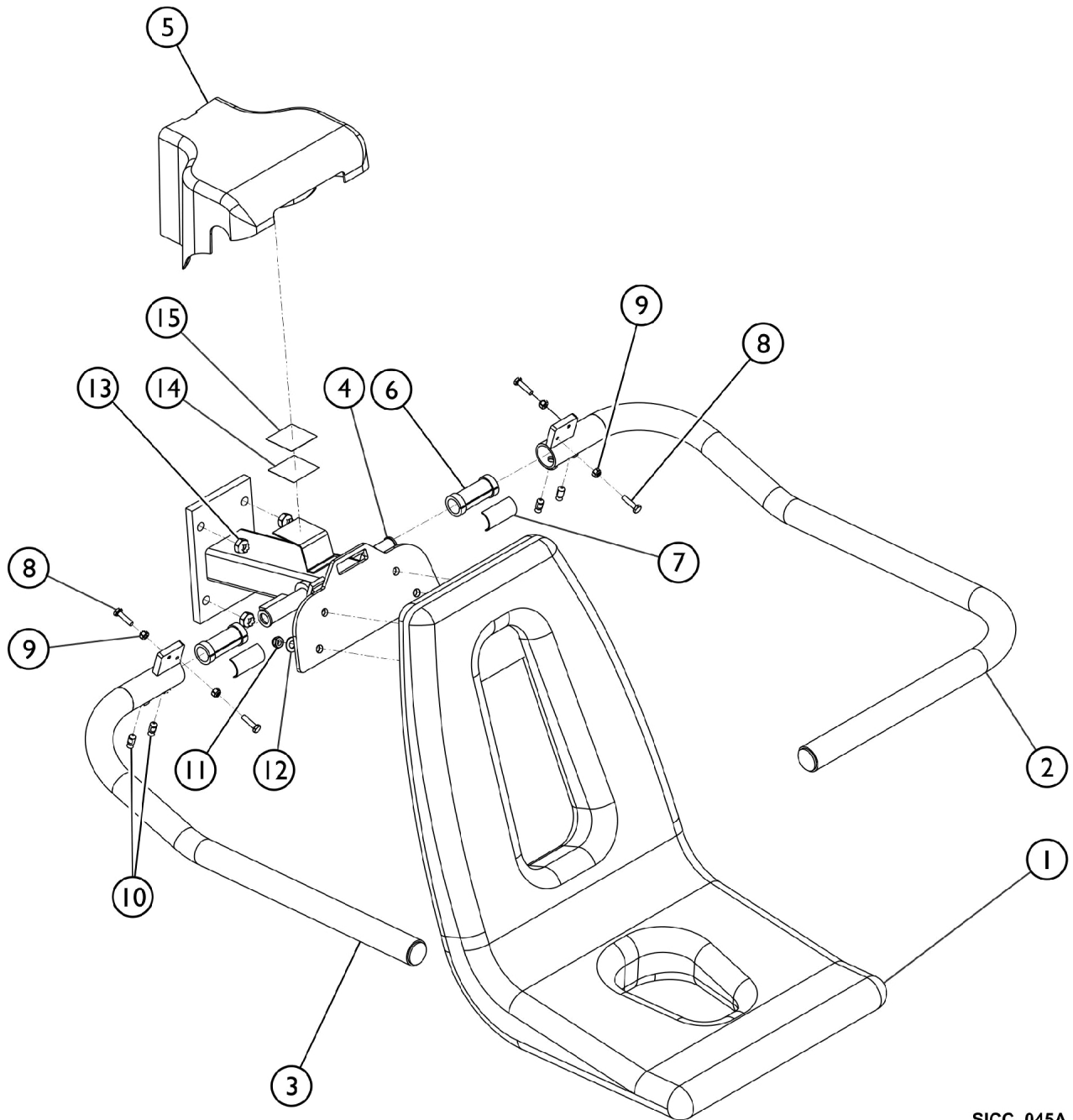

SICC_045A

Model 1900 - Patient Seat Assembly

Includes Patient Seat, Arms and Shroud

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1147282	Seat, Patient, Lift	1	1
2		1147278	Arm, Seat, Lift - Right	1	1
3		1147277	Arm, Seat, Lift - Left	1	1
4		1147283-NLA	NLA - Support, Seat, Lift w/o Load Cell	1	1
5		1147304	Shroud, Seat, Arm	1	1
6		1147279-NLA	NLA - Bearing, Arm, Seat	1	2
7		1147372-NLA	NLA - Shoe, Brake, Bearing	1	2
8		PAR9954-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	4
9		IHTRP1807-NLA	NLA - Locknut (1/4-20)	1	4
10		1142191	Screw, Set, Socket, Cup Point (3/8-16 x 5/8")	1	4
11		40363X000	Hex Nut (3/8"-16)	1	4
12		1153346-NLA	NLA - Washer, Flat (13/32 x 13/16 x 1/16")	1	4
13		1095483-NLA	NLA - Locknut (1/2-13)	1	4
14	A	1033800-NLA	NLA - Hook, 2" Adhesive Back	1	1
15	A	1026082-NLA	NLA - Fastener, Dual Lock	1	1
	B	1147313	Belt, Lap, Seat	1	1
	B	1151805	Belt, Chest, Seat	1	1
NOTE: A - Cut item from roll into 2" by 2" square pieces (1 each). B - Item not shown in illustration.					