


# Electronics - Arro BA3180

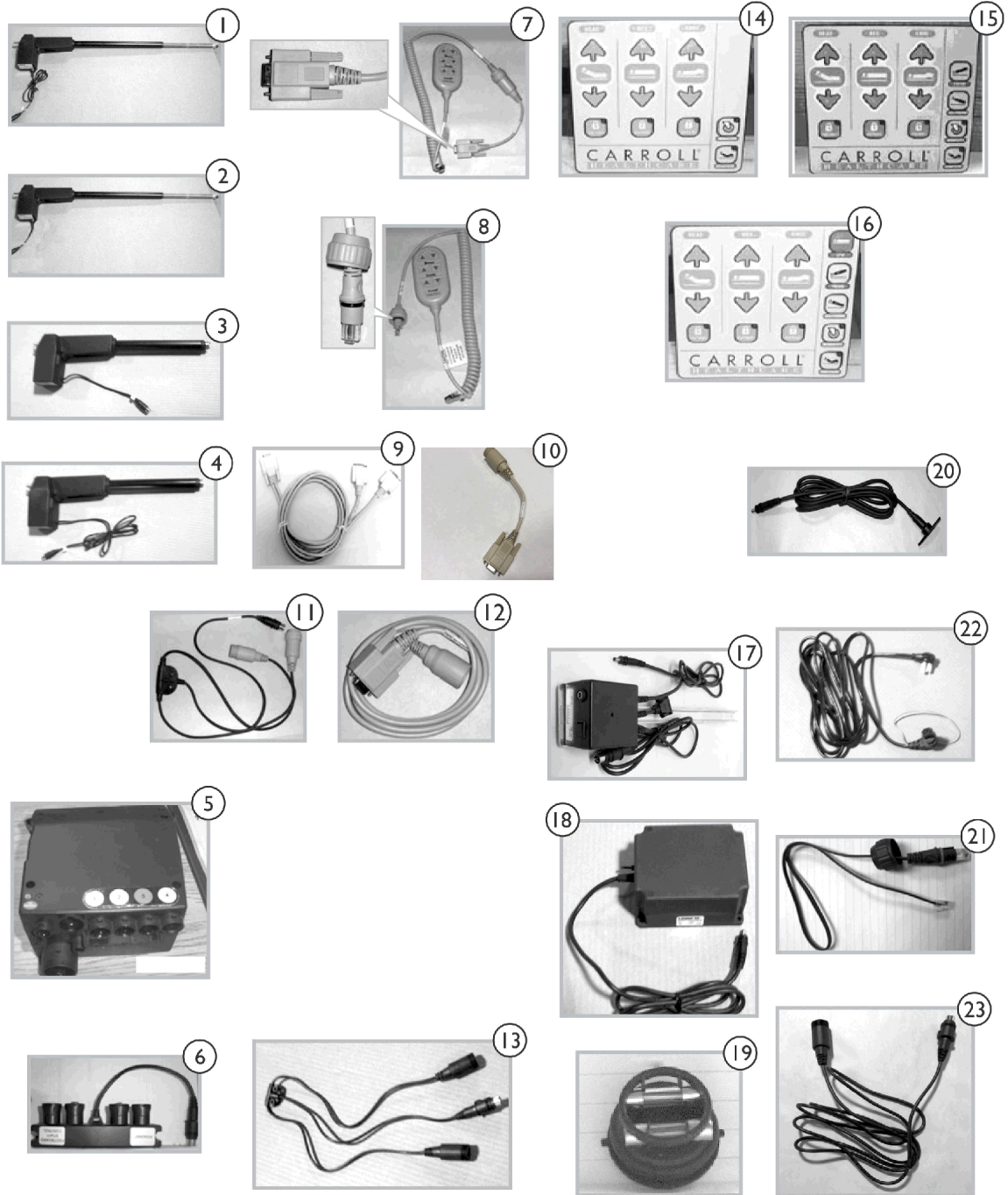

## Electronics - Arro BA3180

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	MTRM25031	Actuator, Head (Red) Linak # LA31-U086	1	1
1	B	MTRM25032	Actuator, Head (Red) Linak # LA31-U336	1	1
2		MTRM25021	Actuator, Foot (Blue) Linak # LA31-U087	1	1
3		MTRM25473-NL A	NLA - Actuator, Hilo, Foot, Short Cord (White) Linak # LA31-U191	1	1
4		MTRM25474	Actuator, Hilo, Head, Long Cord (Yellow) Linak # LA31-U192	1	1
5	C	JNCTNDC1D	Box, Control (111) Linak # CB09-U044-XX	1	1
5	C	JNCTNDC1E-NL A	NLA - Box, Control (151/171) Linak # CB09-U045-XX	1	1
6		JNCTNDC35	Box, DJB (151/171) Linak # DJBX-001-XX	1	1
7	D	1190227	Pendant, Six Function (w/ Loop Adhesive Back Strip and DB9 Connector) Linak # PNDNT-6FNC-1-2	1	1
8	E	PNDNT6FNCIV3	Pendant, Six Function w/ RJ48 Connector) Linak # PNDNT-6FNC-1-2	1	1
9	F	PNDNTYCABLE	Cable, Y, Pendant	1	1
10		1190225	Cable, Conversion	1	1
11	G	JNCTNDC40	Cable, Y, Pendant	1	1
12	I	PNDNTEXTCABLE	Cable, Pendant	1	1
13	H	JNCTNDC34-NL A	NLA - Cable, T, Pendant, Fine Thread Nut (151/171)	1	1
13	E	JNCTNDC47A-N LA	NLA - Cable, T, Pendant, Coarse Thread Nut (151/171)	1	1
14		JNCTNDC202	Pad, Membrane (151)	1	1
15	I	JNCTNDC20D-N LA	NLA - Pad, Membrane w/o CPR (171 w/o CPR)	1	1
16		JNCTNDC20A2	Pad, Membrane w/ CPR (171)	1	1
17		JNCTNBOXKIT3 -NLA	NLA - Box, Lockout (111)	1	1
18		1152564	Battery, On Board Backup w/ Hardware	1	1
19		JNCTNDCFUSE K2-NLA	NLA - Kit, Fuse, Arro, 2.5 Amps (151/171)	1	1
19		JNCTNDCFUSE K1	Kit, Fuse (1.6 Amps)	1	1
20		JNCTNDC31-NL A	NLA - Cable, Battery	1	1
21		JNCTNDC32	Cord, Membrane (Footboard)	1	1
22		JNCTNDCCORD KI	Cable, Power	1	1
23		JNCTNDC33-NL A	NLA - Cable, Membrane (DJB)	1	1

NOTE: A - On Beds built prior to June 24, 2007.  
 B - On Beds built after June 24, 2007.  
 C - Control Box type determined by the item number on the box. In the item number CB09-Uxxx-(xx), the last two digits are a revision level and have nothing to do with part identification.  
 D - Used on Pendants for Beds built prior to November 22, 2007. Also used DB9 Connector.  
 E - Used on Beds built after November 22, 2007. Use RJ48 Connector.  
 F - Used on Beds built prior to November 22, 2007.  
 G - Beds built after November 22, 2007 require this part.  
 H - Order JNCTNDC47A (Coarse Thread) to match Pendant Thread.  
 I - Order JNCTNDC20AZ (w/CPR)