


Pivot Slide Tube with Footplate For AS4A, AHS4A Assemblies (14" to 15" Chairs)

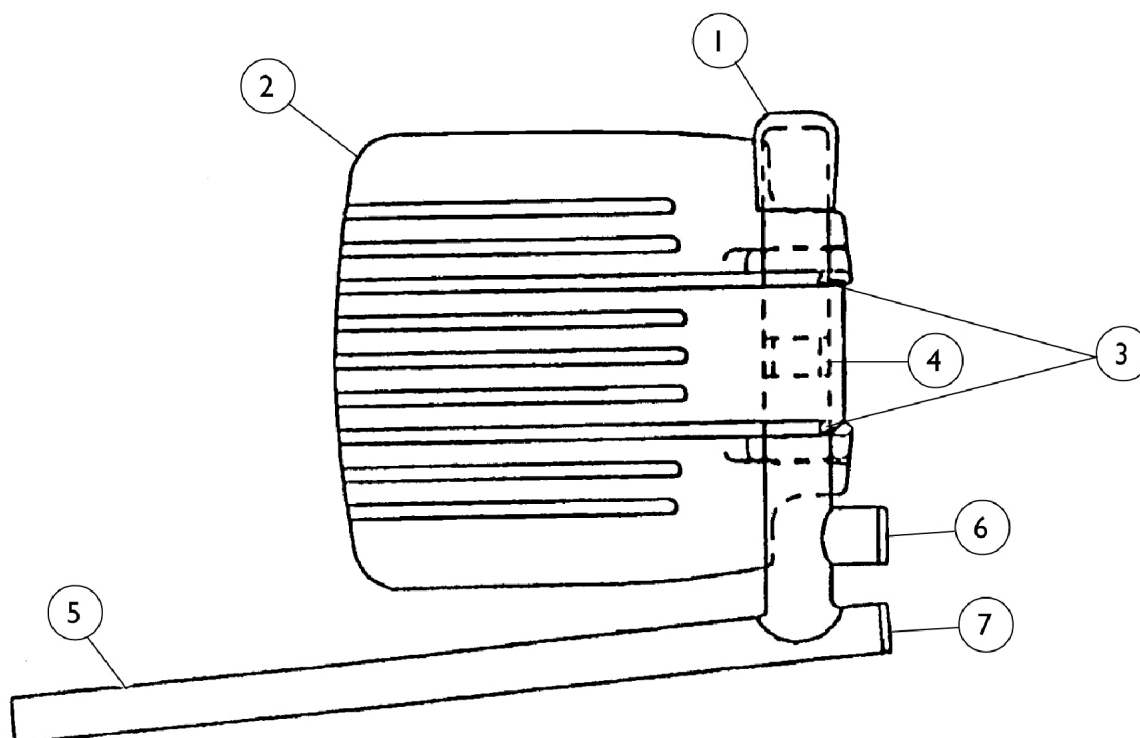

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1016346	Cap, End, Tube, Black (7/8")	1	1
2	A,C	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
3		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	1	1
4		1006981	Spring, Tension	25	2
5		1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
6		1027097	Plug Button, Black (3/4")	1	1
7		1018133	Plug Button, Black (5/8")	1	1
	B	1023905-NLA	NLA - Pivot/Slide Tube w/ Aluminum Footplate Assembly, Chrome - Right	1	1
	B	1023906-NLA	NLA - Pivot/Slide Tube w/ Aluminum Footplate Assembly, Chrome - Left	1	1

NOTE: A - Aluminum footplates on chairs built before 2/96 had a "Bright Aluminum" finish. Footplates after 1/96 now have a black finish. For Front Riggings before 2/96, order in pairs to match.
B - Not Shown
C - Right or Left