

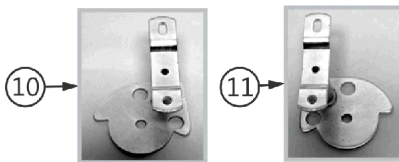
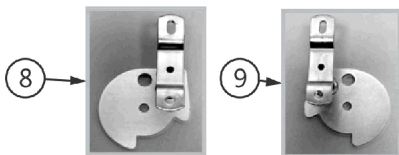
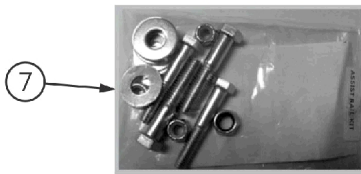
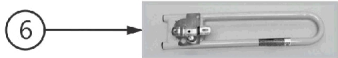
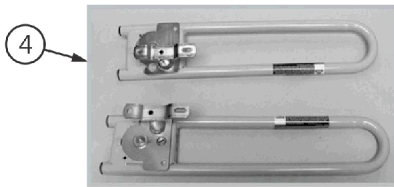
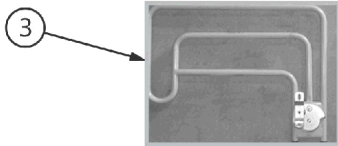
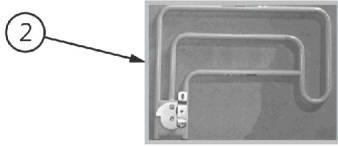
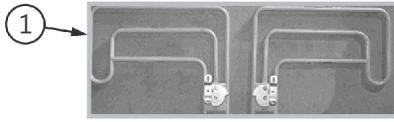


Side Rail Parts - BA1180, BA2180, BA3180


Assist Rails

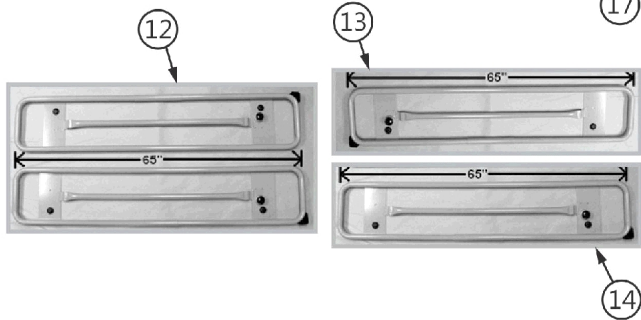


Side Rails - Non Tuck Including Hardware

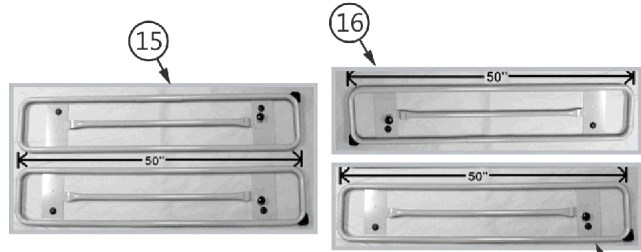

Rail Arms

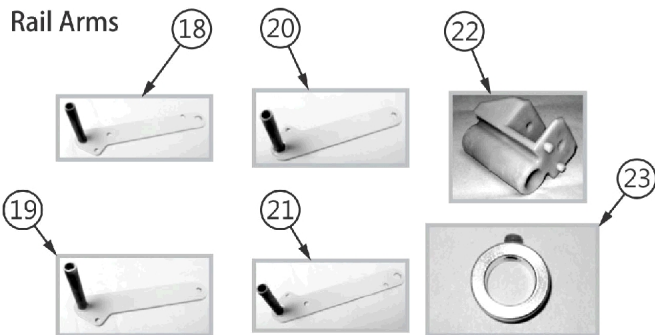

Pull Knob Assembly for All Rails

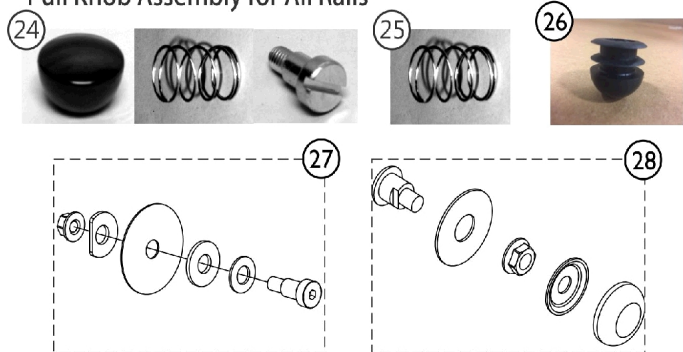

Side Rail Parts - BA1180, BA2180, BA3180

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		IHECRLAS	Assist Rails (Pair)	1	1
2		RAILSAELT-NLA	NLA - Assist Rail Assembly - Left Side	1	1
3		RAILSAERT-NL A	NLA - Assist Rail Assembly - Right Side	1	1
4		RAILSBARPRU- NLA	NLA - Assist Bars (Pair), "U" Bracket	1	1
4		IHECBAAS	Set of Assist Bars	1	1
5		RAILABLT-NLA	NLA - Assist Bar - Left Side	1	1
6		RAILABRT-NLA	NLA - Assist Bar - Right Side	1	1
7		KITASSISTR1	Kit, Assist Rail Mounting Hardware	1	1
8		RLBRACKAEP- NLA	NLA - Bracket, Mounting, Assist Rail - Left Side	1	1
9		RRBRACKAEP- NLA	NLA - Bracket, Mounting, Assist Rail - Right Side	1	1
10		RLBRACKASTR P-NLA	NLA - Punched Bracket, No Guard - Left Hand	1	1
11		RRBRACKASTR P-NLA	NLA - Punched Bracket, No Guard - Right Hand	1	1
12		IHARRLFUL	NLA - Full Rails (Set)	1	1
13		RAILS0080FLNT -NLA	NLA - Full Rail - Left Hand	1	1
14		RAILS0080FRN T-NLA	NLA - Full Rail - Right Hand	1	1
15		IHARRLPAR	NLA - Partial Rails (Set)	1	1
16		RAILS0080PLNT -NLA	NLA - Partial Rail - Left Hand	1	1
17		RAILS0080PRN T-NLA	NLA - Partial Rail - Right Hand	1	1
18		LRA001RA-NLA	NLA - Long Rail Arm - Right	1	1
19		SRA001RA-NLA	NLA - Short Rail Arm - Right	1	1
20		SRA001LA-NLA	NLA - Short Rail Arm - Left	1	1
21		LRA001LA-NLA	NLA - Long Rail Arm - Left	1	1
22		RAILBRACKNT- NLA	NLA - Rail Bracket	1	1
23		COL34R-NLA	NLA - Locking Set Collar	1	1
24		BLT1212SLOTS H	Bolt, Shoulder (1/2 x 1/2", 3/8-16)	1	1
25		LATCH0002	Knob, Black Ball	1	1
26		WSR3834116N	NLA - Washer, Nylon (3/8 x 3/4 x 1/16")	1	1
27		SPRASSIST-NL A	NLA - Spring, Compression	1	1