
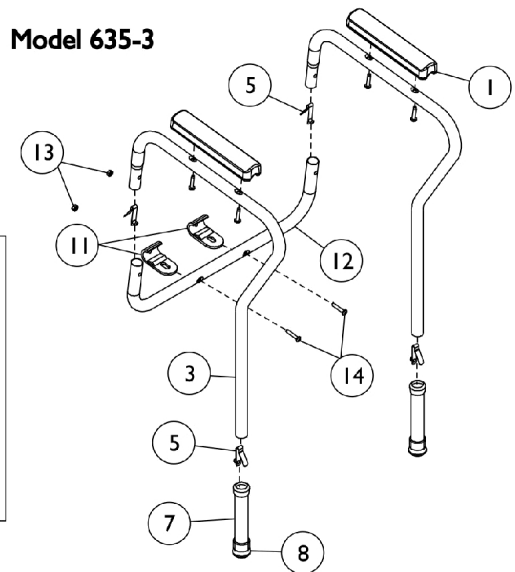


# Toilet Safety Frame - Model 1392 and 635-3

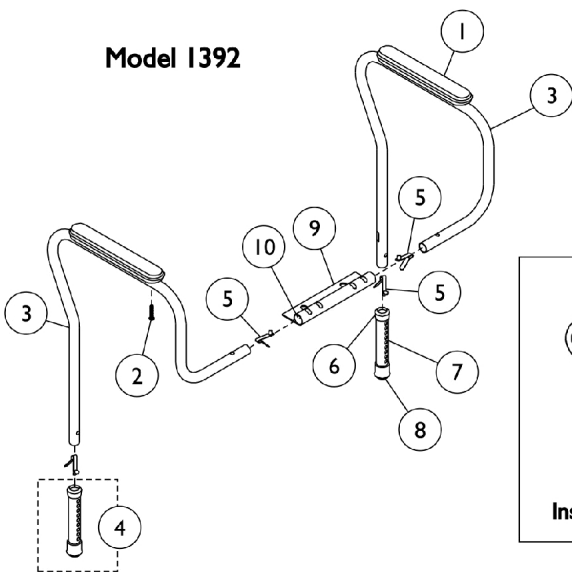

## Toilet Safety Frame - Model 1392 and 635-3

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	01000X001-NLA	NLA - Armrest, Full Length, Beige	1	2
1	A,C	1163889	Armrests (Pair)	1	1
1	D,E	1073833-NLA	NLA - Armrest, Full Length, White	1	2
1	D,F,N	1059660-NLA	NLA - Armrest, Full Length, Grey	1	2
	L,M	1132261-NLA	NLA - Kit, Armrest with Hardware, Full Length - Grey	1	1
1			NLA - Armrest, Full Length, Grey	1	2
2			NLA - Screw, Phillips Pan Head (1/4 x 1-1/4")	1	4
2	A	48127X001-NLA	NLA - Screw, Phillips Round Head (#10-32 x 1/2")	1	4
2	D	1061431-NLA	NLA - Screw, Phillips Pan Head (1/4 x 1-1/4")	1	4
3	A,K	48016X000-NLA	NLA - Frame, Side - Right	1	1
3	A,K	48016X001-NLA	NLA - Frame, Side - Left	1	1
3	D	1082559-NLA	NLA - Frame, Side Rail, Chrome - Right/Left	1	2
4	A,G	1058806-NLA	NLA - Leg Extension Assembly	1	2
4	A,G,N	1058807-NLA	NLA - Kit, Leg Extension Sub-Assembly	1	1
			Anti-Rattle, Grey (1-1/8")	1	2
			Tube, Leg Extension	1	2
			Tip, Rubber, Light Grey (1-1/8")	1	2
5		1094945	Button, Snap	1	4
6		1063124	Anti-Rattle, Grey (1-1/8")	1	2
7	D	1073450-NLA	NLA - Leg Extension Assembly	1	2
8		1057797	Tip, Rubber, Light Grey (1-1/8")	1	2
9	A	1018602-NLA	NLA - Bracket, Aluminum	1	1
10		1057287	Plug Button, Grey (1")	1	2
11		1073237-NLA	NLA - Clamp, Toilet Rail	1	2
12	D	1073452-NLA	NLA - Brace	1	1
	D,J	1074444-NLA	NLA - Kit, Hardware	1	1
13			Locknut (1/4-20)	1	2
14			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	2

NOTE: A - To be used with Model 1392.  
 B - Before 5/17/99.  
 C - After 5/14/09.  
 D - To be used on Model 635-3.  
 E - Before 3/18/99.  
 F - After 3/19/99.  
 G - Kit Includes: Leg Tube, Anti-Rattle and Rubber Tip.  
 J - Kit Includes Items 13 and 14.  
 K - Item no longer available, suggested replacement is a complete unit 1392KD.  
 L - Kit includes (2) Armrests Part #1059660 and (4) Screws Part #1061431.  
 M - Not Shown.  
 N - Item no longer available, refer to Kit Part #1132261.