




Crossbrace Tubes and Hardware

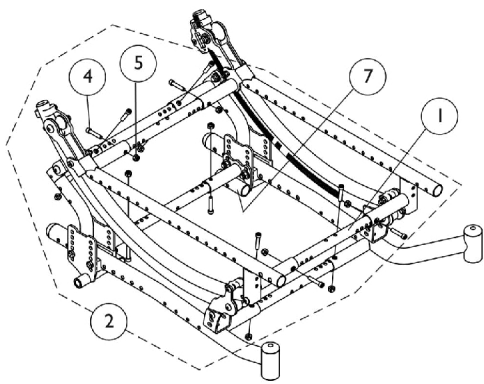


1164100
Crossbrace Tube Mounting Hardware

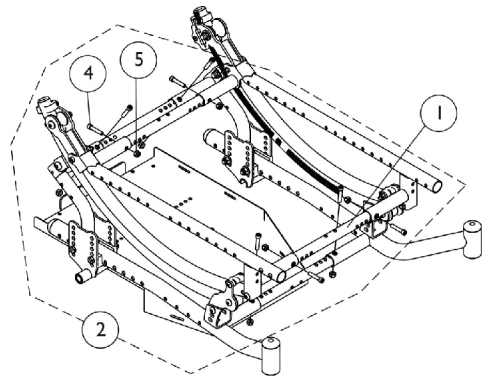


1164100
Crossbrace Tube Mounting Hardware
with Vent Tray Option

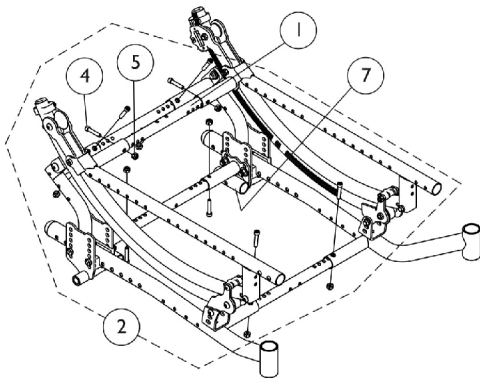

1164100
Crossbrace Tube Mounting Hardware
with Contracture Footrest Option

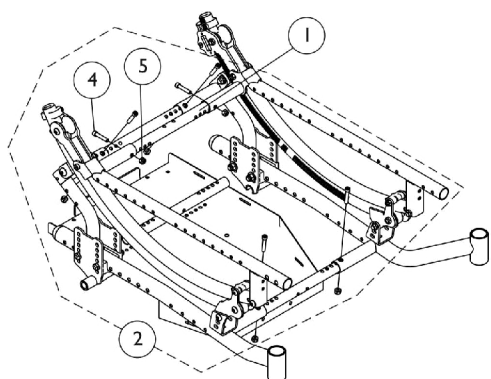

1164100
Crossbrace Tube Mounting Hardware
with Contracture Footrest and
Vent Tray Option

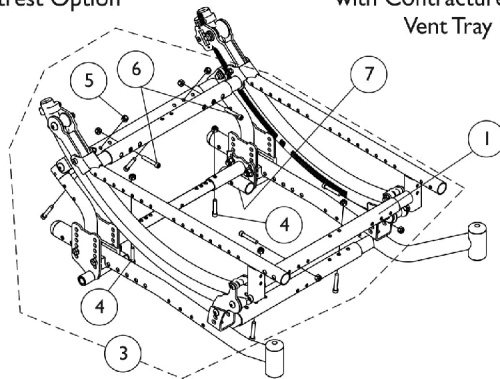

1164101
Crossbrace Tube Mounting Hardware
with Heavy Duty Option

Crossbrace Tubes and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	Crossbrace Tube Kits - Non-Heavy Duty	1	1
1		See Chart	Crossbrace Tube Kits - Heavy Duty	1	1
2	A	1164100	Kit, Crossbrace Tube Mounting Hardware, Non-Heavy Duty	1	1
4			Screw, Socket Head (1/4-28 x 1-1/4")	1	10
5			Locknut (1/4-28)	1	10
3	B	1164101	Kit, Crossbrace Tube Mounting Hardware, Heavy Duty	1	1
4			Screw, Socket Head (1/4-28 x 1-1/4")	1	2
5			Locknut (1/4-28)	1	10
6			Screw, Socket Head (1/4-28 x 1-1/2")	1	8
7		1150930	Bracket, Crossbrace and Anti-Tipper Receiver, Black	1	2

NOTE: A - The hardware was created to be used with all options. Some variations may not require all the hardware. Use for 12" - 22" wide Non-Heavy Duty chairs
 B - The hardware was created to be used with all options. Some variations may not require all the hardware. Use for 14" - 24" wide Heavy Duty and 20" - 24" wide Non-Heavy Duty chairs

#1 Crossbrace Tube Kits - Non-Heavy Duty

Kits were created to be used with all options. Some variations may not require 5 crossbrace tubes

Seat Width Range	Crossbrace Tube Kit	Crossbrace Tube Mounting Hardware Kit
12" - 16" Wide	1164092	1164100
14" - 18" Wide	1164093	1164100
16" - 20" Wide	1164094	1164100
18" - 22" Wide	1164095	1164100
20" - 24" Wide	1164096	1164101

#1 Crossbrace Tube Kits - Heavy Duty

Kits were created to be used with all options. Some variations may not require 5 crossbrace tubes

Seat Width Range	Crossbrace Tube Kit	Crossbrace Tube Mounting Hardware Kit
14" - 18" Wide	1164097	1164101
16" - 20" Wide	1164098	1164101
18" - 22" Wide	1164099	1164101
20" - 24" Wide	1164096	1164101