
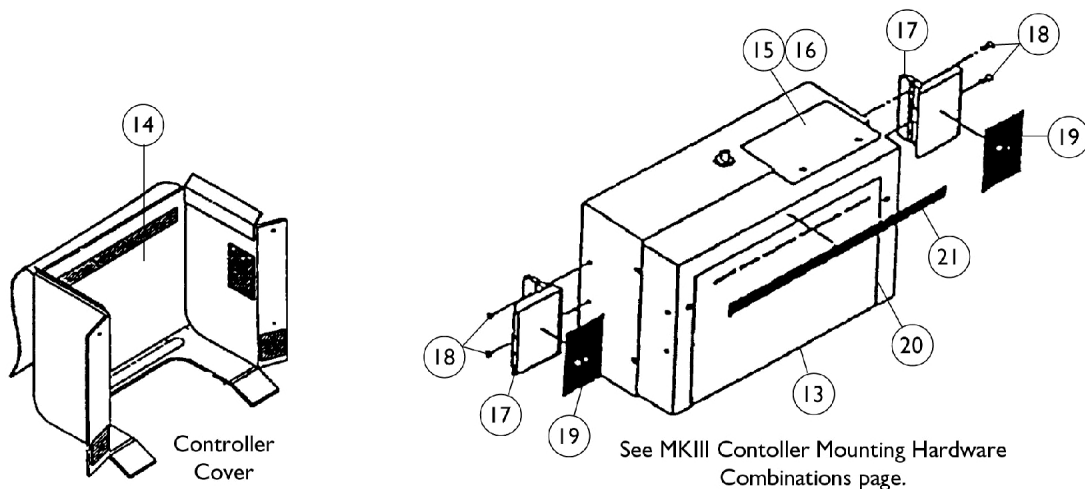


MKIII Controller Mounting Hardware

Before 4/1/96

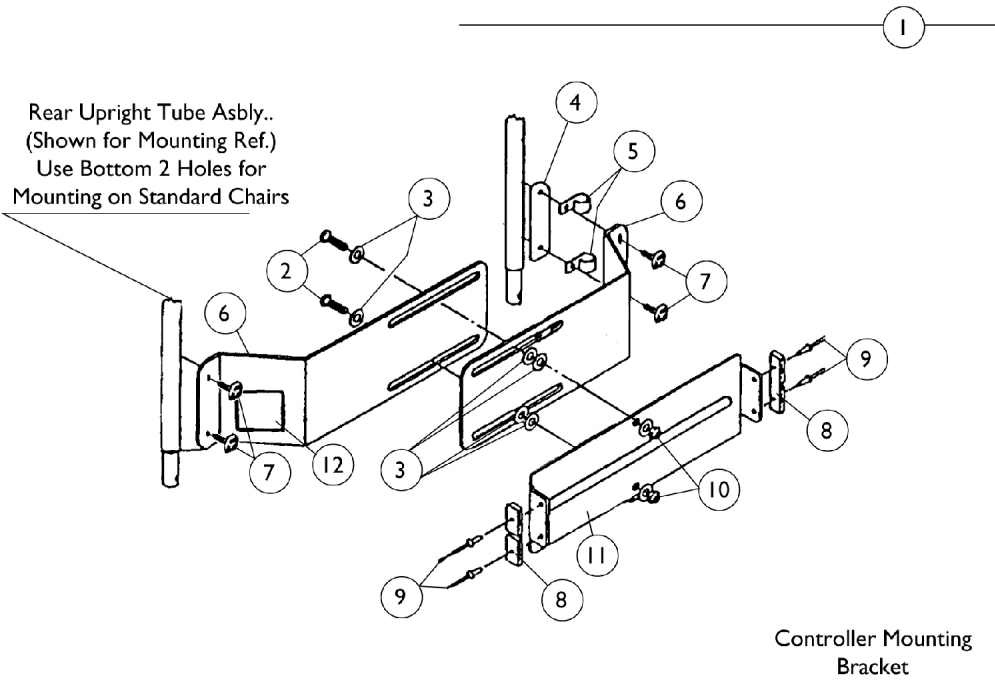

MKIII Controller Mounting Hardware

Before 4/1/96

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,E	1029175-NLA	NLA - Controller Adjustment Bracket Assembly, Black	1	1
1	A,F	1029174-NLA	NLA - Controller Adjustment Bracket Assembly, Black	1	1
2		1029232-NLA	NLA - Screw, Socket Button Head (5/16-24 x 1")	1	2
3		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
4		1029168-NLA	NLA - Plate, Clamp	1	1
5	B	50000X403-NLA	NLA - Clamp, Black Plastic	1	2
6	G	1034031-NLA	NLA - Plate, Control Adjustment (14"-16" Bracket)	1	2
6	F	1034032-NLA	NLA - Plate, Control Adjustment (17"-24" Bracket)	1	2
7		1031095	Screws with Washers (#10-16 x 7/8")	4	4
8		1007406-NLA	NLA - Keepers	1	2
9		1003177-NLA	NLA - Rivet	1	4
10		7930330-NLA	NLA - Locknut (5/16-24)	25	2
11		1028023-NLA	NLA - Bracket, Main Control, Black	1	1
12		1013149-NLA	NLA - Label, Caution	1	1
13	C	See Page	See Footnote	1	1
14		1012814-NLA	NLA - Cover, Controller	1	1
15		1026552-NLA	NLA - Cover, Key Pad	1	1
16		1023102-NLA	NLA - Key Pad	1	1
17		50000X025-NLA	NLA - Clip, Lock Over Latch	1	2
18		1009083-NLA	NLA - Screw, Flat Head (#6-32 x 3/8")	1	4
19		1029228-NLA	NLA - Fastener, Hook Self Adhesive (2" W x 2-15/16" L)	1	2
20		1029231-NLA	NLA - Decal, Micro Hook-Up	1	1
21		1029230-NLA	NLA - Hook Material (9-15/16" x 5/8")	1	1
	D	1046117-NLA	NLA - Printer and Interface	1	1
	D	1026572-NLA	NLA - Paper, Printer	1	1
	D	1027280-NLA	NLA - Harness, Printer Wiring	1	1
NOTE: A - Also used to mount Ranger X controllers B - Clamp color alternated between black and white C - See page titled "MKIII Controller Mounting Hardware Combinations". D - Not Shown E - 14" or 16" Widths F - 17" thru 22" Wide Chairs G - 14" thru 16" Wide					