



# Handrims and Mounting Hardware - Natural Fit and Natural Fit LT

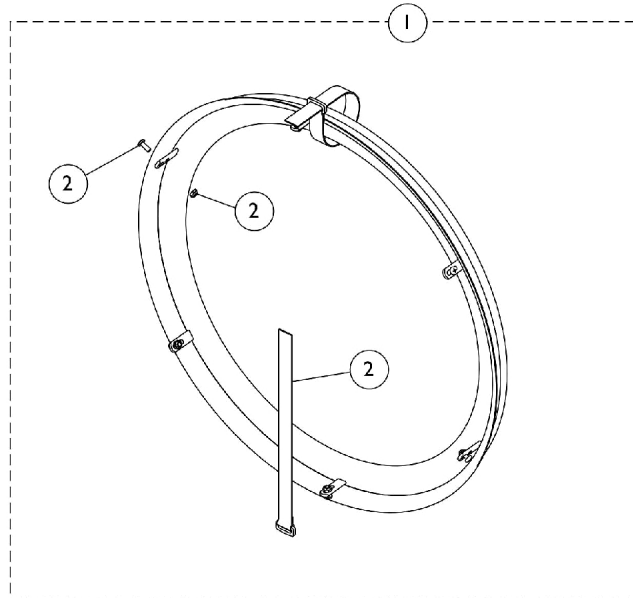

**6 Point Attaching Wheels (Spoke Wheels)**

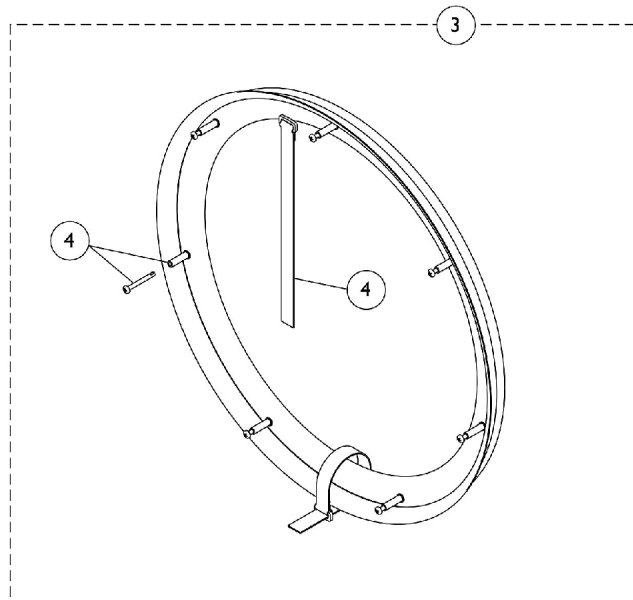

**7 Point Attaching Wheels (Composite Wheels)**

# Handrims and Mounting Hardware - Natural Fit and Natural Fit LT

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Natural Fit Handrims - 6 Point Attaching Wheels (Spoke Wheels)	1	1
1		See Chart	#1 Natural Fit LT Handrims - 6 Point Attaching Wheels (Spoke Wheels)	1	1
2	A	1145679	Kit, Handrim Attaching Hardware - 6 Point	1	1
			Screw, Phillips Pan Head (#10-32 x 1/2")	1	6
			Locknut (#10-32)	1	6
			NLA - Strap, Securing	1	2
3		See Chart	#3 Natural Fit Handrims - 7 Point Attaching Wheels (Composite Wheels)	1	1
3		See Chart	#3 Natural Fit LT Handrims - 7 Point Attaching Wheels (Composite Wheels)	1	1
4	B	1145680-NLA	Kit, Handrim Attaching Hardware - 7 Point	1	1
			Screw, Phillips Pan Head w/ Patch (#10-32 x 1-3/4")	1	7
			NLA - Spacer (3/16 x 5/16 x 1-1/16")	1	7
			NLA - Strap, Securing	1	2
	C	1144492	Instruction Sheet, Natural Fit Handrims	1	1

NOTE: A - Includes Screws, Locknuts, and Securing Straps.  
 B - Includes Screws, Spacers, and Securing Straps.  
 C - Not Shown

## #1 Natural Fit Handrims - 6 Point Attaching Wheels (Spoke Wheels)

Handrims sold as 1 each. Includes all Attaching Hardware (item 2) and Instruction Sheet.

Grip Style	24" Wheels	25" Wheels	26" Wheels
Standard	1145546-NLA	1145547	1145548-NLA
Super Grip	1145550-NLA	1145551	1145552

## #1 Natural Fit LT Handrims - 6 Point Attaching Wheels (Spoke Wheels)

Handrims sold as 1 each. Includes all Attaching Hardware (item 2) and Instruction Sheet.

Grip Style	24" Wheels	25" Wheels	26" Wheels
Standard	1163646	1163647-NLA	1163648-NLA
Super Grip	1163649	1163650-NLA	1163651

## #3 Natural Fit Handrims - 7 Point Attaching Wheels (Composite Wheels)

Handrims sold as 1 each. Includes all Attaching Hardware (item 4) and Instruction Sheet.

Grip Style	24" Wheels
Standard	1145553

# Handrims and Mounting Hardware - Natural Fit and Natural Fit LT

## #3 Natural Fit Handrims - 7 Point Attaching Wheels (Composite Wheels)

Handrims sold as 1 each. Includes all Attaching Hardware (item 4) and Instruction Sheet.

Grip Style	24" Wheels
Super Grip	1145554

## #3 Natural Fit LT Handrims - 7 Point Attaching Wheels (Composite Wheels)

Handrims sold as 1 each. Includes all Attaching Hardware (item 4) and Instruction Sheet.

Grip Style	24" Wheels
Standard	1163652-NLA
Super Grip	1163653