



MKIII Arrow Joystick and Hardware

Before 4/1/96

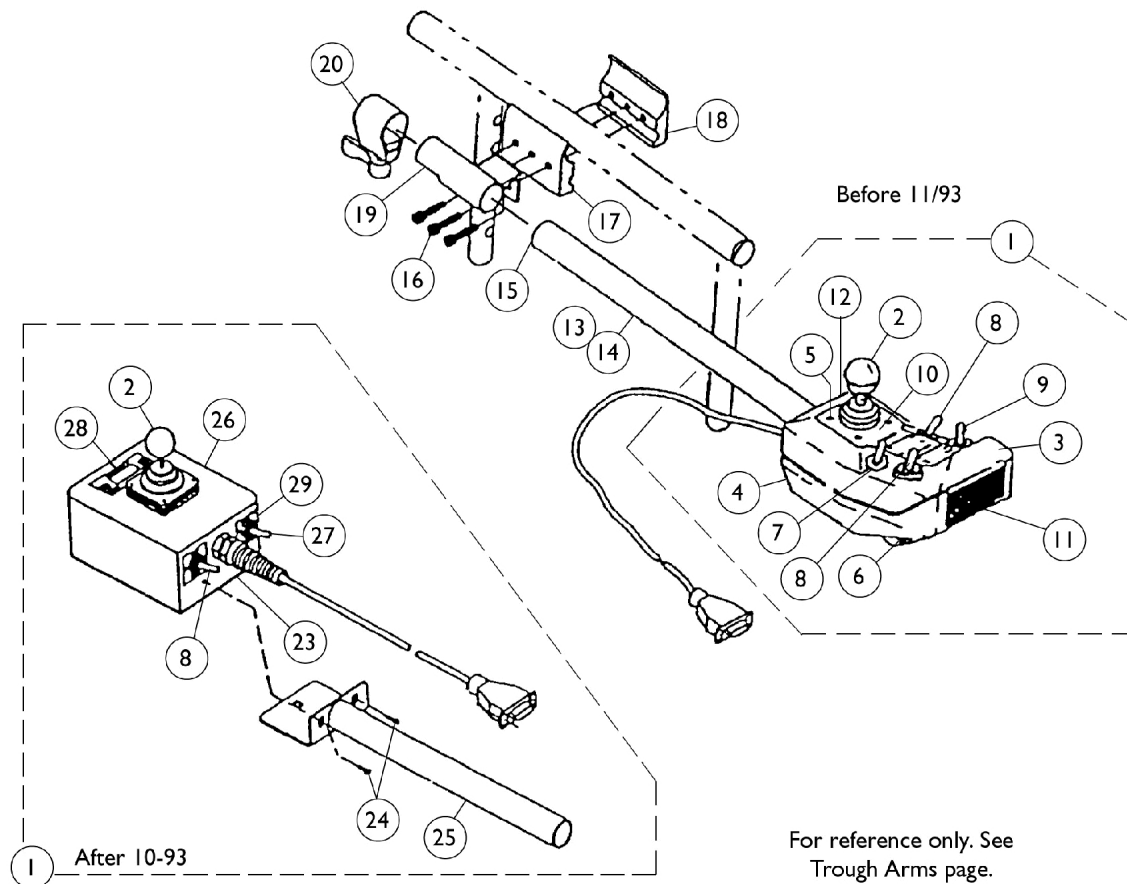

For reference only. See
Trough Arms page.

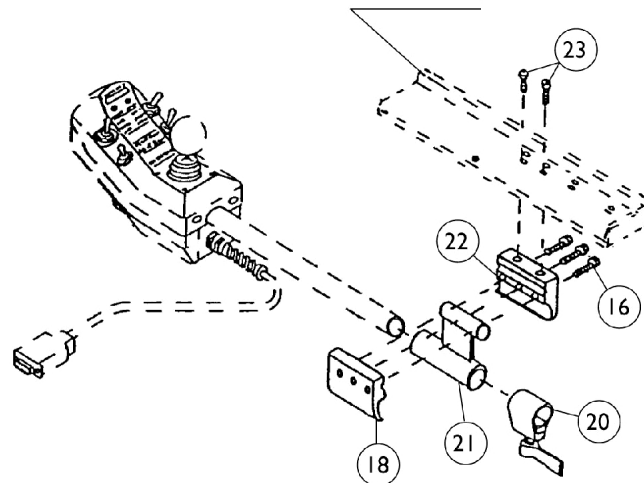

MKIII Arrow Joystick and Hardware

Before 4/1/96

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1031039-NLA	NLA - Joystick with Tube	1	1
1	A,C	1028750-NLA	NLA - Joystick with Tube	1	1
1	A,E	1035609-NLA	NLA - Joystick with Tube	1	1
2		1032697-NLA	NLA - Inductive, Joystick	1	1
2		1006191-NLA	NLA - Joystick Ball Knob w/ Tolerance Ring Assembly	1	1
2		50000X364-NLA	NLA - Boot, Joystick	1	1
2		1007309-NLA	NLA - Screw, Phillip Pan Head (#4-40 x 1")	1	4
2		1013235-NLA	NLA - Pad, Spacer, Inductive Joystick	1	1
3		1028872-NLA	NLA - Joystick Box Top, Grey	1	1
4		1028871-NLA	NLA - Joystick Box, Bottom, Grey	1	1
5		1020390-NLA	NLA - Screw, Phillips Flat Head (1 x 7/64")	1	4
6		1010051-NLA	NLA - Horn (8-16V)	1	1
7		1010358-NLA	NLA - Washer, Panel Seal (16/32 x 5/8 x 7/64")	1	4
8	G	1010042-NLA	NLA - Switch, Toggle SPDT	1	2
9	H	See Note	See Footnote	1	1
10		1028017-NLA	NLA - Decal, Joystick Face	1	1
11		1003502-NLA	NLA - Decal, Reflective	1	1
12		1005891-NLA	NLA - Plug Button, Switch Hole	1	1
13		1028877-NLA	NLA - Bracket, Control, Black	1	1
14		1028477-NLA	NLA - Screw, Self Tapping (#6 x 1/2")	1	4
15		1027095	Plug Button, Black (7/8")	1	1
16		1027268	Screw, Allen Head (1/4-20 x 1")	1	3
17		1027266	Half Clamp, Non-Threaded Holes	1	1
18		1027267-NLA	NLA - Half Clamp, Threaded Holes	1	1
19		1030890-NLA	NLA - Bracket, Joystick Mounting, Black	1	1
20		1027274	Adjustment Lock Assemnbly, Black	1	1
21	F	1029260-NLA	NLA - Bracket, Joystick Mounting, Black	1	1
22	F	1029256-NLA	NLA - Half Clamp, Non-Threaded Holes	1	1
23		11008X014-NLA	NLA - Screw, Phillips Round Head (#10-32 x 1")	1	2
24		1016401-NLA	NLA - Screw, Phillips Round Head (#10-32 x 3/8")	1	2
25		1030343-NLA	NLA - Joystick Bracket Assembly	1	1
26		1024064-NLA	NLA - Joystick Box, Top and Bottom	1	1
27		1028914-NLA	NLA - Switch, Hi / Lo / Off (Black)	1	1
28		1024151-NLA	NLA - Label, Battery	1	1
29		1024066-NLA	NLA - Label, Drive A - B	1	1
	D	1006271	Extension, Joystick, Straight Style (1561) (1/2" W x 5" L)	1	1
	D	1006270	Extension, Joystick, "T" Style (1560) (4" W x 5-1/2" L)	1	1

MKIII Arrow Joystick and Hardware

Before 4/1/96

NOTE: A - Service is repair only. Retrofittable with 1031039.
B - After 10/93
C - Before 11/93 - Grey Box
D - Not Shown
E - Before 11/91 - Black Box
F - Trough Arm
G - Black Switch is no longer available. The Chrome Switch part number 1010042 serves as a direct replacement
H - Center Return Switch is no longer available for aftermarket sales. Please contact Invacare Customer Service to obtain a return authorization number for estimate/repair.