
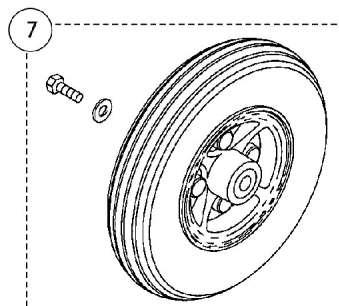
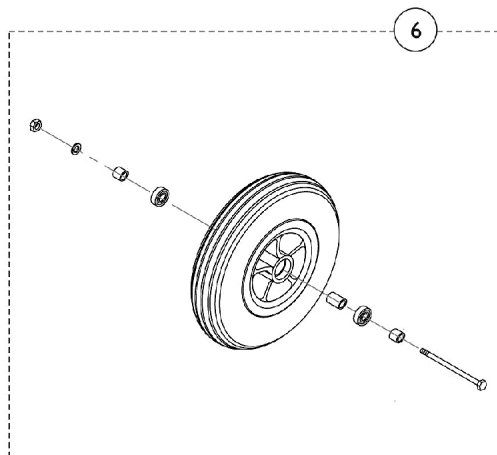
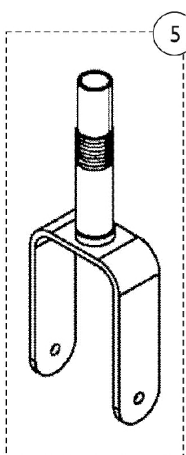
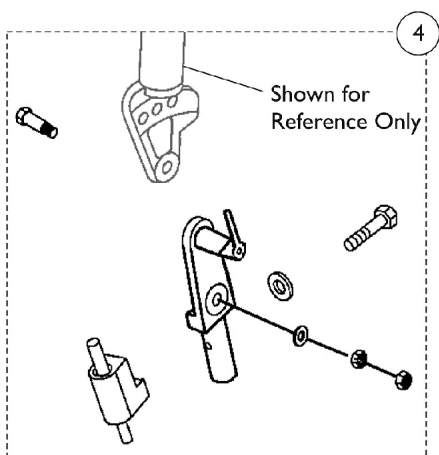
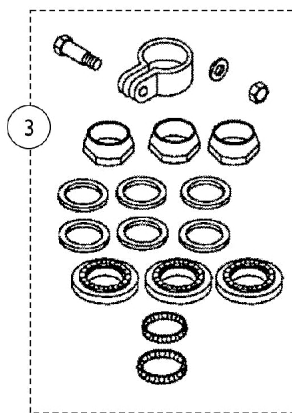
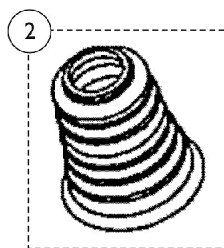


Lower Tiller Assembly & Front Wheels

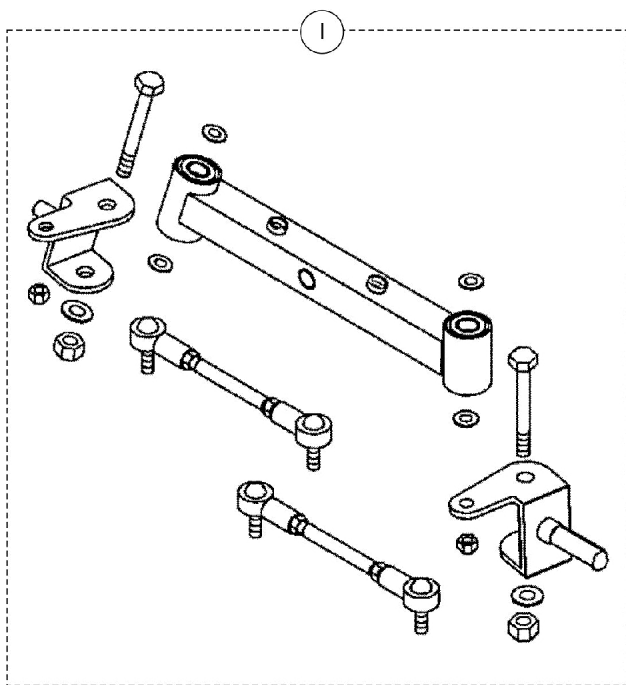

Lower Tiller Assembly & Front Wheels

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1144826	Kit, Front Steering (L-4) Includes: (1) ea. Swing Assembly, (4) ea. Nylon Washer, (1) ea. Axle - Right, (1) ea. Axle - Left, (2) ea. Bolts, (2) ea. Washers, (2) ea. Nylon Nuts, (2) ea. Rod Assemblies, and (2) ea. Nuts	1	1
2		1144806-NLA	NLA - Boot, Bellow, Tiller	1	1
3		1144803	Kit, Headset Bearing Includes: (1) ea. Tube Clamp with Hardware, (3) ea. Steering Nut, (3) ea. Plain Washer, (3) ea. Tooth Washer, (3) ea. Steering Race, and (2) ea. Bearings	1	1
4		1144804	Kit, Lower Hardware (L-3) Includes: (1) ea. Guide, (1) ea. Washer, (1) ea. Bolt, (1) ea. Bolt, (2) ea. Nylon Nut, (1) ea. Washer, and (1) ea. Steering Adjustment Bracket	1	1
5		1144805	Fork, Front (L-3)	1	1
6		1144822	Kit, Front Wheel (L-3) Includes: (1) ea. Nylon Nut, (1) ea. Plain Washer, (2) ea. Spacer, (1) ea. Bolt, and (1) ea. Front Wheel	1	1
7		1144832-NLA	NLA - Kit, Front Wheel (L-4) Includes: (1) ea. Bolt, (1) ea. Plain Washer, and (1) ea. Front Wheel	1	1