


MCCX Joystick and Controller

Before 4/1/96

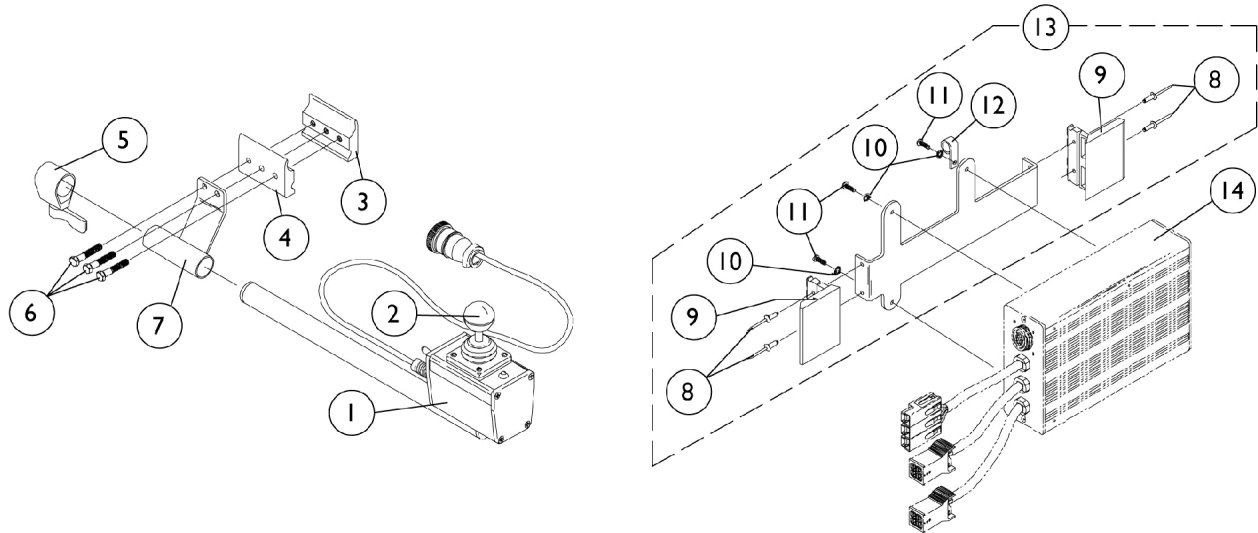

MCCX Joystick and Controller Before 4/1/96

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1027409-NLA	NLA - Joystick with Tube Assembly	1	1
1	B	1006715-NLA	NLA - Joystick with Tube	1	1
2		1032696-NLA	NLA - Joystick, Inductive	1	1
2		1006191-NLA	NLA - Joystick Ball Knob w/ Tolerance Ring Assembly	1	1
2		50000X364-NLA	NLA - Boot, Joystick	1	1
2		1028079-NLA	NLA - Screw, Phillips Pan Head (#4-40 x 5/8")	1	4
3		1027267-NLA	NLA - Half Clamp, Threaded Holes	1	1
4		1027266	Half Clamp, Non-Threaded Holes	1	1
5		1027274	Adjustment Lock Assemnbly, Black	1	1
6		1027268	Screw, Allen Head (1/4-20 x 1")	1	3
7		1027252-NLA	NLA - Bracket, Joystick Mounting, Black	1	1
8		16348X004-NLA	NLA - Rivet	1	4
9	G	50000X025-NLA	NLA - Clip, Lock Over Latch	1	2
10		1027263-NLA	NLA - Washer, Lock Internal Tooth (#10)	1	3
11		1027264-NLA	NLA - Screw, Phillips Pan Head (#8-32 x 1/2")	1	3
12		50000X403-NLA	NLA - Clamp, Black Plastic	1	1
13		1027238-NLA	NLA - Bracket without Hardware, Controller	1	1
13		1037178-NLA	NLA - Bracket with Hardware, Controller	1	1
14	C	1036494-NLA	NLA - Controller without Bracket	1	1
14	D,F	1026451-NLA	NLA - Controller without Bracket	1	1
	E	1006271	Extension, Joystick, Straight Style (1561) (1/2" W x 5" L)	1	1
	E	1006270	Extension, Joystick, "T" Style (1560) (4" W x 5-1/2" L)	1	1
	E	1036494E-NLA	NLA - Controller without Bracket, MCCX	1	1
NOTE: A - After 11/91 B - Before 12/91- Repair Only C - After 10/92 D - Before 11/92 E - Not Shown F - Repair only or use 1036494E reconditioned G - For Keeper					