


Adjustable Back Width Rigidizing Bar (ADJRGD)

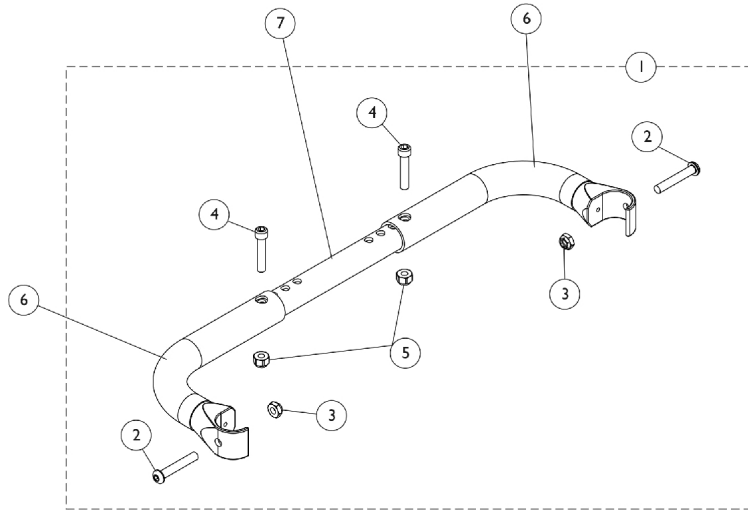

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1127793	Kit, Rigidizing Bar Hardware	1	1
2			Screw, Socket Button Head (1/4-28 x 1-1/2")	1	2
3			Locknut (1/4-28)	1	2
4			Screw, Socket Head (1/4-28 x 1-1/4")	1	2
5			Locknut (1/4-28)	1	2
6	B	1127792	Handle, Rigidizing Bar, Black - Right/Left	1	2
7		See Chart	#7 Crossbrace Tube	1	1

NOTE: A - Includes items 2-5
B - Item #6 accommodates the right or left side

#7 Crossbrace Tube

12" - 16" Wide	14" - 18" Wide	16" - 20" Wide	18" - 22" Wide	20" - 24" Wide
1129473	1129474	1129475	1129476	1159455