


Hydraulic Pump Model - 9099 and 9099P

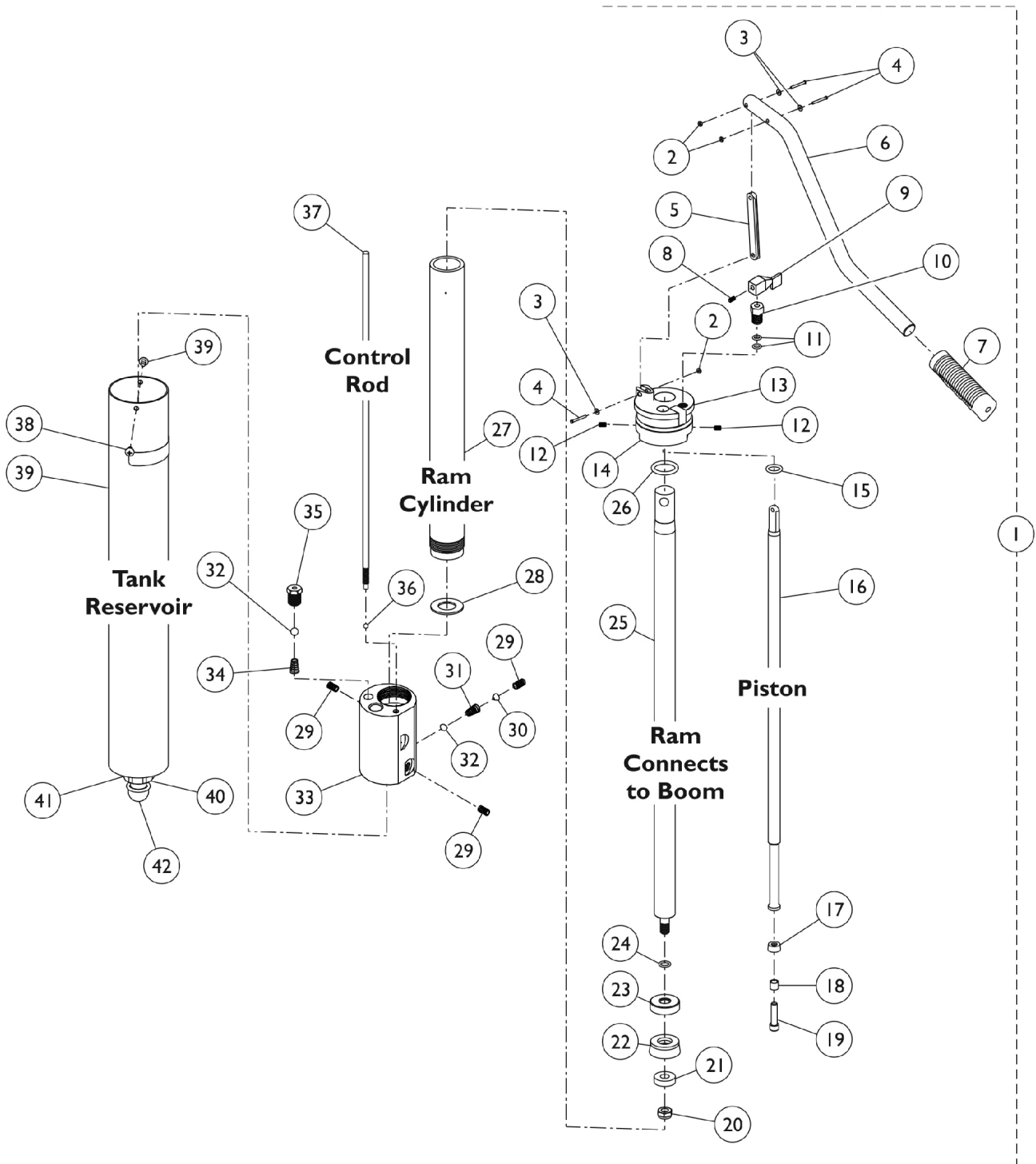

Hydraulic Pump

Model - 9099 and 9099P

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	9099-NLA	NLA - Pump	1	1
1	A	9099P	Pump, Hydraulic	1	1
1	A	9099E-NLA	NLA - Pump, Reconditioned, Chrome	1	1
1	A	9099R-NLA	NLA - Pump, Hydraulic	1	1
1	A	9099RE-NLA	NLA - Pump, Reconditioned, Reliant, Isolet Grey	1	1
	B,D	1011941-NLA	NLA - Kit, Pump Handle Hardware Upgrade	1	1
2			Nut, Hex (#6-32)	1	3
3			Washer, Flat (11/64 x 3/8 x 1/32")	1	3
4			T-Bolt, Threaded End (6-32 x 25/32)	1	3
			Instruction Sheet	1	1
5		90013X000-NLA	NLA - Link	1	1
6		90012X000-NLA	NLA - Handle, Pump, Chrome	1	1
7		90007X000	NLA - Grip, Handle - 9805	1	1
7		1065926-NLA	NLA - Grip, Hand, Black	1	1
8		60284X007	Screw, Socket Set Flat Point (1/4-20 x 3/8")	1	1
9		90015X000-NLA	NLA - Handle, Control, Chrome	1	1
10	C	See Note	See Footnote	1	1
11	C	See Note	See Footnote	1	2
12		90032X000	Screw, Set Socket Cup Point (#8-32 x 1/4")	1	2
13	C	See Note	See Footnote	1	1
14		90024X000-NLA	NLA - O-Ring, Tank (2-7/64 x 2-3/8 x 1/8")	1	1
15		90023X000-NLA	NLA - O-Ring, Piston (1/2 x 11/16 x 7/64")	1	1
16		90033X000-NLA	NLA - Piston	1	1
17		90034X000-NLA	NLA - Cup, Piston	1	1
18	C	See Note	See Footnote	1	1
19		90036X000-NLA	NLA - Screw, Allen Head (1/4-28 x 1")	1	1
20		40363X000	Hex Nut (3/8"-16)	1	1
21		90028X000-NLA	NLA - Washer, Inside Ram (3/8" x 57/64" x 1/4")	1	1
22		90026X000-NLA	NLA - Cup, Ram	1	1
23		90029X001-NLA	NLA - Washer, Outside Ram (3/8" x 1-1/4" x 11/32")	1	1
24	C	See Note	See Footnote	1	1
25		90052X000-NLA	NLA - Ram	1	1
26		90022X000-NLA	NLA - O-Ring, Ram (7/8 x 1-1/8 x 1/8")	1	1
27		90031X000-NLA	NLA - Cylinder, Ram	1	1
28		90135X000-NLA	NLA - Washer, Nylon (3/4" x 1/34/65" x 1/16")	1	1
29		90043X000	NLA - Plug, Pressure (1/8")	1	3
30		90053X000	NLA - Tube, Bonnet	1	1
31		90045X000-NLA	NLA - Spring, Helical Compression	1	1
32		90040X000	NLA - Ball, Steel (5/16")	1	2
33		90037X000-NLA	NLA - Body, Pump	1	1
34		90041X000	NLA - Spring, Helical Compression	1	1
35		90039X000-NLA	NLA - Inlet Fitting	1	1
36		90042X000-NLA	NLA - Ball, Steel (3/16")	1	1
37	C	See Note	See Footnote	1	1
38		90017X000-NLA	NLA - Screw, Phillips Pan Head (#10-32 x 5/16")	1	2
39		90008X000-NLA	NLA - Reservoir, Tank - Chrome	1	1

Hydraulic Pump

Model - 9099 and 9099P

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
39		1065927-NLA	NLA - Reservoir, Tank, Isolet Grey	1	1
40		90009X000-NLA	NLA - Nut, EMT (1-1/8-18 x 5/16")	1	1
41		90011X000-NLA	NLA - Screw, 9099 Pump Only (#8-32 x 1/8")	1	1
42		91100X238-NLA	NLA - Ring, Retaining	1	1
	D	90046X000	NLA - Oil, Hydraulic (16 oz)	1	1
NOTE: A - Includes items 2 - 41. B - Kit Includes Items 2, 3, and 4 and Instruction Sheet C - Items is no longer serviceable in the field. D - Not shown.					