


9000XT Recliner Removable Conventional Adjustable Height Arms

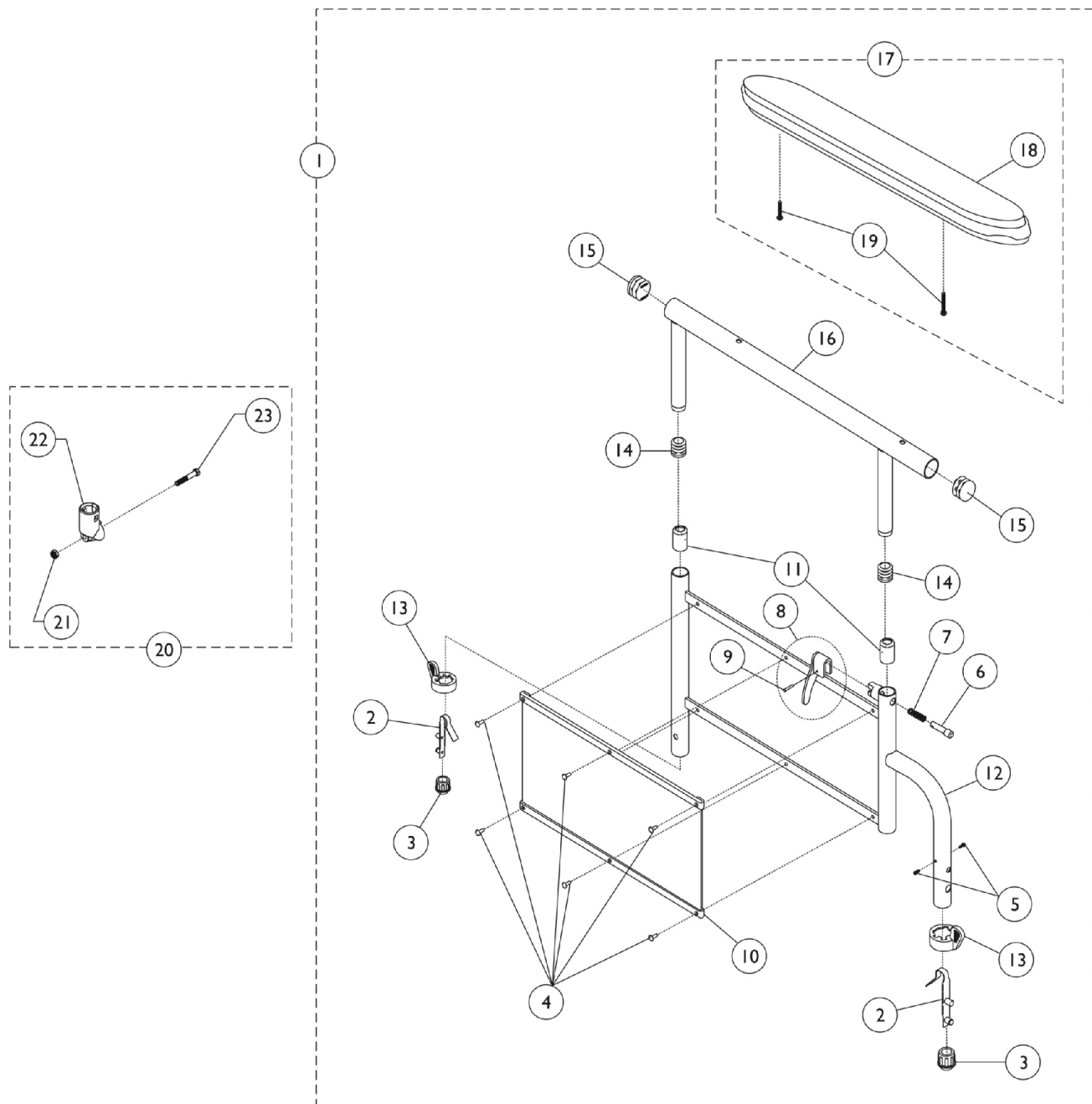

9000XT Recliner Removable Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,D,H	8881135530	Kit, Armrest, Desk Length, Color - Right (Style 29)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	1
3			Plug, Armrest, Black (3/4") (20GA)	1	1
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Right	1	1
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Anti-Rattle	1	2
12			Arm Base, Color	1	1
13			Handle, Quad Release	1	2
14			Plug Button, Black (7/8")	1	2
15			Plug Button, Black (5/8")	1	2
16			Tube, Armrest, Desk Length, Black	1	1
18			Arm Pad, Desk Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
4	A,D,H	1028645	Rivet	1	6
10		1064129	Guard, Clothing	1	1
1		8881135531	Kit, Armrest, Desk Length, Color - Left (Style 29)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	1
3			Plug, Armrest, Black (3/4") (20GA)	1	2
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Left	1	1
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Anti-Rattle	1	2
12			Arm Base, Color	1	1
13			Handle, Quad Release	1	2
14			Plug Button, Black (7/8")	1	2
15			Plug Button, Black (5/8")	1	2
16			Tube, Armrest, Desk Length, Black	1	1
18			Arm Pad, Desk Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
4	A,G,H	1028645	Rivet	1	6
10		1064129	Guard, Clothing	1	1
1		8881135532-NL A	NLA - Kit, Armrest, Desk Length, Chrome - Right (Style 29)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	1

9000XT Recliner Removable Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
3			Plug, Armrest, Black (3/4") (20GA)	1	2
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Right	1	1
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Anti-Rattle	1	2
12			Arm Base, Chrome	1	1
13			Handle, Quad Release	1	2
14			Plug Button, Black (5/8")	1	2
15			Plug Button, Black (7/8")	1	2
16			Tube, Arm, Desk Length, Black	1	1
18			Arm Pad, Desk Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
4		1028645	Rivet	1	6
10		1064129	Guard, Clothing	1	1
1	A,G,H	8881135533-NL A	NLA - Kit, Armrest, Desk Length, Chrome - Left (Style 29)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	1
3			Plug, Armrest, Black (3/4") (20GA)	1	1
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Left	1	1
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Anti-Rattle	1	2
12			Arm Base, Chrome	1	1
13			Handle, Quad Release	1	2
14			Plug Button, Black (5/8")	1	2
15			Plug Button, Black (7/8")	1	2
16			Tube, Arm, Desk Length, Black	1	1
18			Arm Pad, Desk Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
4		1028645	Rivet	1	6
10		1064129	Guard, Clothing	1	1
1	A,D,H	8881137321	Kit, Armrest, Full Length, Color - Right (Style 19)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	1
3			Plug, Armrest, Black (3/4") (20GA)	1	1
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2

9000XT Recliner Removable Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Right	1	1
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Anti-Rattle	1	2
12			Arm Base, Color	1	1
13			Handle, Quad Release	1	2
14			Plug Button, Black (7/8")	1	2
15			Plug Button, Black (5/8")	1	2
16			Tube, Armrest, Full Length, Black	1	1
18			NLA - Pad, Arm, Full Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
4		1028645	Rivet	1	6
10		1064129	Guard, Clothing	1	1
1	A,D,H	8881137322	Kit, Armrest, Full Length, Color - Left (Style 19)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	1
3			Plug, Armrest, Black (3/4") (20GA)	1	2
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Left	1	1
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Anti-Rattle	1	2
12			Arm Base, Color	1	1
13			Handle, Quad Release	1	2
14			Plug Button, Black (7/8")	1	2
15			Plug Button, Black (5/8")	1	2
16			Tube, Armrest, Full Length, Black	1	1
18			NLA - Pad, Arm, Full Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
4		1028645	Rivet	1	6
10		1064129	Guard, Clothing	1	1
1	A,G,H	8881135534-NL A	NLA - Kit, Armrest, Full Length, Chrome - Right (Style 19)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	1
3			Plug, Armrest, Black (3/4") (20GA)	1	1
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Right	1	1

9000XT Recliner Removable Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Anti-Rattle	1	2
12			Arm Base, Chrome	1	1
13			Handle, Quad Release	1	2
14			Plug Button, Black (7/8")	1	2
15			Plug Button, Black (5/8")	1	2
16			Tube, Arm, Full Length, Black	1	1
18			NLA - Pad, Arm, Full Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
4		1028645	Rivet	1	6
10		1064129	Guard, Clothing	1	1
1	A,G,H	8881135535-NL A	NLA - Kit, Armrest, Full Length, Chrome - Left (Style 19)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	1
3			Plug, Armrest, Black (3/4") (20GA)	1	1
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Left	1	1
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Anti-Rattle	1	2
12			Arm Base, Chrome	1	1
13			Handle, Quad Release	1	2
14			Plug Button, Black (7/8")	1	2
15			Plug Button, Black (5/8")	1	2
16			Tube, Arm, Full Length, Black	1	1
18			NLA - Pad, Arm, Full Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
4		1028645	Rivet	1	6
10		1064129	Guard, Clothing	1	1
17	B,G	8881127509	Kit, Arm Pad w/Screws - Full Length	1	1
18			NLA - Pad, Arm, Full Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
17	B,G	8881127510	Kit, Arm Pad w/Screws - Desk Length	1	1
18			Arm Pad, Desk Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
20	C	1058139	Kit, Armrest Hardware	1	1
21			Locknut (1/4-20)	1	1
22			Cup, Socket Pivot	1	1
23			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	1
	E,F	7100053	Arm Lock Assembly, Black - Right	1	1
			Handle, Lock, Black - Right	1	1

9000XT Recliner Removable Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	E,F	7100054	Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
			Arm Lock Assembly, Black - Left	1	1
			Handle, Lock, Black - Left	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
NOTE: A - Includes items 2-19. B - Includes items 18 and 19. C - Includes items 21-23. D - Specify frame finish and upholstery color when ordering, see category titled, "Frame Finishes, Upholstery Colors, Decals and Packaging." E - Not shown. F - Includes items 6-9. G - Specify upholstery color when ordering, see category titled, "Frame Finishes, Upholstery Colors, Decals and Packaging." H - Item #3 Plug Button for 18GA Tubing used in front only.					