


9000XT, 9000XDT & 9000XT Recliner Non-Removable Conventional Adjustable Height Arms

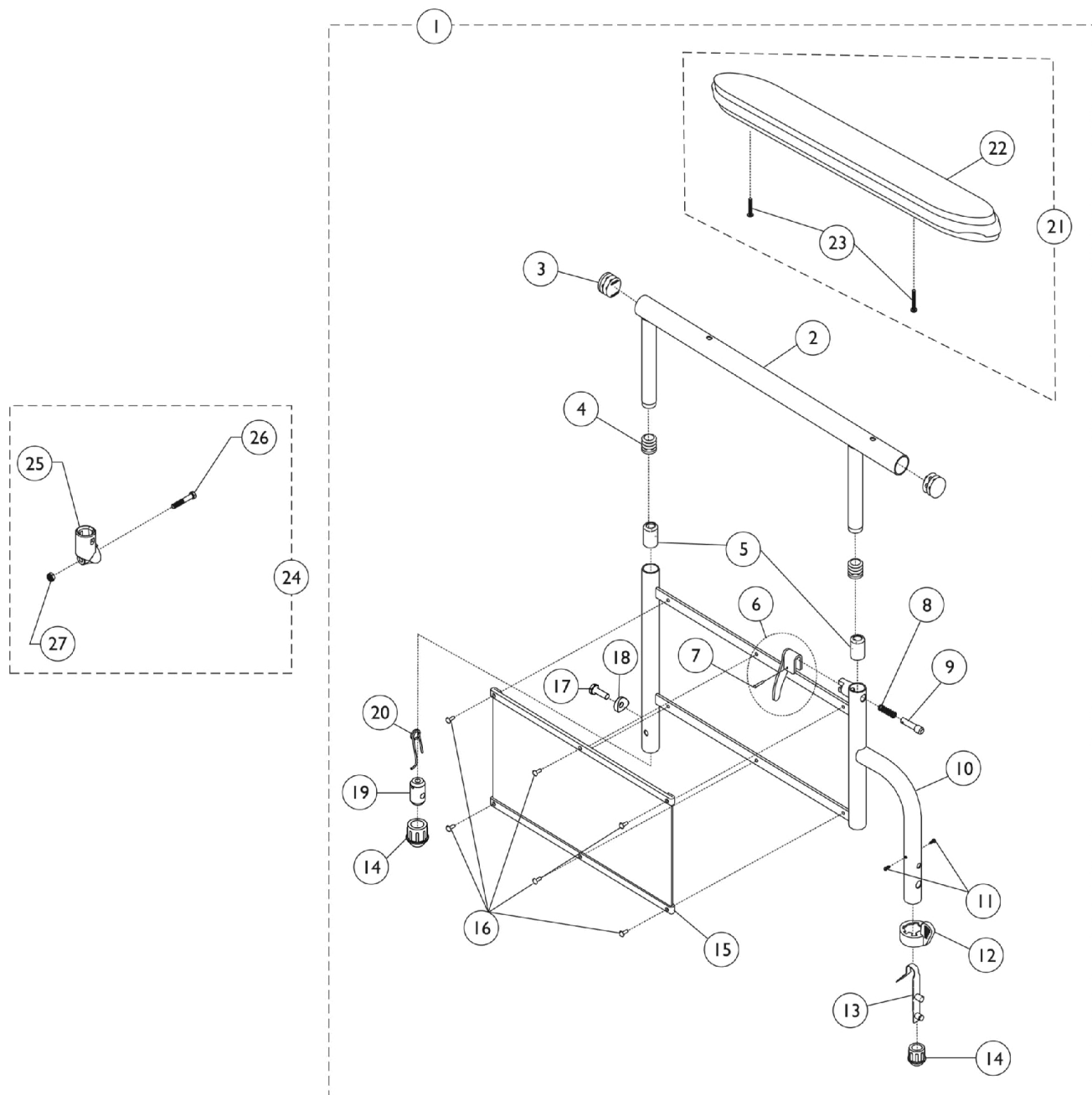

9000XT, 9000XDT & 9000XT Recliner Non-Removable Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,I	8881130803	Kit, Armrest, Desk Length, Color - Right (Style 29)	1	1
2			Tube, Armrest, Desk Length, Black	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	2
5			Anti-Rattle	1	2
6			Handle, Lock, Black - Right	1	1
7			Rivet, Tubular	1	1
8			Spring, Arm Lock	1	1
9			Pin, Locking	1	1
10			Arm Base, Color	1	1
11			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
12			Handle, Quad Release	1	1
13			Spring, Torsion with Button Lock	1	1
14			Plug, Armrest, Black (3/4") (18GA)	1	1
14			Plug, Armrest, Black (3/4") (20GA)	1	1
15			Guard, Clothing	1	1
16			Rivet	1	6
17			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
18			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
19			Insert, Threaded	1	1
20			Spring, Release	1	1
22			Arm Pad, Desk Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
15	A,B,I	1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1		8881130804	Kit, Armrest, Desk Length, Color - Left (Style 29)	1	1
2			Tube, Armrest, Desk Length, Black	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	2
5			Anti-Rattle	1	2
6			Handle, Lock, Black - Left	1	1
7			Rivet, Tubular	1	1
8			Spring, Arm Lock	1	1
9			Pin, Locking	1	1
10			Arm Base, Color	1	1
11			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
12			Handle, Quad Release	1	1
13			Spring, Torsion with Button Lock	1	1
14			Plug, Armrest, Black (3/4") (18GA)	1	1
14			Plug, Armrest, Black (3/4") (20GA)	1	1
15			Guard, Clothing	1	1
16			Rivet	1	6
17			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
18			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
19			Insert, Threaded	1	1
20			Spring, Release	1	1

9000XT, 9000XDT & 9000XT Recliner Non-Removable Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
22			Arm Pad, Desk Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
15		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,H,I	8881130805-NL A	NLA - Kit, Armrest, Desk Length, Chrome - Right (Style 29)	1	1
2			Tube, Arm, Desk Length, Black	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	2
5			Anti-Rattle	1	2
6			Handle, Lock, Black - Right	1	1
7			Rivet, Tubular	1	1
8			Spring, Arm Lock	1	1
9			Pin, Locking	1	1
10			Arm Base, Chrome	1	1
11			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
12			Handle, Quad Release	1	1
13			Spring, Torsion with Button Lock	1	1
14			Plug, Armrest, Black (3/4") (18GA)	1	1
14			Plug, Armrest, Black (3/4") (20GA)	1	1
15			Guard, Clothing	1	1
16			Rivet	1	6
17			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
18			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
19			Insert, Threaded	1	1
20			Spring, Release	1	1
22			Arm Pad, Desk Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
15		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,H,I	8881130806-NL A	NLA - Kit, Armrest, Desk Length, Chrome - Left (Style 29)	1	1
2			Tube, Arm, Desk Length, Black	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	2
5			Anti-Rattle	1	2
6			Handle, Lock, Black - Left	1	1
7			Rivet, Tubular	1	1
8			Spring, Arm Lock	1	1
9			Pin, Locking	1	1
10			Arm Base, Chrome	1	1
11			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
12			Handle, Quad Release	1	1
13			Spring, Torsion with Button Lock	1	1
14			Plug, Armrest, Black (3/4") (18GA)	1	1
14			Plug, Armrest, Black (3/4") (20GA)	1	1

9000XT, 9000XDT & 9000XT Recliner Non-Removable Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
15			Guard, Clothing	1	1
16			Rivet	1	6
17			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
18			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
19			Insert, Threaded	1	1
20			Spring, Release	1	1
22			Arm Pad, Desk Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
15		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,B	8881135526	Kit, Armrest, Full Length, Color - Right (Style 19)	1	1
2			Tube, Armrest, Full Length, Black	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	2
5			Anti-Rattle	1	2
6			Handle, Lock, Black - Right	1	1
7			Rivet, Tubular	1	1
8			Spring, Arm Lock	1	1
9			Pin, Locking	1	1
10			Arm Base, Color	1	1
11			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
12			Handle, Quad Release	1	1
13			Spring, Torsion with Button Lock	1	1
14			Plug, Armrest, Black (3/4") (18GA)	1	1
14			Plug, Armrest, Black (3/4") (20GA)	1	1
15			Guard, Clothing	1	1
16			Rivet	1	6
17			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
18			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
19			Insert, Threaded	1	1
20			Spring, Release	1	1
22			NLA - Pad, Arm, Full Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
15		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,B,I	8881135527	Kit, Armrest, Full Length, Color - Left (Style 19)	1	1
2			Tube, Armrest, Full Length, Black	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	2
5			Anti-Rattle	1	2
6			Handle, Lock, Black - Left	1	1
7			Rivet, Tubular	1	1
8			Spring, Arm Lock	1	1
9			Pin, Locking	1	1
10			Arm Base, Color	1	1
11			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2

9000XT, 9000XDT & 9000XT Recliner Non-Removable Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
12			Handle, Quad Release	1	1
13			Spring, Torsion with Button Lock	1	1
14			Plug, Armrest, Black (3/4") (18GA)	1	1
14			Plug, Armrest, Black (3/4") (20GA)	1	1
15			Guard, Clothing	1	1
16			Rivet	1	6
17			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
18			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
19			Insert, Threaded	1	1
20			Spring, Release	1	1
22			NLA - Pad, Arm, Full Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
15		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,H,I	8881135528-NL A	NLA - Kit, Armrest, Full Length, Chrome - Right (Style 19)	1	1
2			Tube, Arm, Full Length, Black	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	2
5			Anti-Rattle	1	2
6			Handle, Lock, Black - Right	1	1
7			Rivet, Tubular	1	1
8			Spring, Arm Lock	1	1
9			Pin, Locking	1	1
10			Arm Base, Chrome	1	1
11			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
12			Handle, Quad Release	1	1
13			Spring, Torsion with Button Lock	1	1
14			Plug, Armrest, Black (3/4") (18GA)	1	1
14			Plug, Armrest, Black (3/4") (20GA)	1	1
15			Guard, Clothing	1	1
16			Rivet	1	6
17			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
18			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
19			Insert, Threaded	1	1
20			Spring, Release	1	1
22			NLA - Pad, Arm, Full Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
15		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,H,I	8881135529-NL A	NLA - Kit, Armrest, Full Length, Chrome - Left (Style 19)	1	1
2			Tube, Arm, Full Length, Black	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	2
5			Anti-Rattle	1	2

9000XT, 9000XDT & 9000XT Recliner Non-Removable Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
6			Handle, Lock, Black - Left	1	1
7			Rivet, Tubular	1	1
8			Spring, Arm Lock	1	1
9			Pin, Locking	1	1
10			Arm Base, Chrome	1	1
11			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
12			Handle, Quad Release	1	1
13			Spring, Torsion with Button Lock	1	1
14			Plug, Armrest, Black (3/4") (18GA)	1	1
14			Plug, Armrest, Black (3/4") (20GA)	1	1
15			Guard, Clothing	1	1
16			Rivet	1	6
17			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
18			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
19			Insert, Threaded	1	1
20			Spring, Release	1	1
22			NLA - Pad, Arm, Full Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
15		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
21	C,H	8881127509	Kit, Arm Pad w/Screws - Full Length	1	1
22			NLA - Pad, Arm, Full Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
21	C,H	8881127510	Kit, Arm Pad w/Screws - Desk Length	1	1
22			Arm Pad, Desk Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
24	D	1058139	Kit, Armrest Hardware	1	1
25			Cup, Socket Pivot	1	1
26			Locknut (1/4-20)	1	1
27			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	1
	G	7100053	Arm Lock Assembly, Black - Right	1	1
			Handle, Lock, Black - Right	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
	G	7100054	Arm Lock Assembly, Black - Left	1	1
			Handle, Lock, Black - Left	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2

9000XT, 9000XDT & 9000XT Recliner Non-Removable Conventional Adjustable Height Arms

NOTE: A - Includes items 2-23.
B - Specify frame finish and upholstery color when ordering, see category titled, "Frame Finishes, Upholstery Colors, Decals and Packaging."
C - Includes items 22 and 23.
D - Includes items 25-27.
G - Not Shown. Includes items 6-9.
H - Specify upholstery color when ordering, see category titled, "Frame Finishes, Upholstery Colors, Decals and Packaging."
I - Item #14 Plug Button for 18GA Tubing used in front only.