


9000XT Space Saver Adjustable Height Arms - 79JY & 89JY

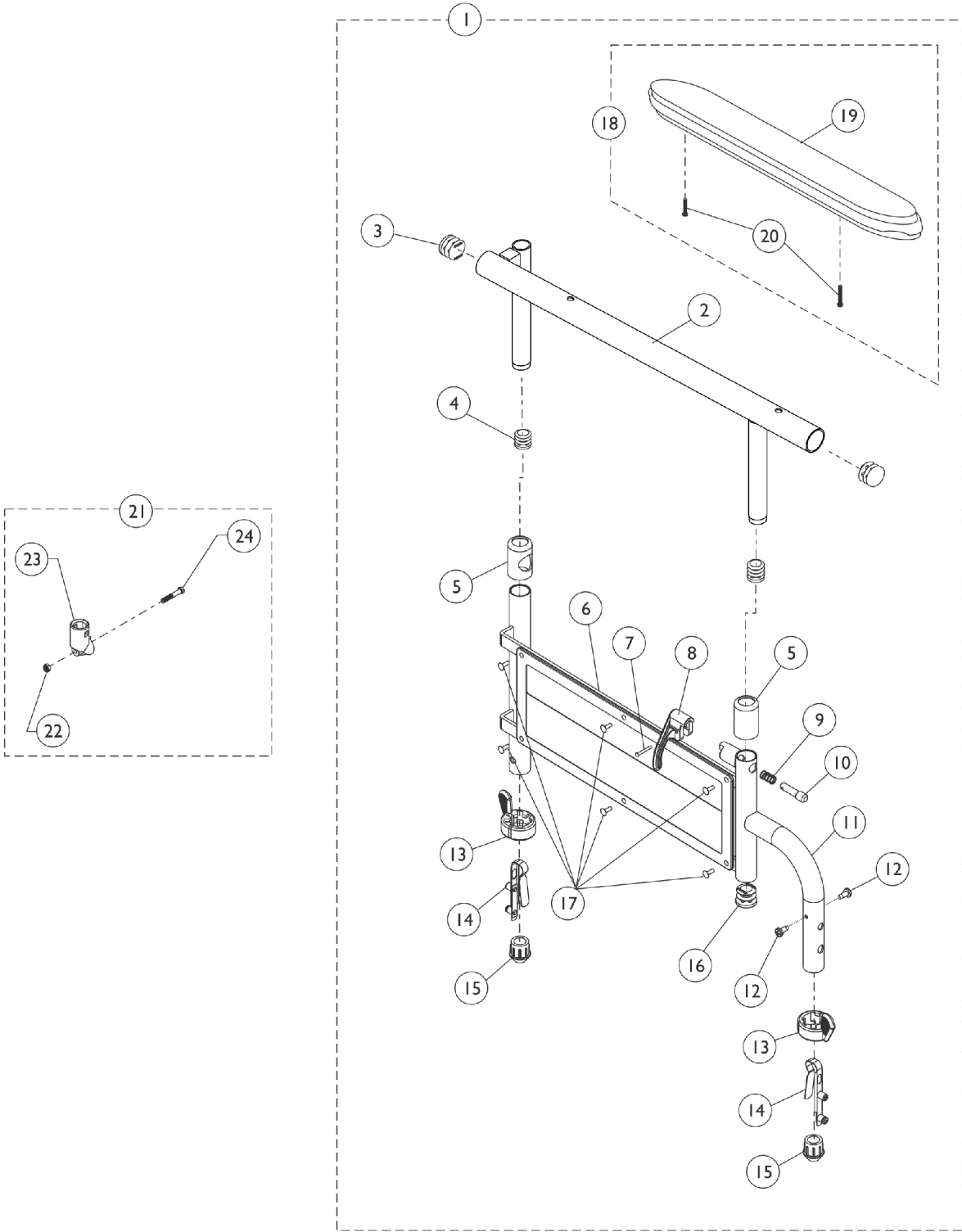

9000XT Space Saver Adjustable Height Arms - 79JY & 89JY

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,E	8881130862	Kit, Armrest, Desk Length, Black - Right (Style 89JY)	1	1
2			Tube, Armrest, Desk, Black - Right	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Right	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Tube, Armrest Base, Adjustable Height, Black -Right	1	1
12			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	2
16			Plug Button, Black (3/4")	1	1
17			Rivet	1	6
19			Arm Pad, Desk Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
6		1068511	Guard, Clothing	1	1
17		1028645	Rivet	1	6
1	A,E	8881130863	Kit, Armrest, Desk Length, Black - Left (Style 89JY)	1	1
2			Tube, Armrest, Desk, Black - Left	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Left	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Tube, Armrest Base, Adjustable Height, Black - Left	1	1
12			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	2
16			Plug Button, Black (3/4")	1	1
17			Rivet	1	6
19			Arm Pad, Desk Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
6		1068511	Guard, Clothing	1	1
17		1028645	Rivet	1	6
1	A,E	8881130874	Kit, Armrest, Full Length, Black - Right (Style 79JY)	1	1
2			Tube, Armrest, Full Length, Black - Right	1	1

9000XT Space Saver Adjustable Height Arms - 79JY & 89JY

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Right	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Tube, Armrest Base, Adjustable Height, Black -Right	1	1
12			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	2
16			Plug Button, Black (3/4")	1	1
17			Rivet	1	6
19			NLA - Pad, Arm, Full Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
6		1068511	Guard, Clothing	1	1
17		1028645	Rivet	1	6
1	A,E	8881130875	Kit, Armrest, Full Length, Black - Left (Style 79JY)	1	1
2			Tube, Armrest, Full Length, Black - Left	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Left	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Tube, Armrest Base, Adjustable Height, Black - Left	1	1
12			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	2
16			Plug Button, Black (3/4")	1	1
17			Rivet	1	6
19			NLA - Pad, Arm, Full Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
6		1068511	Guard, Clothing	1	1
17		1028645	Rivet	1	6
18	B,E	8881127510	Kit, Arm Pad w/Screws - Desk Length	1	1
19			Arm Pad, Desk Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
18	B,E	8881127509	Kit, Arm Pad w/Screws - Full Length	1	1
19			NLA - Pad, Arm, Full Length w/ Law Label	1	1

9000XT Space Saver Adjustable Height Arms - 79JY & 89JY

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
21	C	1058139	Kit, Armrest Hardware	1	1
22			Cup, Socket Pivot	1	1
23			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	1
24			Locknut (1/4-20)	1	1
	D	7100053	Arm Lock Assembly, Black - Right	1	1
			Handle, Lock, Black - Right	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
	D	7100054	Arm Lock Assembly, Black - Left	1	1
			Handle, Lock, Black - Left	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
NOTE: A - Includes items 2-20. B - Includes items 19 and 20. C - Includes items 22-24. D - Not Shown. Includes items 7-10. E - Specify upholstery color when ordering. See page titled "Frame Finishes, Upholstery Colors, Decals and Packaging."					